
Surgical Face Mask


EN 14683:2019+AC:2019 **TYPE IIR**

PRODUCT INFO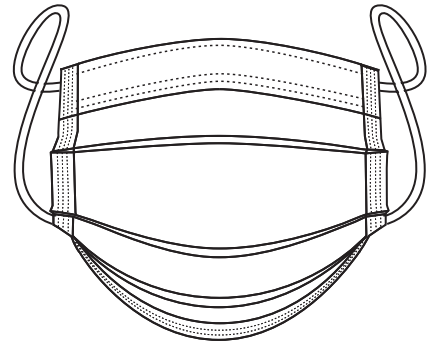

- SE** Kirurgiskt Munskydd
- FR** Masque Chirurgical
- DE** Chirurgische Maske
- ES** Màscara Quirùrgica
- IT** Maschera Chirurgica
- NO** Kirurgisk Munnbind
- DK** Kirurgisk Mundbind
- FI** Kirurginen Suusuoja
- NL** Chirurgisch Masker

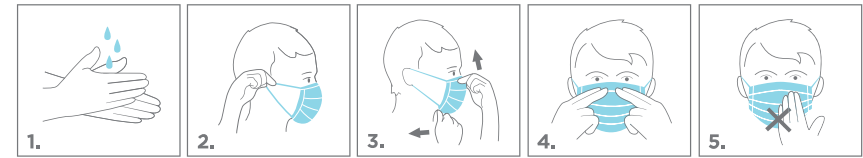GIBSONMEDICAL.SE

Imported by Gibson Medical AB
Ideon Science Park SE-223 70 Lund

EN Manufacturer **SV** Tillverkare **FR** Fabricant
DE Hersteller **ES** Fabricante **IT** Produttore **NO**
Produsent **DK** Producent **FI** Valmistaja **NL** Fabrikant

EN Date of manufacture **SV** Tillverkningsdatum
FR Date de fabrication **DE** Herstellungsdatum
ES Fecha de fabricación **IT** Data di produzione
NO Produksjonsdato **DK** Fremstillingsdato **FI**
Valmistuspäivä **NL** Productiedatum

EN Use-before date **SV** Sista förbrukningsdag
FR Date limite d'utilisation **DE** Verfallsdatum
ES Fecha de caducidad **IT** Data di scadenza
NO Utløpsdato **DK** Sidste anvendelsesdato **FI**
Viimeinen käyttöpäivä **NL** Houdbaarheidsdatum

EN Batch code **SV** Batchkod **FR** Code de lot
DE Chargennummer **ES** Número de lote
IT Codice lotto **NO** Batch-kode **DK** Batchkode
FI Eräkoodi **NL** Partijcode

EN Article number **SV** Artikelnummer **FR**
Numéro d'article **DE** Artikelnummer **ES** Número
de arti- culo **IT** Codice articolo **NO** Artikkelnum-
mer **DK** Artikelnummer **FI** Tuotenumero **NL**
Artikelnummer

EN Keep away from sunlight **SV** Förvaras ej i solljus
FR Conserver à l'abri de la lumière du soleil **DE** Vor
Sonnenlicht schützen **ES** Mantener alejado de la luz
solar directa **IT** Conservare al riparo dalla luce del
sole **NO** Oppbevar utenfor direkte sollys **DK** Skal
holdes væk fra sollys **FI** Suojaa auringonvalolta **NL**
Beschermen tegen zonlicht

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Range of application: A fluid resistant disposable face mask that creates a physical barrier between the mouth and nose of the wearer and the immediate environment. Designed to prevent spreading of infections by catching pathogens shed in liquid droplets and particles from the wearer's mouth and nose. Also protects the wearer from potentially contaminated fluid splash.

Structure and composition: Mask body made of three layers; spunbond and melt-blown polypropylene and silk-like ES-fiber. Nose strip made of aluminium. Ear loops made of spandex polyurethane and nylon. Product is free from latex, PVC and nickel.

Storage requirements: Store in a clean, dry and well-ventilated environment in original packaging, away from contaminated areas, dust, sunlight and damaging chemicals.

Recycling: Packaging is sorted as paper. The product as general waste.

Caution: Make sure the product is intact before use. Confirm the label, production date and use-before-date. Use the product within the validity period and do not re-use.

Instructions: Clean and disinfect hands before use. The coloured side is the external side. Avoid touching the internal side of the mask. With the nose strip facing upwards, attach the elastic cords around your ears. Hold the mask to your nose and stretch the mask around your chin. Use the index finger of both hands to pinch the nose strip so it takes the shape of your nose bridge. Make sure the mask fits your face shape and that it is comfortable. Do not touch the mask when in use. Only touch the elastic cords when removing it. Discard into a waste container with a lid or according to medical waste regulations in medical environments. Clean and disinfect hands after use.

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