NGPOD[®]

Instructions for Use

NGPOD[®] System, consisting of:

- NGPOD[®] Handheld Device  
- NGPOD[®] Sensor  
1639 0120

Mode d'emploi

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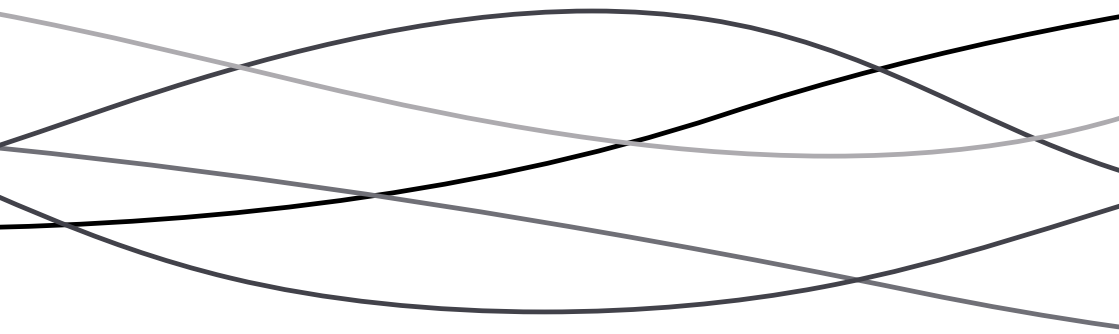
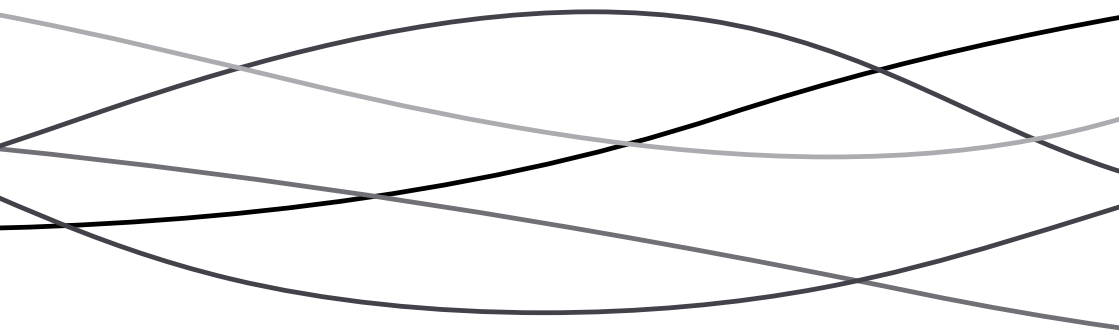
Revision: V11.3

Product Codes: NGPOD-01, NGPOD Sensor-01

The NGPOD® System is to be used **ONLY** by personnel trained in its operation

Registered Office (England & Wales):

NGPOD Global Ltd

Unit 7 Berkeley Court

Manor Park

Runcorn

Cheshire

WA7 1TQ

UK

Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

For all enquiries please contact our customer support team on:

☎ +44 (0) 161 696 6400

✉ info@ngpodglobal.com

Contents

Section 1: General and Safety Information

- 1.1 Preface
- 1.2 About this Manual
- 1.3 NGPOD® and Sensor Introduction
 - Device Description and Applications
 - Intended Use
 - Contraindications
- 1.4 Advisory Terms and Symbols
 - Warnings, Precautions and Notes
 - Operating Precautions
 - General Precautions
- 1.5 System Symbols
 - System Symbol Identification and Description
 - NGPOD® Symbols and Description
 - Sensor Symbols and Description
- 1.6 NGPOD® Technical Specifications
- 1.7 Device Inspection and Unpacking

Section 2: NGPOD® and Sensor Product Description

- 2.1 NGPOD® Feature Recognition
- 2.2 Sensor Feature Recognition

Section 3: NGPOD® and Sensor User Instructions

- Introduction
 - 3.1 Carrying Out a NGPOD® Test
 - 3.1.1 Prepare the patient and equipment
 - 3.1.2 Carry out the NGPOD® test
 - 3.1.3 Remove and dispose of equipment

Section 4: NGPOD® Cleaning, Maintenance, Storage and Disposal

- 4.1 NGPOD® Introduction
- 4.2 Cleaning/Disinfecting the POD
 - Cleaning
 - Disinfecting
 - Storage
 - Disposal
- 4.3 NGPOD® and Sensor Storage and Disposal
 - NGPOD® Storage and Disposal
 - Sensor Storage and Disposal
- 4.4 POD Collar Assembly – cleaning, removal and refitting
 - 4.4.1 Removal and Refitting
 - 4.4.2 Inspecting and Cleaning the Pod lens

Section 5: Troubleshooting

Section 1: General and Safety Information

1.1 Preface

This manual and any information contained within, may not be reproduced, distributed, or transmitted in any form, or by any means, for any purpose, without the express written permission of NGPOD Global Ltd. To order additional copies of this manual, contact NGPOD Global Ltd by using the contact number provided on page 2. You can also access the digital copy on our website www.ngpodglobal.com/IFU using password: showmethepod1 or scanning the QR code on the Sensor and POD labels.

The information in this document has been carefully compiled and is believed to be entirely accurate. Furthermore, NGPod Global Ltd reserves the right to make changes to any products herein to improve readability, function, or design. NGPOD Global Ltd does not assume any liability arising from the mis-use of any product described herein; neither does it cover any license under its patent rights nor the rights of others.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member state in which the user is established.

1.2 About This Manual

Prior to use, the operator must be trained in the use of the NGPOD®, a system to assist in the placement confirmation of Nasogastric (NG) feeding tubes. The system consists of a detection device (hereinafter referred to as the NGPOD®) and a single-use disposable Sensor (hereinafter referred to as the Sensor).

If a software change occurs and the operation/specification for the device changes, new or additional operating instructions will be issued, if required.

1.3 NGPOD® and Sensor Introduction

Device Description and Applications

The NGPOD® is a device to assist in position confirmation of a NG tube and consists of a re-usable, portable handheld detection device and a single use fibre-optic Sensor tipped with a pH indicator.

It has been designed to be an alternative to conventional feeding tube position confirmation tests, delivering a visual indication of the pH environment of the NG tube tip. The result is a YES/NO indication, reducing the risk of clinical interpretation error.

The Sensor attaches to the NGPOD® handheld device. The NGPOD® handheld device couples light into the fibre-optic Sensor. Light reflected from the sensor tip is received by the pod. Data from the pod sensors is processed to assess whether the colour of the indicator has passed a threshold that corresponds to the sensor tip being in an acidic environment.

The NGPOD® can only be used with the NGPOD® fibre optic sensor. The system is intended to aid in the correct placement of NG tubes by detecting an acidic environment through a colour change in the pH indicator.

As the consequences of feeding tube misplacement can be very serious, the quick, clear indication delivered by the NGPOD® can facilitate safer and more efficient NG placement in the clinical environment.

This device is fully compliant with NPSA (National Patient Safety Agency) feeding tube positioning confirmation guidelines.

Intended Use

NGPOD® is designed to signal to the clinical practitioner that the presence of a substance with a pH below 5.5 has been detected. When the NGPOD® device is used in the correct manner, a positive signal can typically be understood to mean that the presence of stomach acid has been detected.

It is the responsibility of the practitioner, using the result from the NGPOD® device, together with their own skills and experience, to make the final judgement as to the successful emplacement of the feeding tube and therefore whether feeding may safely proceed. These instructions DO NOT replace local guidelines for NGT placement. NGPOD® has been designed to replace aspirate testing as a 1st line, bedside test to assist in the confirmation of nasogastric tubes.

If a user chooses to use the NGPOD® system following an attempt to obtain aspirate from the nasogastric tube, the Operating Precaution 'If using after an attempt to obtain aspirate..' should be observed – see below.

Contraindications

The NGPOD® system should not be used where a pH test is clinically contraindicated for assisting in nasogastric tube placement, such as when a patient's condition may result in acid being present in the lungs, or where the respiratory tract may have a pH below pH 6.

1.4 Advisory Terms and Symbols

This section describes standard terms used to signal advisory statements and other messages in this manual and system labelling. Please read and understand this manual and all warnings and precautions before operating the NGPOD® system.

Warnings, Precautions and Notes

The information may be a warning, a precaution or a note. These are described as:

	WARNING: Signals a warning statement that alerts the reader to a potential safety hazard which, if not avoided, could result in serious injury or death
	PRECAUTION: Signals a precaution statement that alerts the reader to a potential safety hazard or unsafe practice which, if not avoided, may result in minor or moderate injury to the user or patient, or damage to the device or other property
	NOTE: Indicates that the information that follows is additional information, a tip that will help you recover from an error or point you to related information within the manual

Operating Precautions

	The NGPOD® has been designed and manufactured to exact specifications; however, it is not intended to replace trained personnel in the supervision of NG tube placement
	The NGPOD® Sensor is designed for use with specific lengths of NG Tube. Please ensure that the Sensor labelling has been checked prior to use to ensure compatibility and safety
	IF USING NGPOD® AFTER AN ATTEMPT TO OBTAIN ASPIRATE FROM A NASOGASTRIC TUBE the following should be observed: • Insufflate air into the nasogastric tube prior to inserting the NGPOD sensor into the lumen of the nasogastric tube. This should be done even if the attempt to obtain aspirate has failed.
	The fibre-optic Sensor is packaged separately to the NGPOD® handheld device. The Sensor is a single-use item and the NGPOD® handheld device is a semi re-usable item. A new Sensor must be used for each patient
	THE TIP OF THE SENSOR MUST NOT BE TOUCHED BEFORE INSERTING DOWN THE NG TUBE, AS IT COULD POTENTIALLY CONTAMINATE THE pH INDICATOR
	If significant resistance is felt when advancing the sensor down an NGT, the test should be stopped and the NGT confirmed as being validated with the NGPOD system using the Appendix to the NGPOD Instructions for use.
	If the sensor cannot be removed easily, remove the NGT with the sensor in situ and repeat the test using a new NGT and NGPOD Sensor.

	NGPOD Global Ltd will assume no responsibility for incidents which may occur if the product is not used, stored or transported in accordance with the environmental conditions stipulated in this document and on the package labelling
	The NGPOD® is tested to IP21. Wipe clean only
	The NGPod system has been tested and certified (to IEC-60601-1-2) for EMC (electromagnetic compatibility) to safely function in general and professional healthcare environments. The system should not be used within extreme electric or magnetic fields, for example in close vicinity of active MRI scanners, powerful radio transmitters or X-Ray equipment. In the unlikely event that such fields affect operation of the system, the pod should detect anomalous behaviour, safely block test progression and power down
	The NGPOD® is designed for regular use and should withstand everyday handling. If the NGPOD® is dropped onto a hard surface, or is suspected of being dropped, the operation should be checked by running the self-test. For self-test process, please see page 13 of this document
	Using the NGPOD® near operating equipment which radiates high-energy radio frequencies (such as electrosurgical/ cauterising equipment, two-way radios, or cellular telephones) may cause failure to operate correctly. If this happens, reposition the NGPOD® away from the source of interference, or turn off the NGPOD® or other equipment as appropriate
	An explosion hazard exists if the NGPOD® is used in the presence of flammable anaesthetics, such as Desflurane, Enflurane, Halothane etc. Exercise care to locate the NGPOD® away from such hazardous sources
	BEFORE USE, ENSURE THAT YOU HAVE PERFORMED THE SENSOR SAFETY CHECK OUTLINED IN SECTION 3.1.2 PAGE 13 OF THIS USER GUIDE
	Caution should be used in patients who have significantly altered anatomy of the mouth, oropharynx or upper gastrointestinal tract due to trauma or surgery, particularly where this affects the ability to clear secretions from the upper gastrointestinal tract. In these patient groups, a clinical assessment should be made as to whether x-ray rather than a bedside pH test is the most appropriate method of confirmation
	Do not use acidic lubricator with the sensor

General Precautions

	Do not use hard or sharp objects to press buttons
	The specified accuracy of the NGPOD® can only be achieved if the NGPOD® is used and maintained in accordance with the instructions given in this manual
	Follow the cleaning schedule and methods defined under Section 4 to ensure proper operation of the NGPOD®. Wipe off spills immediately. Although the NGPOD® is splash-proof, do not allow fluid or residues to remain on the NGPOD® as these could cause biological hazards or equipment malfunction
	The NGPOD® has been designed to be as safe as possible to handle; however, care should always be exercised in order, to avoid user injury or device breakage

1.5 System Symbols

The following symbols are used on the NGPOD® and Sensor. Labels on the device or statements in this manual preceded by any of the following words and/or symbols are of special significance and/or are intended to help you to operate the device in a safe and successful manner.

NGPOD® Symbols	
Symbols	Description
	CE mark indicates conformance to the Medical Device Regulation EU/2017/745
	Indicates conformance to the UK Medical Device Regulations 2002
	Not for disposal in domestic or municipal waste: this product is compliant with the WEEE (Waste Electrical and Electronic Equipment) directive
	Manufacturer
	Part number
	Indicates the need for the user to consult the instructions for use
	Serial number

Sensor Symbols	
Symbols	Description
	Sterilized with Ethylene Oxide
	Safety symbol. There are specific warnings and precautions associated with the device which are not otherwise found on the label
	Indicates the need for the user to consult the instructions for use
	DO NOT RE-USE: The sensor will not function if re-used and re-use could lead to cross-contamination issues e.g. infection
	Do not use if package is damaged. If the sensor packaging looks like it has been damaged/ opened, contact the storage department for a replacement
	Phthalate (DEHP) free
	Store sensors in box at ambient temperature 0 - 30°C

	Latex free
	Manufacturer reference/part number
	Keep out of direct sunlight. To indicate that transport package shall not be exposed to sunlight
	Keep dry
	CE mark indicates conformance to Medical Device Regulations EU 2017/745
	UKCA marked in accordance with the UK Medical Device Regulations 2002 0120
	The unique lot code (batch number) for the device (The lot number YYYY is year of manufacture and NNNN is a unique number)
	Date of manufacture
	Expiry date. This symbol is accompanied by a date to indicate that the device should not be used after the end of the year, month or day shown
	Manufacturer
	Do Not Restерilise

1.6 NGPOD® Technical Specifications

Component	Description
Housing Materials	Casing - Polycarbonate ABS (fire retardant). Front & Rear Labels - Polyester
Dimensions	170mm x 40mm x 45mm
Weight	118g
Power Supply	Battery type: Non-rechargeable AA 1.5Volt Alkaline
Battery Operation	At least 5000 tests under normal use conditions
Classification	Class 1 device
Protection against current leakage	Type CF equipment
Standards	Manufactured in accordance to ISO 9001 and ISO 13485. CE Marked in accordance to Medical Devices Regulation EU/2017/745 UKCA marked in accordance with the UK Medical Device Regulation 2002
Electrical safety	Electromagnetic compatibility (EMC) BS EN 60601-1-2:2015
Transport	Must be transported in the original packaging or equivalent to maintain protection

1.7 Device Inspection and Unpacking

Inspecting The NGPOD® Before Use

Remove the NGPOD® from the packaging and inspect for damage that could have occurred during shipment or storage.

Make sure you have the following items:


NGPOD® Handheld Device & Quick User Guide (QUG)

If any items are missing or damaged, please contact your supplies department.

	PRECAUTION: Visually inspect packaging and contents before each use
	PRECAUTION: Do not drop as this may cause damage to the NGPOD® handheld device. Do not use the NGPOD® and accessories if there are any obvious signs of damage. Return for inspection to NGPod Global Ltd


Section 2: NGPOD® and Sensor Product Description

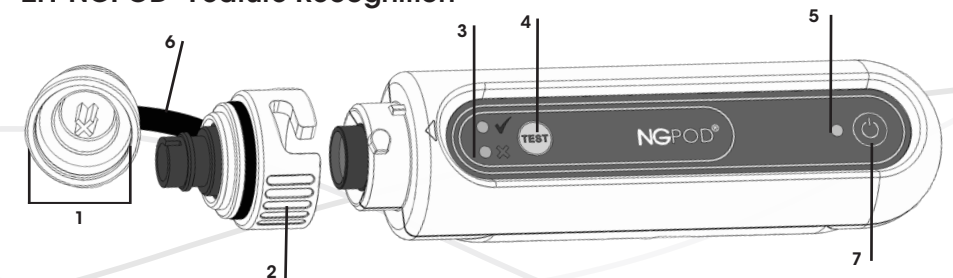2.1 NGPOD® Feature Recognition

	Feature	Description
1.	Self-Test Cap	Protects internal mechanisms. Remove cap to insert sensor connector to the NGPOD®
2.	Lock-Ring	To guide the locking and unlocking of the collar to NGPOD®
3.	Green Tick and Red Cross pH status indicators	Displays test result: Green indicates pH reference in range Red indicates pH reference out of range / undetected
4.	Test button	Press to start pH test
5.	Amber LED indicator light	Flashing amber light indicates NGPOD® is in a ready state for pH testing
6.	Tether	Attaches the NGPOD® cap to the body
7.	Power on/off button	Powers the NGPOD® on and off

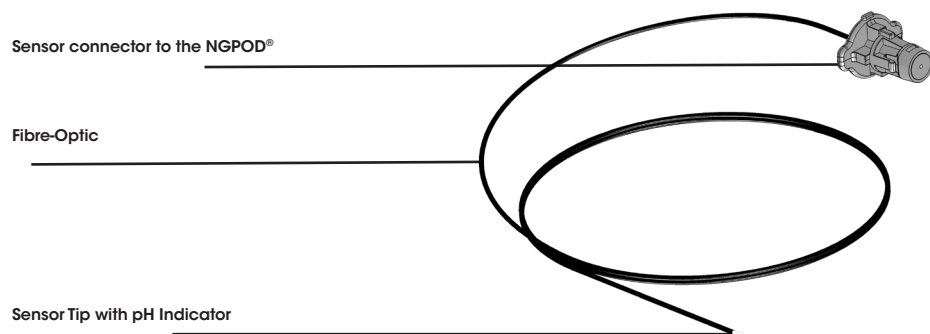2.2 Sensor Feature Recognition

Section 3: NGPOD® and Sensor User Instructions

Introduction

These instructions for use do not include instructions for insertion or management of a Nasogastric (NG) feeding tube; please refer to local policy and guidance for inserting and managing NG tubes.

Local guidance will indicate the frequency and circumstances of pH checks which can include;

- o Following initial insertion
- o Before administering feed and/or fluids
- o Before administering medication
- o Once daily check if on continuous feeding
- o Following episodes of vomiting, retching or coughing

3.1 Carrying Out the NGPOD® Test

Sequence for pH Testing

The key steps for carrying out pH testing are:

- 3.1.1 Prepare the patient and equipment
- 3.1.2 Carry out the NGPOD® pH test
- 3.1.3 Remove and dispose of sensor

3.1.1 Prepare the patient and equipment

Prepare patient

Follow local policy for infection prevention + control

Explain the procedure to patient, gain consent and position the patient to carry out the procedure.

Prepare equipment

Assemble equipment:

- o NGPOD® Handheld device
- o NGPOD® Sensor
- o Gloves
- o Apron
- o pH neutral, non-acidic lubricant

Check NGPOD®

- o Examine the outside of the NGPOD® for any signs of damage, including cracked housing
- o Do not use the NGPOD® if it has been damaged to the extent that cracks are evident in the casing or parts of it are missing
- o Ensure the lock ring is fully engaged

Check Sensor

Check that the sensor packaging is sealed, intact and undamaged before use. DO NOT USE if packaging is open or damaged.

Always ensure pre-test Sensor check is carried out prior to inserting the Sensor down the NG tube

Visually check that the sensor tip is **BLUE**

WARNING: ENSURE NGPOD AND SENSOR COMBINED SAFETY CHECK ON PAGE 13 IS PERFORMED PRIOR TO INSERTING THE SENSOR DOWN NG TUBE

WARNING: Prior to use of the NGPOD® system, compatibility with the nasogastric tube(s) (NGT) to be used should be established. Please refer to Appendix 1 of this IFU for a list of validated NGTs, and the correct sensor for use with each NGT. If the nasogastric tube you wish to use is not included in Appendix 1 please contact NGPod Global for advice or to arrange a validation

PRECAUTION: If the tip of the sensor is green/yellow, then it must not be used. Yellow discolouration indicates contamination

3.1.2 Carry out the NGPOD® test

Power on NGPOD® Handheld Device to perform Self-Test

Ensure the NGPOD® self-test cap is fully seated.

To power on, press and release the power button

When the device is powered on, the self-test begins automatically and takes a few seconds to complete. This checks the LED's, internal sensors and the battery.

During the self-test, observe the LED light sequence (Red - Amber - Green) :

The LED lights will cycle through the above light sequence, if self-test is successful then the device will enter Sensor Check mode. This is indicated by a repeating sequence of three flashes of the amber LED followed by a single flash of the red LED.

DO NOT proceed if any of the indicator LEDs fail to illuminate during self-test.

If the NGPOD detects an error during self-test, the red LED will flash 4 times and the device will power off.

If the NGPOD detects low battery voltage, the amber LED will flash 10 times after the self test is complete. The NGPOD will continue to operate but arrangements should be made to replace the NGPOD at the earliest opportunity.

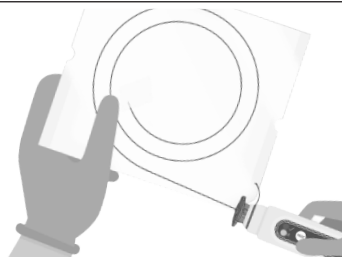
NOTE: The NGPOD® will remain in 'Sensor-Check' mode until the device has been connected to a new, unused Sensor and the software has validated all components are working

Sensor Check Mode

Remove NGPOD® handheld device self-test cap.

Open the Sensor package slightly from the corner (not fully) to expose the Sensor connector cap and retain the end of the Sensor in the packaging for protection.

Push the connector of the NGPOD® sensor into the collar of the NGPOD handheld device and rotate until you hear it click.

With the Sensor connected the device automatically runs through the Sensor Check.

If successful, the green indicator light will briefly illuminate and the device will enter pH test mode, signified by a repeated flashing amber light every second. Disconnect NGPOD® handheld device from the Sensor.

If the NGPOD fails to enter pH test mode with the sensor connected correctly, the sensor has failed the sensor check. Discard the sensor and try again with a new sensor.

THE DEVICE IS NOW READY TO BE USED

NOTE: The NGPOD® will power off after five minutes to conserve battery power. If the unit powers off, repeat the self-test as above

Clear the NG tube

When the NGPOD system is used for a repeat NG Tube position test consider insufflating air in the NG Tube before attempting to insert the NGPOD sensor.

NOTE: This will clear any debris from the end of the tube and dislodge the tip of the NG tube if it is against/adjacent to the gastric mucosa. The patient's medical condition must be considered prior to insufflating air down the NG tube, and if there is any doubt as to whether this is appropriate, air should not be insufflated

NOTE: In order to maintain the ability to successfully pass the NGPOD Sensor it is important that local or national guidelines in relation to the care of the NG Tube are followed to prevent the obstruction of the lumen of the NG Tube

Insert sensor into tube

Remove sensor from packaging, Insert the Sensor tip into the NG tube and gently advance the Sensor slowly down the NG tube until the NGPOD® Sensor connector is reached.

NB: The sensor is not attached to the POD during insertion

WARNING: Do not touch the tip of the sensor or allow it come into contact with any liquid prior to insertion into the nasogastric tube

NOTE: NG Tubes differ in exact length, the NGPOD® Connector may not come into direct contact with the NG Tube Connector. However, if the sensor cannot be advanced to closer than 5mm from the NG Tube Connector the sensor tip is likely to be at the distal tip of the NG Tube

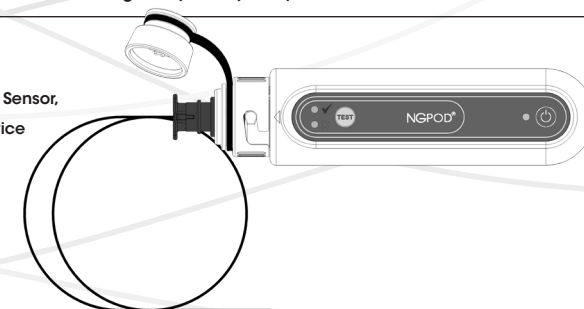
NOTE: Take care not to create any sharp or permanent bends in the sensor

PRECAUTION: Do not use force to advance the Sensor, the NG tube may be displaced, blocked or kinked. Follow local policy for instructions on checking tube patency and position

Without changing position of the NGPOD Sensor, reconnect it to the NGPOD handheld device

Perform pH test

When the amber LED light is flashing intermittently, press and release the pH TEST button. The result will display in 4 seconds

GREEN LED light indicates that the sensor tip is in a pH environment associated with the stomach. Follow local policy to commence feeding or administer medications.

A RED LED light indicates that the sensor tip is in a pH environment NOT associated with the stomach.

When a red LED light displays, DO NOT proceed with feeding or administration of liquids. Whilst the sensor is in place the NG tube may be advanced or withdrawn in small increments.

Refer to National/local policies and guidelines on repositioning and re-testing

	WARNING: It may take up to 5 minutes following initial insertion of the NGPOD Sensor and performance of a pH test for a Green Tick result to occur. This should be anticipated in patients where a higher pH may be expected (eg patients on acid reducing medication and/or patients with low residual stomach volume due to continuous feeding regimes)
	WARNING: Medications affecting pH The pH can be affected by factors such as; medication and frequency of feeds. Check whether the patient is on an acid inhibiting medication. Medications such as antacids, h2 antagonists and proton pump inhibitors can affect the gastric pH. The results of the NGPOD® test should be documented. If the patient is on medication that could elevate the pH level of gastric contents, refer to local policy and instructions for checking gastric pH
	WARNING: Dilution of gastric acid by enteral feed The presence of enteral feed/food in the stomach may alter the pH. If the test fails four times, remove and discard the sensor, wait for up to an hour to allow time for the stomach to empty and the pH to fall and then retest. The time interval will depend on the clinical need of the patient and if they are on continuous or bolus feeds
	PRECAUTION: Lens dirty/ not clean Check the NGPOD® lens. If the lens appears dirty or debris is visible, clean the lenses according to the cleaning instructions on page 16, then repeat the NGPOD® test from the beginning
	WARNING: Excessive/abnormal gastric secretions The NGPOD® device works by responding to a colour change when the tip of the sensor comes into contact with acidic liquids. Please note that patients experiencing excessive or abnormal gastric secretions may have higher levels of acidic liquids in the Oesophagus and Pharynx. The NGPOD® should not be used if there is a risk of a positive result through contact with acidic liquids in areas other than the stomach

Document test result

Follow local policy for recording test results in patient's medical records or on a test monitoring chart.

	WARNING: If the patient is subsequently moved before feed is administered, then a re-test should be conducted to ensure that the NG tube has not become dislodged from the position that it was in when the test was conducted
	WARNING: If the patient vomits during or immediately prior to the NGPOD® test then the user should revert to X-ray for NG placement confirmation as there is a possibility of gastric contents in the lung which could result in a positive result for a tube placed in the lung
	NOTE: After feeding, the NG tube should be flushed as per local policy

3.1.3 Removal and disposal of equipment

Remove the Sensor connector from the NGPOD®

Replace the cap onto the NGPOD®

Gently withdraw the Sensor from the NG tube

	NOTE: AFTER USE, ON VISUAL INSPECTION THE SENSOR TIP SHOULD BE GREEN/YELLOW. IF THE SENSOR TIP IS STILL BLUE, DO NOT COMMENCE FEED AND REPEAT TEST USING A NEW SENSOR
---	--

Dispose of Sensor in clinical waste

	WARNING: During pH testing, the Sensor has contact with gastric contents. Therefore, used Sensors carry a risk of infection
	WARNING: On removal of the sensor from the nasogastric tube, dispose of immediately and avoid any cross-contamination with the NGPOD hand-held device (POD)'

Following completion of every NGPOD test cycle, i.e. a green or persistent red result has been obtained and testing stopped, the device should be turned off by pressing and holding the power button until the amber light flashes five times. This procedure resets the device ready for a test on the next patient.

Section 4: NGPOD® Cleaning, Maintenance, Storage and Disposal

4.1 NGPOD® Handheld Device Cleaning

The NGPOD® requires no maintenance or servicing and contains no user serviceable parts.

	WARNING: Do not attempt to open or modify the NGPOD® as this will void warranty and may cause the unit to malfunction and/or give incorrect readings and/or may cause irreparable damage
	PRECAUTION: If the NGPOD® is dropped, subjected to excessive moisture, humidity or high temperature, or otherwise suspected to have been damaged, remove it from service
	PRECAUTION: The NGPOD® is designed for regular use, and should withstand everyday handling. If the NGPOD® is dropped onto a hard surface, or is suspected of being dropped, the operation and calibration should be checked by running the self-test

4.2 Cleaning and disinfecting the NGPOD®

Only the external parts of the NGPOD® should be regularly cleaned/disinfected.

Before routine cleaning/disinfecting the NGPOD®, ensure the cap is in place.

There may be infrequent occasions where it is necessary to remove the POD collar assembly for cleaning the collar and cleaning the area around the lens. See 4.4.2 for cleaning of the POD lens.

Cleaning and Disinfecting Procedure

- Clean and disinfect the NGPOD® with the cap in position
- The exterior of the NGPOD® can be cleaned with soft disinfecting wipes
- Test any cleaner on a small area of the cover before using it on the NGPOD®
- Always remove any soapy residue after cleaning.
- Do not use hard instruments for cleaning
- Always clean/disinfect the NGPOD® after each use to avoid cross-infection
- For a NGPOD® that has been in an isolation area, select those agents that both clean and disinfect
- Refer to infection prevention + control local policy for cleaning of non-invasive medical devices.

FOR FURTHER DETAILS ON CLEANING/DISINFECTING RECOMMENDATIONS AND SUITABLE CLEANING AGENTS, PLEASE REFER TO THE 'NGPOD CLEANING GUIDELINES' ON THE NGPOD WEBSITE. (www.ngpodglobal.com)

	WARNING: Avoid cleaning the inside of the cap when cleaning the optical lens. Do not allow scratching or obscuring of the optical pathway or allow objects or debris such as dust, dirt, hair, fluff, etc to interfere with the optical pathway. Device performance can be severely degraded and may cause incorrect operation. Keep the NGPOD® cap fitted when the Sensor is removed to prevent this
	PRECAUTION: Avoid abrasive cleaners or harsh chemicals which could affect the surface finish. e.g. scouring powder, wire wool, acetone cleaner
	PRECAUTION: Do not allow fluid or residues to remain on the NGPOD® as these could cause biological hazards or equipment malfunction. While the product design safeguards against minor fluid spillage, if fluid enters the NGPOD®, this will cause device malfunction and may void the warranty
	PRECAUTION: Do not autoclave or use ethylene oxide gas, this will cause device malfunction and may void the warranty

4.3 NGPOD® Handheld Device and Sensor Storage and Disposal

NGPOD® Storage

- Clean and disinfect the NGPOD® after use before storage
- Store the NGPOD® between use in a designated storage area
- Do not store the NGPOD® in an environment of high temperature, humidity, explosive or inflammable atmosphere or strong magnetic fields

NGPOD® Disposal

To dispose of the device, accessories or packaging, please ensure disposal minimises any negative impact on the environment. You may be able to use special recycling or disposal schemes. To find out about these, contact your technical service department or local waste disposal service. Existing national or local regulations concerning waste disposal must take precedence over the above advice.

In the event of difficulty with disposal, please return your device for disposal to NGPod Global Ltd at the address on the label. NGPod Global Ltd will dispose of the device in accordance with the WEEE directive.

Sensor Storage

- Store sensors in box at an ambient temperature of 0 - 30°C
- Do not store Sensor's in direct sunlight
- Do not store in high humidity
- Do not store near a strong electromagnetic field

Sensor Disposal

- Sensor is SINGLE USE ONLY
- Single use items must be disposed of following local policy and procedures
- Dispose in clinical waste as per local policy
- Do not clean or disinfect the sensor

4.4 POD Collar Assembly – cleaning, removal and refitting

The collar assembly consists of the pod collar (which accepts the sensor connector), the lock-ring and the test cap (connected to the lock-ring by a rubber tether). At the far end of the collar is a slot that engages with a feature of the sensor connector. When the collar is fitted this slot should be in line with the arrow on the pod casing.

Removal of the collar assembly is not recommended as part of normal use of the pod.

	WARNING: Before each use, the collar lock-ring should be inspected to ensure that it is fully engaged in the locked position – the D-shaped boss should be at the end of the slot in the lock-ring and the slot at the end of the collar should be in line with the arrow on the pod casing
---	--

4.4.1 Removal and refitting

If it is suspected that the pod lens is dirty (this could be indicated by repeated failure of the pod self-test), the collar assembly can be removed to allow inspection and cleaning

Removal

- Firmly hold the lock-ring by the grip features on either side of the lock-ring
- Twist the lock-ring in an anti-clockwise direction (when viewed from the front of the pod)
- Pull the collar assembly away from the pod

Refitting

- Taking care not to touch the pod lens, push the collar assembly on to the front of the pod. The D-shaped boss should fit easily into the slot in the lock-ring. Do not force the lock-ring into an incorrect orientation
- Firmly hold the lock-ring by the grip features on either side of the lock-ring
- Twist the lock-ring in a clockwise direction (when viewed from the front of the pod) until the end of the slot is in contact with the D-shaped boss

	NOTE: The collar assembly can only be fitted in one orientation. The slot at the end of the collar should be in line with the arrow on the pod casing. Do not attempt to force the collar assembly into a different orientation
---	--

4.4.2 Inspecting and cleaning the Pod lens

- Remove the collar as in 4.4.1
- Wipe the interior and exterior of the collar assembly with disinfectant wipes
- Carefully wipe the pod lens and surrounding area with disinfectant wipes. Wipe away any liquid remaining on the lens. Take care not to leave dust, lint or any other contaminants on the lens surface
- Replace the collar assembly as in 4.4.1

Section 5: Troubleshooting

Problem	Symptoms	Remedies
Pod fails self-test	Red LED flashes 4 times before pod powers down	• Ensure test cap is fitted and fully seated. • Ensure inside of test cap is clean • Ensure pod collar is fitted correctly and lock-ring is fully engaged – see section 4.4 • Ensure pod lens is clean – see section 4.4 • Do not operate pod under very bright direct lighting • Replace pod
LEDs are not functioning	Red, Amber or Green LED does not flash during self-test	• Replace pod
Low battery	Amber LED flashes 10 times during self-test	• Replace the NGPOD® at the earliest opportunity
Sensor fails sensor-check	Pod will not enter pH Test mode from Sensor Check mode	• Check that the sensor connector is correctly fitted to pod • Remove sensor from pod and long-press power button to switch pod off. Ensure test cap is fitted and switch pod on to repeat self-test. Remove test cap and reconnect sensor to pod • Sensor is damaged or defective – dispose of sensor. Use a new sensor

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