


DIAGNOMATIC

www.diagnostic.com

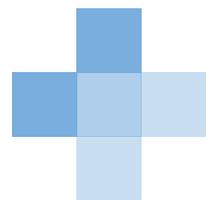

Dentistry



ISO
9001

ISO
13485

ISO
14001

ISO
45001

ISO
27001



Dentistry

Quality Control kits

Powered by:



We have prepared several QA / QC kits consisting of must-have phantoms, accessories and software that you can use in different situations depending on your requirements.

These can be your go-to selections when you are not sure what to choose for tests of a given modality.

We have introduced gradation of kits depending on the purpose and level of sophistication required:

BASIC: these sets are meant for constancy level testing purposes - tests that can be done practically by everyone who can use a diagnostic device

PRO: sets meant for acceptance and specialized testing - performed by specialized personnel, for example a medical physicist

Pro-Dent 2D BASIC kit



01-001



Powered by:



This kit is a versatile set of phantoms and software for carrying out constancy tests of intra-oral and OPG dental X-ray units. Thanks to the Diagnostics subscription, all tests can be quickly and effortlessly automatically analysed online.

Standard kit configuration:

- Pro-Dent set All (01-203)
- Diagnostics ECO annual subscription
- carrying case with foam inlay

The kit can be used to measure:

- dose reproducibility
- development process stability
- alignment and beam geometry
- spatial resolution
- low contrast resolution

Product features:

- Complies with:
 - OEC 61223-2-7 and IEC 61223-3-4
 - DIN 6868-5 and DIN V 6868-151
 - ÖNORM S 5240-5 and ÖNORM S 5240-11
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration

Pro-Dent 3D BASIC kit



01-011



Powered by:



This kit is a versatile set of phantoms and software for carrying out constancy tests of CBCT, DVT and other 3D imaging devices. Thanks to the Diagnostics subscription, all tests can be quickly and effortlessly automatically analysed online.

Standard kit configuration:

- Pro-Dent CT MINI (01-303)
- Diagnostics ECO annual subscription
- carrying case

The kit can be used to measure:

- image geometry
- pixel (matrix) size
- artefacts, noise
- homogeneity
- linearity
- contrast
- high-contrast resolution
- low-contrast resolution (contrast sensitivity) dose reproducibility

Product features:

- Complies with:
 - IEC 61223-3-4 and IEC 61223-3-5
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration

Pro-Dent All BASIC kit



01-021



Powered by:



This kit is a versatile set of phantoms and software for carrying out constancy and acceptance tests of Intra-oral, OPG, CBCT, DVT and other 3D imaging devices. Thanks to the Diagnostics subscription all tests can be quickly and effortlessly automatically analysed online.

Standard kit configuration:

- Pro-Dent set All (01-203)
- Pro-Dent CT MINI (01-303)
- Diagnostics BASIC annual subscription
- carrying case with dedicated foam inlay

The kit can be used to measure:

- dose reproducibility
- development process stability
- alignment and beam geometry
- image geometry
- pixel (matrix) size
- artefacts, noise
- homogeneity
- linearity
- contrast
- low-contrast resolution (contrast sensitivity)

Product features:

- Complies with:
 - IEC 61223-3-4, IEC 61223-3-5 and IEC 61223-2-7
 - DIN 6868-5 and DIN V 6868-151
 - ÖNORM S 5240-5 and ÖNORM S 5240-11
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration

Pro-Dent All PRO kit



01-022



Powered by:



This is a versatile set of phantoms and software for carrying out constancy and acceptance tests of intra-oral, OPG, CBCT, DVT and other 3D imaging devices. Thanks to the Diagnostics subscription, all tests can be quickly and effortlessly automatically analysed online.

Standard kit configuration:

- Pro-Dent set All (01-203)
- Pro-Dent CT mk II (01-501)
- Diagnostics PRO annual subscription
- carrying cases with dedicated foam inlay

The kit can be used to measure:

- dose reproducibility
- development process stability
- alignment and beam geometry
- image geometry
- pixel (matrix) size
- artefacts, noise
- beam hardening artefacts
- homogeneity
- linearity
- contrast
- high-contrast resolution in XY and Z plane
- low-contrast resolution (contrast sensitivity)
- line spread function
- point spread function
- MTF
- contrast to noise for different materials
- focal spot size

Product features:

- Complies with:
 - IEC 61223-3-4, IEC 61223-3-5 and IEC 61223-2-7
 - DIN 6868-5 and DIN V 6868-151
 - ÖNORM S 5240-5 and ÖNORM S 5240-11
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration



Dentistry

Phantoms





Pro-Dent Set

01-201 - intra-oral devices
01-203 - intra-oral and OPG devices



The Pro-Dent set is a universal set of phantoms for carrying out constancy and acceptance tests of conventional and digital dental X-ray units (intra-oral, panoramic and cephalometric).

This is not an all-in-one device where results of tests blur each other out. This is the only solution on the market that makes it possible to measure the X-ray beam collimation with a dental film or a digital detector.

Technical data (can be modified to customer specifications):



Pro-Dent Alpha phantom

- 3 step wedge (first step made of copper foil 0.3 mm thick, the next ones are made of polytetrafluoroethylene 8 mm and 16 mm thick)
- cover made of PMMA



Pro-Dent Beta phantom (optional)

- cone for perpendicular X-Ray beam control in the range of $0^\circ \div 1.5^\circ$
- pattern for beam radius estimation
- cover made of PMMA



Pro-Dent Gamma phantom

- pattern for line pair resolution evaluation (from 4 to 8 LP/mm)
- optional second pattern for line pair resolution evaluation (from 1.6 to 3 LP/mm) – OPG units
- four holes in 0.5 mm Al foil for low contrast resolution tests
- additional 6 mm aluminium filter
- cover made of PMMA



With the Pro-Dent set you can do the following tests:

- dose reproducibility
- development process stability
- perpendicular X-ray beam (range $0^\circ \div 1.5^\circ$)
- limitation and alignment of the X-Ray beam (including beam radius measurement)
- spatial / line pair resolution (perpendicular, parallel and rotated 45° to anode-cathode line)
- low contrast resolution

Accessories:

- 0.8 mm copper filter – patient's head equivalent
- positioning stand with a space for an analogue film or a digital detector
- band for firm, perpendicular attachment of the phantom to the X-Ray unit's beam applicator
- rings for centering phantoms on the X-Ray unit's beam applicator
- CD with documentation
- elegant and convenient box for storing phantoms


Product features:

- Complies with:
 - OEC 61223-2-7 and IEC 61223-3-4
 - DIN 6868-5 and DIN V 6868-151
 - ÖNORM S 5240-5 and ÖNORM S 5240-11
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration



Pro-Dent DIN 2D

01-220

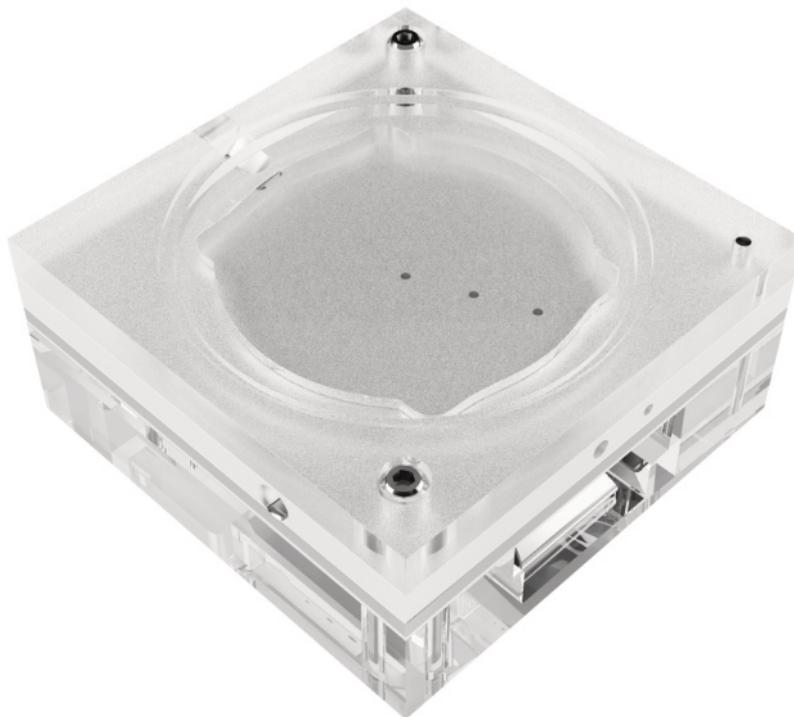

The Pro-Dent DIN 2D is a universal phantom for carrying out constancy and acceptance tests of conventional and digital dental X-ray units (intra-oral, panoramic and cephalometric).

Technical data (can be modified to customer specifications):

- Size 80 x 80 x 36 mm
- 3 centring rings with different diameters to fit standard tubes
- Slots for digital storage screen, intra-oral sensor, dose detector
- 45° Line Pair test (2.5 / 2.8 / 3.1 / 5.0 / 5.8 / 6.3 LP/mm; 0.05 Pb)
- 4 Low Contrast objects (Ø 2.5/2/1.5/1 mm)
- Integrated filtration 6 mm Al (purity 99.5 %)


Test Parameters:

- spatial resolution
- low-contrast resolution
- radiation field alignment
- image homogeneity
- dose equivalency check
- artefacts, image flaws, etc.

Product features:

- complies with:
 - IEC 61223-2-7 and IEC 61223-3-4
 - DIN 6868-5 and DIN V 6868-151
 - ÖNORM S 5240-5 and ÖNORM S 5240-11
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration

Accessory holders

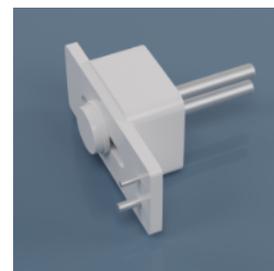

Pro-Dent CT MINI



01-303



Pro-Dent CT MINI phantom is a versatile quality control tool of dental Cone-Beam CT, Dental Volume Tomography (DVT), and other 3D imaging devices with even the smallest FOV (Field Of View).

The phantom consists of a main PMMA cylinder that houses modules with different test objects.

It allows performing most important imaging quality tests such as:

- image geometry
- slice geometry
- pixel (matrix) size
- artefacts, noise
- homogeneity
- linearity
- contrast
- high-contrast resolution
- low-contrast resolution (contrast sensitivity)

Accessories:

- test stand with spirit level for accurate placing of the phantom in the test position
- convenient, portable case for storing and transporting the phantom

Product features:

- complies with:
 - IEC 61223-3-4 and IEC 61223-3-5
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration





Technical data (can be modified to customer specifications):



Main cylinder

- contains markings for easy positioning in the dental unit and geometric distortion section
- an array of 2.0 mm diameter, 3.0 mm long holes uniformly pitched at 10.0 mm intervals
- diameter: 110 mm
- length: 130 mm
- made of PMMA (1.19 g/cm³)



Noise / uniformity module

- 20 mm thick



Linearity module

- containing 15 mm rods made of PTFE, polyamide, LDPE, air and water emulating epoxy embedded PMMA - pixel intensity / HU values samples



High contrast resolution module

- containing 7 objects for resolution evaluation: 10, 11, 12, 13, 14, 15, 16 LP/cm
- 0.3 mm bead for Modular Transfer Function (MTF) calculation



Low contrast section

- containing rods of a different diameter: 2, 3, 4, 6, 8, 10, 12 mm, filled with a substance whose density is 3% different from the body of the module



Slice geometry module

- 4 air rods, 3mm in diameter, placed in vertices of the 30mm square
- two aluminium wire ramps

Pro-Dent CT mk II



01-501



The new Pro-Dent CT mk II phantom is a versatile quality control tool of dental Cone-Beam CT, Dental Volume Tomography (DVT) and other 3D imaging devices according to the Radiation Protection Report no 172 by SEDENTEXCT.

The phantom consists of a main PMMA cylinder that houses modules with different test objects. Thanks to this design, you can perform tests with devices with a small FOV at different positions in the 160 mm phantom.

Accessories

- test stand with spirit level for accurate placing of the phantom in the test position
- folding base for test stand positioning on the X-ray unit's chair
- convenient, portable case for storing and transporting the phantom
- optional additional homogeneity disc

Product features:

- complies with:
 - Radiation Protection no 172 report by the SEDENTEXCT
 - IEC 61223-3-4 and IEC 61223-3-5
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration



Technical data (can be modified to customer specifications):



General

- diameter: 160 mm
- made of PMMA (1.19 g/cm³)
- total length: 170 mm
- positioning aids on the outside surface of the phantom

Noise / uniformity section

- uniform PMMA part of the phantom

5 layer modules section with modules containing

- Linear Spread Function (LSF) PTFE / PMMA interface
- Point Spread Function (PSF) - 0.25 mm tungsten steel wire in air
- XY high contrast resolution (aluminum/polymer) - 1.0, 1.7, 2.0, 2.5, 2.8, 4.0, 5.0 LP/mm
- Z high contrast resolution (aluminum/polymer) - 1.0, 1.7, 2.0, 2.5, 2.8, 4.0, 5.0 LP/mm
- 6 low-contrast groups of 1.0, 2.0, 3.0, 4.0, 5.0, 6.0 mm rods made of: aluminum, PTFE, POM-C (delrin), PE-300, air and water emulating epoxy; background made of PMMA
- 10 mm rods made of aluminum, PTFE, POM-C (delrin), PE-300, air and water emulating epoxy suspended in PMMA - pixel intensity / HU values samples
- stacked 20 mm diameter contrast-to-noise discs made of aluminum, PTFE, POM-C (delrin), PE-300 and air suspended in PMMA
- three 5.0 mm diameter titanium rods embedded in PMMA - beam hardening artefacts

Slice geometry section

- 20 mm thick module with two pairs of aluminum ramps and empty rods in vertexes of a square for detailed evaluation of slice thickness and geometry and laser position verification

Geometric distortion section

- an array of 2.0 mm diameter, 3.0 mm long holes uniformly pitched at 10.0 mm intervals

Pro-Dent CT 161



01-401



Phantom for acceptance and constancy testing of 3D function of dental Cone-Beam CT, Dental Volume Tomography (DVT) and 3D imaging devices according to DIN 6868-161 and QS-RL.


Technical data (can be modified to customer specifications):

- diameter: 160 mm
- 20 mm thick main module containing:
 - test object containing equivalents of air, soft tissue and bone
 - positioning markers
 - bubble level
- 20 mm, 50 mm and 60 mm thick PMMA modules with positioning aids
- test stand with a table for placing the phantom in the test position
- convenient, portable case for storing and transporting the phantom

Product features:

- complies with:
 - Radiation Protection no 172 report by the SEDENTEXCT
 - IEC 61223-3-4 and IEC 61223-3-5
- CE certified
- the manual provides detailed guidelines for carrying out each test, results assessment and registration



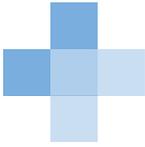

DIAGNOMATIC

www.diagnostic.com

World Headquarters

Kwiatowa 43A Street
22-105 Okszw, Poland
E-mail: contact@diagnostic.com
Phone: +48 668 024 874, +48 606 161 554

US office

8400 West Sunset Road
Black Fire Innovation Center, Suite 300
Las Vegas, NV 89113, USA
E-mail: contact.us@diagnostic.com
Phone: +1 (480) 799-7317

Distributor

