



AEGIS LINERS

AEGIS (e'·jis), n. a protective barrier; the state of being protected or safeguarded, as from danger or hardship

The development and commercialization of the AEGIS silicone liner has created a new class of skin interfacing products. AEGIS has redefined silicone liner technology by virtue of its innovative design and advanced processing.

ESP appreciates your choosing AEGIS for your clients. Our customer service division is ready to assist you - please call us with your inquiries and orders at 1-888-932-7377 (1-888-WEAR-ESP).

CARE AND DONNING INFORMATION

DONNING & DOFFING FABRIC COVERED LINERS: AEGIS Ultimate (6mm), AEGIS Streamline (3mm), AEGIS SoFlex (3mm), AEGIS TF (3mm), AEGIS Seal-Pro TT & TF (4mm)

- 1.) Invert liner fully.
- 2.) Place liner (with shuttle umbrella if used) on a pre-supposed central area of the distal residuum.
- 3.) Roll AEGIS firmly onto the residuum. DO NOT pull liner on. For transtibial liners, flex knee position (20 - 40 degree angle)
- 4.) REMOVAL - ROLL off for easy removal - DO NOT pull liner off.

DONNING & DOFFING UNCOVERED LINERS: AEGIS Streamline, AEGIS Seal-Pro

- 1.) Coat the outside of AEGIS w/alcohol/water spray, then invert liner fully.
- 2.) Spray outside (now, inside) again and rub liner walls together for full coverage of the lubricant.
- 3.) Center the vertical mold line of AEGIS on the mid-line of the limb.
- 4.) Roll AEGIS firmly onto the residuum in a flexed-knee position (20 - 40 degree angle). DO NOT pull liner on.
- 5.) REMOVAL - Spray AEGIS liberally with lubricant; roll off for easy removal - DO NOT pull liner off.

TRIMMING

With its pre-flexed 45 degree angle, AEGIS need never be cut to relieve popliteal/patella pressure, or bunching, nor to increase range of motion. The proximal silicone rings offer extra reinforcement and a gentle radius to reduce skin shear. Trimming or modifying violates manufacturer's warranty.

Note: The AEGIS TF liner may be trimmed to desired length. Trimming should be done smoothly. Apply silicone sealant to the cut edges to reinforce.

WASHING

Hand wash AEGIS with a mild liquid detergent - NEVER antibacterial soap. Wash inside and outside of liner, rinse thoroughly with warm water, pat dry. Soapy residue remaining on liner may cause dermatitis - rinse inside of liner with alcohol once or twice weekly to remove soap residue. Place liner on towel with foam insert and allow to air dry. Use foam insert to retain shape when not in use.

WARRANTY

AEGIS liners come with a 6 month manufacturers warranty.

SIZING - See reverse side.

IMPORTANT

Liner should never be worn on limb with open wounds, unless the wound has been treated by a qualified physician, and is dressed appropriately. If skin irritation develops, discontinue use and seek advice from a physician.


AEGIS Liners

Sizing System

esp

To determine proper BK liner sizing, two measurements are required:

- 1 Take circumference measurement 2" (5-6 cm) proximal to distal end of residual limb.
- 2 Take length measurement from mid-patella tendon to distal end of residual limb.

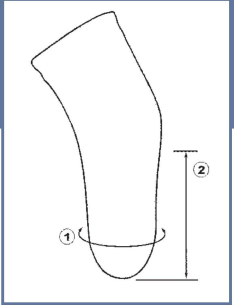

For liner size and item #, see chart below. Please specify Locking or Non-Locking (cushion) liner.

NOTE: On Aegis Streamline Liners, to specify the Covered Liner, add the letter "C" to the end of the item #.

All Aegis Ultimate, TF, SoFlex and Locking Seal Pro Liners have fabric cover.

AEGIS Ultimate				AEGIS Seal-Pro TT			AEGIS Streamline				AEGIS SoFlex		
1 Limb Cir.	2 Length	Ultimate Locking	Ultimate Non-Locking	Seal-Pro TT No Fabric	Seal-Pro TT with Fabric	Seal-Pro TT Locking	1 Limb Cir.	2 Length	Streamline Locking	Streamline Non-Locking	SoFlex Locking	SoFlex Non-Locking	
14-20 cm	4"	BK U-XS-4L	BK U-XS-4C	SP-XS-4	SP-XS-4F	SP-XS-4L	17-19 cm	4"	BK 14-4S	BK 14-4G	SBK 14-4L	SBK 14-4C	
	6"	BK U-XS-6L	BK U-XS-6C	SP-XS-6	SP-XS-6F	SP-XS-6L		6"	BK 14-6S	BK 14-6G	SBK 14-6L	SBK 14-6C	
	8"	BK U-XS-8L	BK U-XS-8C	SP-XS-8	SP-XS-8F	SP-XS-8L		8"	BK 14-8S	BK 14-8G	SBK 14-8L	SBK 14-8C	
20-26 cm	4"	BK U-S-4L	BK U-S-4C	SP-S-4	SP-S-4F	SP-S-4L	19-21 cm	4"	BK 16-4S	BK 16-4G	SBK 16-4L	SBK 16-4C	
	6"	BK U-S-6L	BK U-S-6C	SP-S-6	SP-S-6F	SP-S-6L		6"	BK 16-6S	BK 16-6G	SBK 16-6L	SBK 16-6C	
	8"	BK U-S-8L	BK U-S-8C	SP-S-8	SP-S-8F	SP-S-8L		8"	BK 16-8S	BK 16-8G	SBK 16-8L	SBK 16-8C	
26-32 cm	4"	BK U-M-4L	BK U-M-4C	SP-M-4	SP-M-4F	SP-M-4L	21-23 cm	4"	BK 18-4S	BK 18-4G	SBK 18-4L	SBK 18-4C	
	6"	BK U-M-6L	BK U-M-6C	SP-M-6	SP-M-6F	SP-M-6L		6"	BK 18-6S	BK 18-6G	SBK 18-6L	SBK 18-6C	
	8"	BK U-M-8L	BK U-M-8C	SP-M-8	SP-M-8F	SP-M-8L		8"	BK 18-8S	BK 18-8G	SBK 18-8L	SBK 18-8C	
32-38 cm	4"	BK U-L-4L	BK U-L-4C	SP-L-4	SP-L-4F	SP-L-4L	23-25 cm	4"	BK 20-4S	BK 20-4G	SBK 20-4L	SBK 20-4C	
	6"	BK U-L-6L	BK U-L-6C	SP-L-6	SP-L-6F	SP-L-6L		6"	BK 20-6S	BK 20-6G	SBK 20-6L	SBK 20-6C	
	8"	BK U-L-8L	BK U-L-8C	SP-L-8	SP-L-8F	SP-L-8L		8"	BK 20-8S	BK 20-8G	SBK 20-8L	SBK 20-8C	
38-42 cm	4"	BK U-XL-4L	BK U-XL-4C	SP-XL-4	SP-XL-4F	SP-XL-4L	10"	BK 20-10S	BK 20-10G	SBK 20-10L	SBK 20-10C		
	6"	BK U-XL-6L	BK U-XL-6C	SP-XL-6	SP-XL-6F	SP-XL-6L		25-27 cm	4"	BK 22-4S	BK 22-4G	SBK 22-4L	SBK 22-4C
	8"	BK U-XL-8L	BK U-XL-8C	SP-XL-8	SP-XL-8F	SP-XL-8L			6"	BK 22-6S	BK 22-6G	SBK 22-6L	SBK 22-6C
	8"						8"		BK 22-8S	BK 22-8G	SBK 22-8L	SBK 22-8C	
							10"	BK 22-10S	BK 22-10G	SBK 22-10L	SBK 22-10C		
							27-29 cm	4"	BK 24-4S	BK 24-4G	SBK 24-4L	SBK 24-4C	
						6"		BK 24-6S	BK 24-6G	SBK 24-6L	SBK 24-6C		
						8"		BK 24-8S	BK 24-8G	SBK 24-8L	SBK 24-8C		
							10"	BK 24-10S	BK 24-10G	SBK 24-10L	SBK 24-10C		
							30-32 cm	4"	BK 26-4S	BK 26-4G	SBK 26-4L	SBK 26-4C	
								6"	BK 26-6S	BK 26-6G	SBK 26-6L	SBK 26-6C	
						8"		BK 26-8S	BK 26-8G	SBK 26-8L	SBK 26-8C		
							10"	BK 26-10S	BK 26-10G	SBK 26-10L	SBK 26-10C		
							32-34 cm	4"	BK 28-4S	BK 28-4G	SBK 28-4L	SBK 28-4C	
								6"	BK 28-6S	BK 28-6G	SBK 28-6L	SBK 28-6C	
						8"		BK 28-8S	BK 28-8G	SBK 28-8L	SBK 28-8C		
							10"	BK 28-10S	BK 28-10G	SBK 28-10L	SBK 28-10C		
							35-37 cm	4"	BK 30-4S	BK 30-4G	SBK 30-4L	SBK 30-4C	
								6"	BK 30-6S	BK 30-6G	SBK 30-6L	SBK 30-6C	
						8"		BK 30-8S	BK 30-8G	SBK 30-8L	SBK 30-8C		
							10"	BK 30-10S	BK 30-10G	SBK 30-10L	SBK 30-10C		


To determine proper AK liner sizing, one measurement is required:

1 Take circumference measurement 2" proximal to distal end of residual limb.

AEGIS TF				AEGIS Seal-Pro TF		
1 Limb Cir.	Liner Size	AEGIS TF Locking	AEGIS TF Non-Locking	Seal-Pro TF No Fabric	Seal-Pro TF with Fabric	Seal-Pro TF Locking
24-28 cm	22 cm	TF-22-L	TF-22-C	SP-TF-22	SP-TF-22F	SP-TF-22L
28-32 cm	26 cm	TF-26-L	TF-26-C	SP-TF-26	SP-TF-26F	SP-TF-26L
32-36 cm	30 cm	TF-30-L	TF-30-C	SP-TF-30	SP-TF-30F	SP-TF-30L
36-40 cm	34 cm	TF-34-L	TF-34-C	SP-TF-34	SP-TF-34F	SP-TF-34L
40-44 cm	38 cm	TF-38-L	TF-38-C	SP-TF-38	SP-TF-38F	SP-TF-38L
44-48 cm	42 cm	TF-42-L	TF-42-C	SP-TF-42	SP-TF-42F	SP-TF-42L
48-52 cm	46 cm	TF-46-L	TF-46-C	SP-TF-46	SP-TF-46F	SP-TF-46L

To determine proper AK liner sizing, one measurement is required:

- 1 Take circumference measurement 2" proximal to distal end of residual limb.

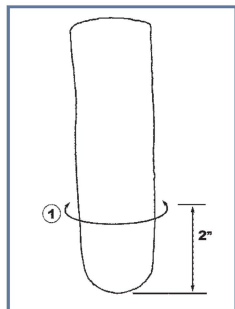

ENGINEERED SILICONE
PRODUCTS, LLC
407 Route 94, Columbia
New Jersey 07832, USA

sales@wearesp.com
+1-973-300-5120
www.WEARESP.com

esp

+1-888-932-7377
+1-973-300-5125 (fax)



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands