



OptiSplint Chairside Conversion

Pre-Surgery



Record Supporting
Anatomy With IOS.
Use Fiducial Markers

We recommend PLY files.
STL files are still acceptable



Finish Implant
Surgery and Place
MUAs

Post-Surgery



Screw OPTISPLINT Scan
Bodies into MUAs. Assemble
with honeycomb FRAME.
Scan outside mouth on plate
with IOS



Using the OPTISPLINT
Screws, secure the
MUA SCANCAPS on
the MUAs



Suture The Patient



Scan the MUA
SCANCAPS, sutured
tissue, and fiducial
marker with your IOS



Send All Acquired
Data to ITXPROS



Same Day
Conversion Within
90 Minutes

Requires Pre-Booking



Delayed Conversion
Within 48 Hours




SCAN QR CODE
TO START



Updated October, 2025

OptiSplint ALL-ON-ITX

We recommend PLY files.
STL files are still acceptable

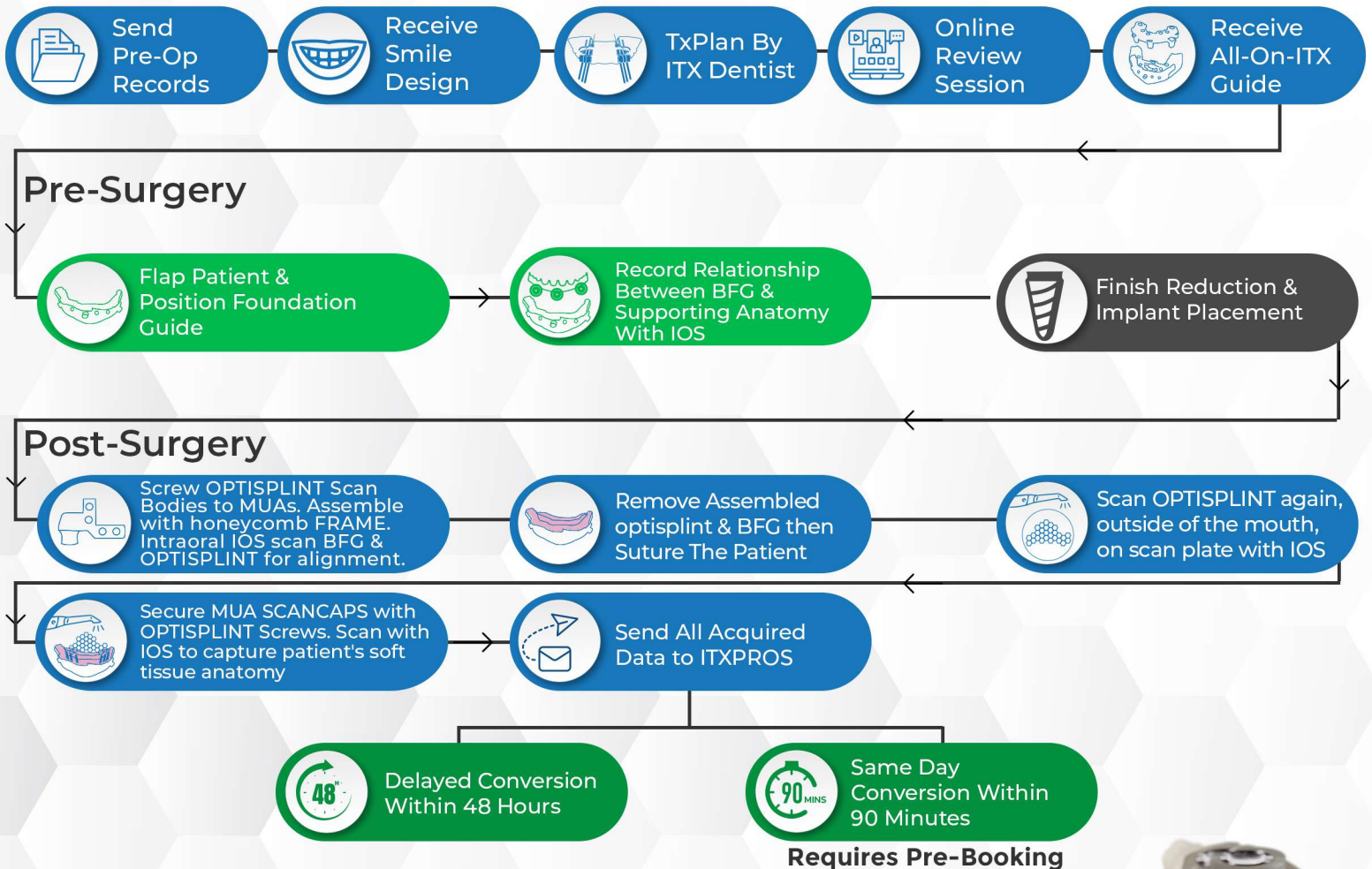

SCAN QR CODE
TO START



Updated October, 2025