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Sampling

Important

This instruction applies to animal blood samples for purely chemical PFAS analysis. Samples from animals or herds with a known or suspected risk of animal disease or zoonotic disease must not be shipped

- Avoid direct contact of the sample with PTFE/Teflon, fluorinated plastics or water/grease-repellent coated materials.
- To prevent contamination with cosmetics, hand creams, etc., nitrile gloves must be used.
- Do not open or decant sample tubes unnecessarily.
- The blood collection tubes (ideally 2x 10ml in Heparin / EDTA) + 1 blank tube must be labelled as one sample with the respective sample number. In case of positive findings, the blank tube may be used for further clarifications. Either Heparin or EDTA can be used for stabilization.

Heparin - Tubes

EDTA - Tubes

- Check whether the sample tube is fully labeled, undamaged and tightly sealed.

Storage after collection

- Cool the samples as quickly as possible after collection.
- Store the samples at 2 to 8 °C until shipment.
- If immediate shipment after sample collection is not possible, blood samples may be frozen and stored at temperatures below -20°C. Multiple freeze-thaw cycles should be avoided, as they may affect sample integrity.
- Avoid direct sunlight and unnecessary temperature changes

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Transport

- Triple packaging

Primary tube	Secondary packaging	Outer packaging
		
Close the tube tightly and check for cracks, leakage or blood traces. Cushion several tubes individually or separate them from each other. Label the sample directly on the tubes - B for blank	Place each primary container in a leak-proof safety bag. Close the secondary packaging securely. Label the sample on the sample bag	Fix the secondary packaging in such a way that it cannot move freely during transport. Enclose the accompanying form in a separate, clean and moisture protected sleeve.

- Cooling
 - Use pre-cooled ice packs; keep samples within the specified temperature range during transport.
 - Position the cooling packs outside the leak-proof secondary packaging.
 - Do not use dry ice or loose ice cubes.

Labelling and shipment

- Please use our web portal or our order sheets, which can be found on the [download page](#), to place your order. Web access with current sample information (interim reports / invoices / validated test report) can be requested via the following link: [New customer form](#)
- Shipping Monday (receipt SQTS Dietikon: Tuesday) to Thursday (receipt SQTS Dietikon: Friday) on the day of sampling.
- Avoid shipment before public holidays or in the event of expected delays.
- Provide the recipient address, sender and a reachable contact person in full.
- Keep the tracking number and provide it to SQTS if required.
- If you have additional questions about shipping, processing time or order processing, you can find the answers in our [FAQ](#) on the website (www.sqts.ch). Do not hesitate to contact our customer advisors directly at food@sqts.ch or by phone at +41 58 577 11 11.
- Shipment to the following address:
SQTS – Swiss Quality Testing Services
Lerzenstrasse 16
8953 Dietikon

Please do not ship, if...

- leaking, damaged or heavily contaminated sample container on the outside
- unlabeled sample or sample that cannot be clearly assigned
- known or suspected case of animal disease or zoonosis

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