DSI Denture Cream

Denture Adhesive Cream (for full and partial dentures)

Directions for Use:

- Cleaning and Drying:** Before applying, thoroughly clean the dentures to remove food debris and any remaining adhesive, then dry them.
- Application:** Apply the cream in thin strips or small dots, avoiding placement too close to the edges. For best results, use the cream sparingly – adjust the exact amount through trial if necessary.
- Placement:** Insert the dentures into your mouth, press firmly, and bite down for a few seconds to ensure secure fixation. For maximum effectiveness, wait about 15 minutes before eating or drinking.
- Removal:** Rinse your mouth with warm water, gently move it back and forth to remove the dentures. Then, remove any remaining cream with a soft brush and warm water.

Important Information:

- Use only a small amount of cream as shown in the diagram – excess may ooze out and reduce effectiveness.
- Apply no more than once per day.
- When used correctly, a 40 g tube may last for 4-5 weeks.
- When opening and closing the tube, make sure no moisture gets into the cream. If the tube's opening becomes blocked, carefully clear it with a needle.
- The cream is not a solution for ill-fitting dentures; in such cases, consult a dentist.

Precautions: Keep out of reach of children. Do not use if you are hypersensitive to any of the ingredients. In the unlikely event of serious reactions, seek medical advice and inform the manufacturer.

Active ingredients: Calcium Sodium PVM/MA Copolymer, Carboxymethyl Cellulose.

Contains menthol.

Packaging: Tube 40g.

Anhui Greenland Biotech Co., Ltd., PRC

Importer/Registration holder: Roman-Royal Dent Ltd. Tel.: +972-8-931-72-35

AMAR Medical Device Registration Number: 27220068

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© Anhui Greenland Biotech Co., Ltd סין
Anhui Greenland Biotech Co., Ltd., KHP