DSI Plaque Highlight

Tablets for revealing dental plaque

Who can use: Plaque-disclosing tablets are suitable for everyone. They are especially effective for children who are still developing good brushing and flossing habits, and for people prone to plaque build-up due to conditions such as receding gums or orthodontic appliances (braces, bridges).

Dosage: One tablet one time, for direct chewing. Children under 6 years of age should use under adult supervision.

Directions for Use:

1. Brush and floss your teeth as usual.
2. Chew one tablet and let it mix with the saliva in your mouth. Swish the liquid around for about 30 seconds, then spit it out and rinse with fresh water.
3. Look in the mirror: purple or red stains reveal the plaque missed during brushing or flossing. Your gums may also be stained – this is normal. Brush and floss again, more thoroughly.

Storage: This product is stored in a ventilated and dry place at room temperature.

Precautions: This product is not recommended for people allergic to any of the ingredients listed. This product is not intended for direct swallowing.

Ingredients: Dextrose monohydrate, PEG-150, Blackcurrant flavour, Silica, Magnesium stearate, Brilliant Blue FCF (CI 42090), Erythrosine (CI 45430), Citric acid, Sodium benzoate.

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