DSI White Pro

Teeth Whitening Strips

DSI White Pro Strips feature a gel with active ingredients that effectively remove stains and whiten enamel on the tooth surface. For optimal results, use the strips daily for 30 minutes over a period of 14 days.

Instructions for Use:

Strips should be applied to clean teeth surfaces, but in order to enhance strip adhesion, do not brush your teeth right before using the strip.

1. Tear open the foil bag and gently peel the strip from the backing.
2. Stretch and apply firmly to the teeth, align with the gums level (larger to the upper and smaller to the lower jaw). Smooth any bubbles or folds.
3. Wear the strips for 30-60 minutes (do not eat, smoke, or drink while wearing, except water).
4. Remove the strips and rinse mouth thoroughly with water. Wait at least 30 minutes before brushing your teeth.

Precautions:

- Do not eat, smoke, chew gum, or sleep while wearing the strips.
- For adults with a permanent dentition only.
- Keep out of reach of children.
- Do not swallow gel or allow it to come into contact with eyes. In case of eye contact, rinse immediately with water.
- Do not use if teeth or gums are inflamed, or during orthodontic treatment (braces).
- Strips whiten only natural teeth and will not change the colour of fillings, crowns, or veneers.
- Do not swallow the plastic strip.
- Do not reuse. Dispose after use.

Possible Side Effects:

- Temporary tooth or gum sensitivity. If you experience increased sensitivity, stop using the strips for 2-3 days. If the discomfort worsens or persists, discontinue use and consult a dentist.
- Mild irritation of the oral mucosa. If irritation increases or lasts for more than 48 hours, discontinue use and consult a dentist.
- After whitening, you may notice white spots on the teeth. These are harmless, caused by temporary water loss in the enamel, and usually disappear on their own. In most cases, the treatment only highlights pre-existing spots (caused by hypocalcification, plaque, or excess fluoride during childhood). With good oral hygiene and additional dental care if needed, the appearance of these spots can be reduced.

Storage: Store in a cool, dry place at room temperature.

Ingredients: Glycerin, Propylene Glycol, Aqua, Sodium Bicarbonate, Carbomer, Cellulose Gum, PVP, Coconut Oil, Menthol, Sodium Hydroxide.

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Nanchang Le Ang Technology Co. Ltd, KHP