DSI Denture Cushions

Adhesive Cushions for Removable Dentures

DSI Denture Cushions provide a secure seal and comfortable fixation of your dentures. Made from soft non-woven material and free from zinc, the cushions fill the gap between the denture and the gums, reducing pressure, preventing irritation, and improving denture stability throughout the day.

Intended Use: Guaranteed to improve the fixation and comfort of removable dentures.

Create an additional soft layer between the denture base and the oral mucosa, reducing pressure points and inflammation, while increasing confidence during chewing and speaking.

Directions for Use:

1. Clean and moisten the denture. If necessary, trim the cushion with scissors to fit the denture shape, leaving a 2 mm gap from the denture's edge (do not cover the margin).
2. Moisten the cushion in warm water for 15 seconds or more.
3. Place it on the denture and press firmly with both thumbs from the center outward until the surface is smooth and fully adapted. Insert the denture into the mouth and hold until secure.


* If the wafer does not hold, moisten the wafer more thoroughly.

Warnings and Precautions: For use only with removable lower jaw dentures. Do not use with dentures that have a soft lining or soft base. Do not use in case of hypersensitivity or allergy to any of the components. If irritation, discomfort, or any other problem occurs, discontinue use and consult a physician. For single use only. Keep out of reach of children. Do not swallow or reuse.

Composition: Sodium alginate, Polyester fiber, Viscose fiber. Zinc-free.

Storage Conditions: Store in a cool, dry place at room temperature. Avoid direct sunlight and moisture.

Packaging: 30 pcs.

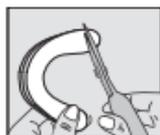Anhui Greenland Biotech Co., Ltd., PRC

Importer/Registration holder: Roman-Royal Dent Ltd. Tel.: +972-8-931-72-35

AMAR Medical Device Registration Number: 27220068

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Intended Use: Guaranteed to improve the fixation and comfort of removable dentures. Create an additional soft layer between the denture base and the oral mucosa, reducing pressure points and inflammation, while increasing confidence during chewing and speaking.

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Warnings and Precautions: For use only with removable upper jaw dentures. Do not use with dentures that have a soft lining or soft base. Do not use in case of hypersensitivity or allergy to any of the components. If irritation, discomfort, or any other problem occurs, discontinue use and consult a physician. For single use only. Keep out of reach of children. Do not swallow or reuse.

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Storage Conditions: Store in a cool, dry place at room temperature.

Avoid direct sunlight and moisture.

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Anhui Greenland Biotech Co., Ltd., KHP