



# DSI Retainer Cleaner

## Tablets for Cleaning Retainers, Aligners, and Other Removable Appliances

### Directions for Use:

- 1. Preparation:** Fill a container large enough to fully submerge the device, add warm (not hot) water, and drop one DSI Retainer Cleaner tablet.
- 2. Soaking:** Place the retainer, aligner, or other removable appliance into the bubbling solution, ensuring it is fully submerged, and leave for 15 minutes. For wire-based appliances – do not exceed 15 minutes; for heavy staining on plastic appliances, a longer soak is permitted (up to overnight).
- 3. Rinsing:** Remove the retainer and rinse thoroughly under running water; gently pat dry with a clean cloth and place in the mouth, or store the appliance in a hygienic case.
- 4. Disposal:** Pour the used solution down the drain with plenty of water, rinse the cup, and wash your hands thoroughly.

**Tip:** For enhanced cleaning and long-term freshness, use the solution together with an ultrasonic cleaner.

**Warning:** Use only as directed and in accordance with the instructions. Do not ingest the tablets or the solution. Do not mix with other cleaning agents. The solution is intended for single use only; do not reuse a dissolved tablet or leftover solution.

Do not soak wire appliances for more than 15 minutes.

Always rinse the retainer thoroughly under running water before placing it into the mouth.

Do not use for cleaning natural teeth or for any purposes other than those recommended.

**Precautions:** Keep tablets out of reach of children and individuals who may have difficulty reading or following instructions. During preparation, soaking, and rinsing, do not eat, drink, or smoke. Do not inhale the dust or vapors of the product. Avoid contact with eyes; in case of contact, rinse immediately with plenty of water. If wearing contact lenses, remove them if possible and continue rinsing. If irritation persists, seek medical advice. Do not swallow. Ingestion may cause stomach irritation and other adverse effects. If swallowed accidentally, seek medical help immediately.

Contains menthol oil and persulfates, which may cause allergic reactions.

**Storage:** Store in a dry place at room temperature, away from moisture and direct sunlight.

**Packaging:** 60 tablets.



Anhui Greenland Biotech Co., Ltd., PRC

Importer/Registration holder: Roman-Royal Dent Ltd. Tel.: +972-8-931-72-35