



For partial or total mummification of the milk tooth root pulp

DSI Formocresol



INSTRUCTIONS FOR USE

PURPOSE: Formocresol is intended for root canal disinfection and for use as a medicament in pulpotomy procedures due to its antibacterial activity.
For professional dental use only.

COMPOSITION: 19% Formaldehyde, 35% Cresol, 17.5% Glycerin, Isopropyl Alcohol, and Distilled Water.

DIRECTION FOR USE:

- **Immediate treatment:**
After removal of the coronal pulp, place a cotton pellet moistened with Formocresol directly on the remaining pulp tissue.
After 5 minutes, remove the cotton pellet, cover the cavity with a ZOE material (DSI Zoer), and complete the permanent restoration.
- **Prolonged treatment:**
Place a cotton pellet moistened with Formocresol on the pulp treatment site and cover the cavity with a temporary filling material (DSI Temp) for one week.
After one week, remove the cotton pellet, cover the cavity with a ZOE material (DSI Zoer), and complete the permanent restoration.

Warning: Contains formaldehyde, which may be harmful if inhaled or swallowed. Contact with skin may cause irritation, burns, or sensitization. In case of skin contact, immediately remove the product and wash the affected area thoroughly with soap and plenty of water.

If skin irritation, sensitization, or any other allergic reaction occurs, discontinue use and seek medical attention.

In case of contact with eyes, rinse immediately and thoroughly with plenty of water and seek medical attention.

CONTRAINDICATIONS:

Do not use in patients with known hypersensitivity to any of the product components.

PRECAUTIONS: Avoid contact with skin and eyes.

Avoid inhalation of vapors and ingestion of the product.

Use appropriate personal protective equipment when handling the product.

STORAGE:

Store in a dry place at +5°C to +25°C and at a relative humidity below 50%.

Keep the container tightly closed. Replace the bottle cap immediately after use.

Protect from moisture and direct sunlight. Keep out of reach of children.

PACKAGING:

Ref: CRESOL – Bottle, Liquid 15ml

Ref: CRESOL30 – Bottle, Liquid 30ml

SYMBOLS:



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



The product is not sterile



Keep Out of Reach of Children



Medical Device



Use by Prescription only



Manufacturer



Catalogue number



Batch code



Use by Date



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.

Visit www.dsisrael.com or email to info@dsisrael.com for more information