



VIST<sup>®</sup>  
GO

QUICK SETUP GUIDE

# QUICK GUIDE TO SETTING UP THE VIST G7 GO

By following this guide, you will be able to start using your VIST G7 GO system within minutes.



You will also find this support quick setup guide at: [support.mentice.com](https://support.mentice.com)



# VIST G7 GO BACKPACK

## UNPACKING THE BACKPACK

### ① VIST G7 Mini

### ② VIST G7 GO Connector Bridge

### ③ Laptop

*Found in the laptop compartment on the back*

### ④ USB-C cables

*Rare outer compartment*

### ⑤ Laptop power cable

*Rare outer compartment*

### ⑥ Clinical Devices

*Compartment foam found inside rare rare lid.*




You will also find this support quick setup guide at: [support.mentice.com](https://support.mentice.com)




## CONNECTING THE BRIDGE

- 1 Insert the two connector ends to the back and front of the two VIST G7 Minis at a 45-degree angle.
- 2 Lock the connector bridge in place by turning 45 degrees.

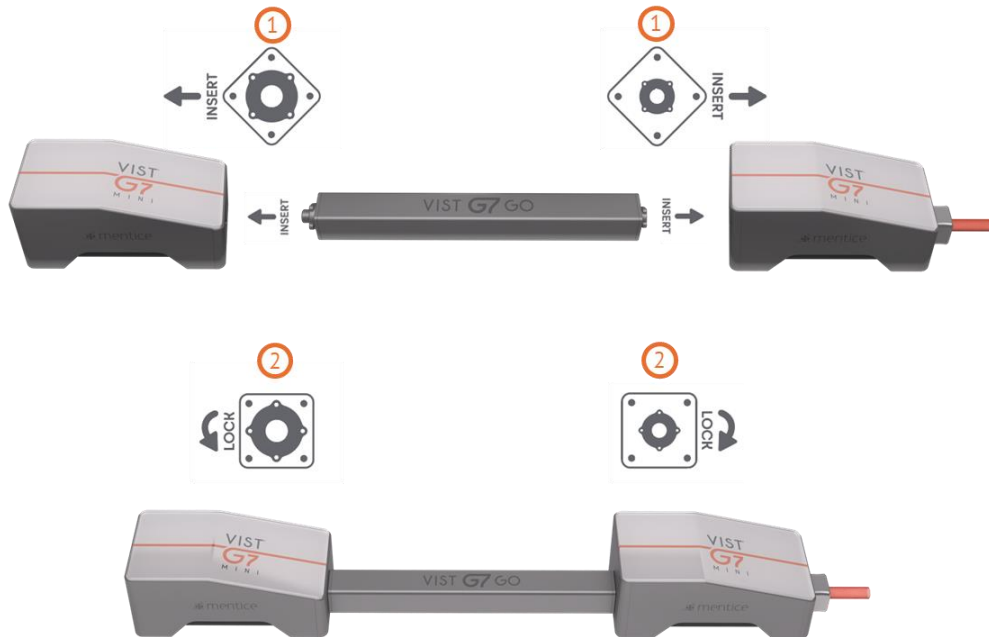

## CONNECTING THE SYSTEM

- 3 Connect the power cable to the laptop
- 4 Connect both USB-C cables to the back of the VIST G7 Minis and laptop

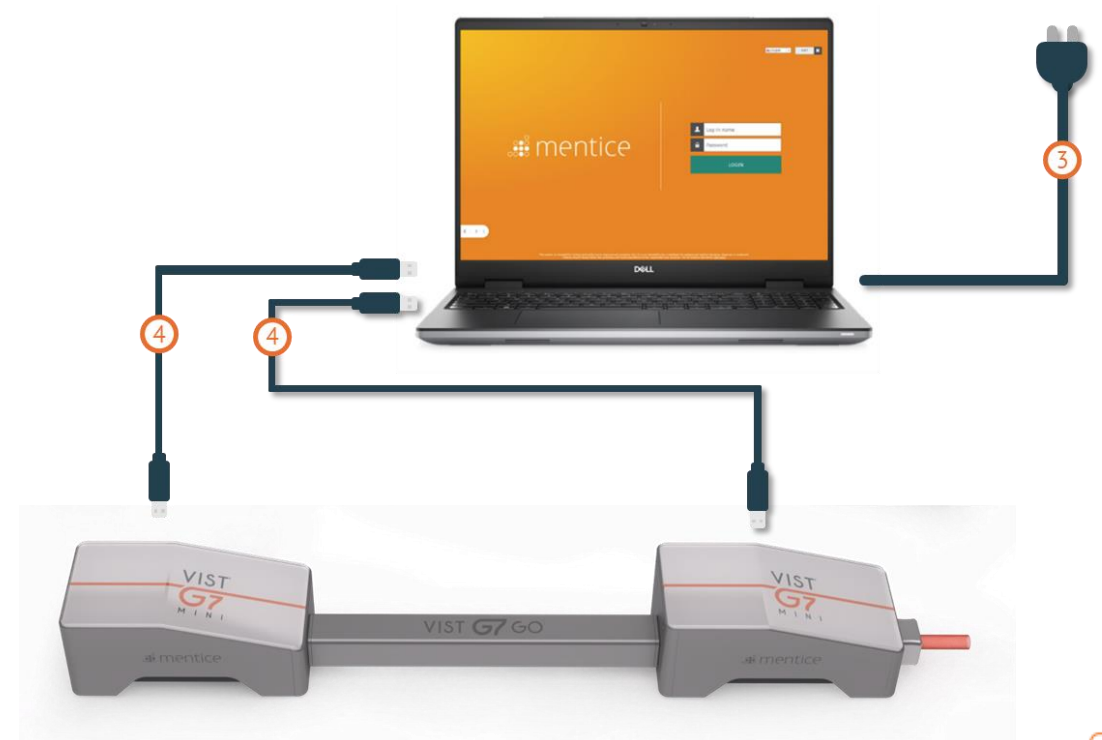

# HELP CENTER

## SUPPORT.MENTICE.COM

Discover and find useful information directly. Download manuals, watch instruction videos, read FAQs, and much more.

### EMEA

+44 20 3885 1601

[emeasupport@mentice.com](mailto:emeasupport@mentice.com)

### APAC

[apsupport@mentice.com](mailto:apsupport@mentice.com)

[support@mentice.com](mailto:support@mentice.com)

### USA

+1 (877) 636-8423

[ustechsupport@mentice.com](mailto:ustechsupport@mentice.com)

### LATAM

[latamsupport@mentice.com](mailto:latamsupport@mentice.com)

[www.mentice.com](http://www.mentice.com)