


ADVANCED WAVE™

REFERENCE CARD

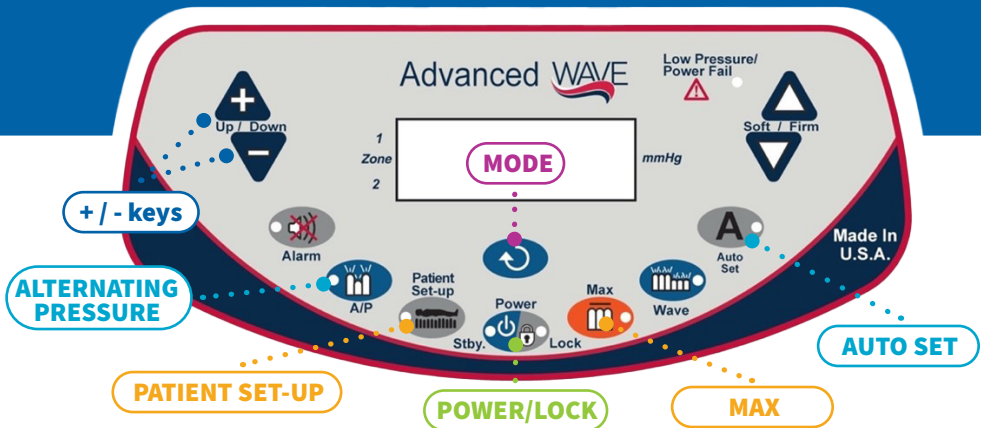

POWER ON / CONFIGURE AUTO-SET

- 1 Press the **POWER/LOCK** key to power on. Then, press the **AUTO SET** key to configure auto-set. Use the **MODE** key to toggle, and the **POWER/LOCK** key to confirm selection. The display will then show "STANDBY"
 - **Standard mode** is for a patient 5'2 - 6'4 weighing 120 - 250 lbs.
 - **Bariatric mode** is for a patient weighing over 250 lbs. (depending on height)
- 2 Press the **POWER/LOCK** key to begin. The unit will default to WAVE Therapy in the optimum therapeutic range for the population that was selected in step 1.

PATIENT SET-UP

- 1 Press the **PATIENT SET-UP** key and follow the prompts to enter weight and height
 - Using the **+ / -** keys increments number up and down for weight
 - **MAX** key toggles between lbs/kgs
- 2 **MODE** key confirms selection
- 3 Using the **+ / -** keys increments number up and down for height
- 4 **MODE** key confirms selection
- 5 Review weight and height on upper left of display. If values are correct, press **MODE** key again to confirm settings and begin therapy
 - To start over, press **PATIENT SET-UP** key twice



ADVANCED WAVE™

REFERENCE CARD

SELECTING A DIFFERENT MODE

The Advanced WAVE defaults to WAVE therapy, but if a different mode is required:

- 1 Press the **MODE** key to toggle through the available therapy modes
 - Therapy (Static low air loss)
 - WAVE
 - Massage
 - Upright (Fowler Boost)
 - Increases the internal pressure in the mattress 80% above the current pressure (mat of 35 mmHg)
 - Used when the head of the bed angle is above 35°

NOTE: The blower should automatically enter “Upright” mode when the head of the bed angle is above 35°. If not, use the **MODE** key to enter “Upright” mode.

- 2 Press the **ALTERNATING PRESSURE (A / P)** key to begin alternating pressure therapy
 - Using the + / - keys increments the A / P interval time (1-99 minutes)

- 3 Pressing the **MAX** key inflates the mattress to its maximum internal pressure of 35 mmHg (+ / - 5)
 - Max inflate is used to firmly inflate the surface during ingress, egress, and during all patient contact. Max inflate is NOT a therapeutic mode
 - While in Max Inflate, an audible tone sound every 3 minutes as a reminder that Max Inflate is still active
 - Max inflate will automatically be canceled after 15 minutes, and the unit will revert back to WAVE therapy

- 4 To prevent unintended key presses, press and **HOLD** the **POWER / LOCK** key until the LOCK LED illuminates. **Please note** a quick press of this key will cause the unit to shut off and the mattress to deflate.

CPR



If CPR is required,

PULL the black mattress connector from the upper right side of the control unit.

At the end of the emergency, connecting the mattress again will automatically resume the previously set therapy.




ADVANCED WAVE™

REFERENCE CARD

ADJUSTING WIDTH OF BARIATRIC MATTRESS

The bariatric Advanced WAVE can be adjusted in width.

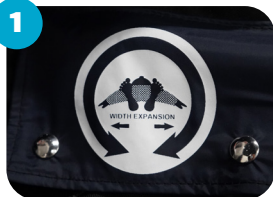

1 Locate the width expansion dial at the footend of the mattress to the left



2 Turn the dial to the desired width and adjust the width of the bed frame accordingly



3 If reducing width of the mattress, locate the **EXPANSION DEFLATE** tag, squeeze gray tabs and disconnect.

ADJUSTING LENGTH OF MATTRESS

The variable length and bariatric Advanced WAVE can be adjusted in length.




1 Locate the length expansion dial at the footend of the mattress to the right



2 Turn the dial to the desired length and adjust the length of the bed frame accordingly

ADJUSTING HEEL ZONE OF MATTRESS

The heel zone firmness of the Advanced WAVE can be adjusted.

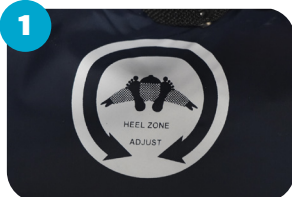

1 Locate the heel zone dial at the footend of the mattress to the right



2 Turn the dial to the desired firmness



3 Use the heel zone adjust guide accordingly

