


Exporting your CT scans to Hybridgerx.com:
ORTHOPHOS SL 3D


1. Open Sidexis and go to the output tab in the top righthand corner.

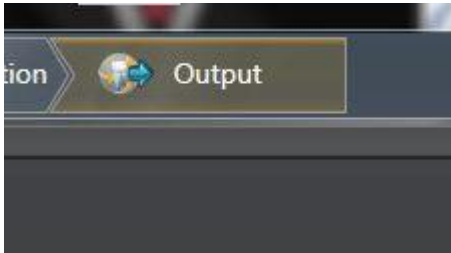

2. Once you have done that you can choose the DICOM exam export option.



3. Save this to the desktop, labeling with patient name GUIDE IN or GUIDE OUT.
4. Once the folder is on the desktop, right click and send to compressed (zipped) folder.

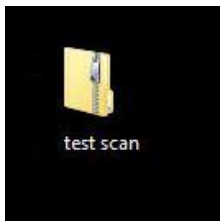

5. Upload your zipped folders to the Hybridge portal.

YOU'VE NOW SUCCESSFULLY UPLOADED YOUR CT TO HYBRIDGE! YAY!!!