



1. On your desktop, right click on an empty space. Go to “New” and click “Folder”. A folder will appear with highlighted blue text. Type in the patient name and push “Enter” on your keyboard.

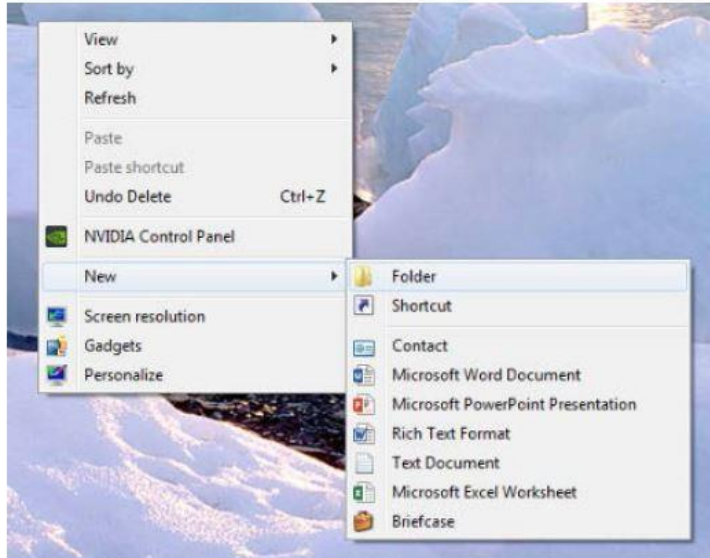


2. Go to patient scan and click on “Output”, then “DICOM Exam Export”.

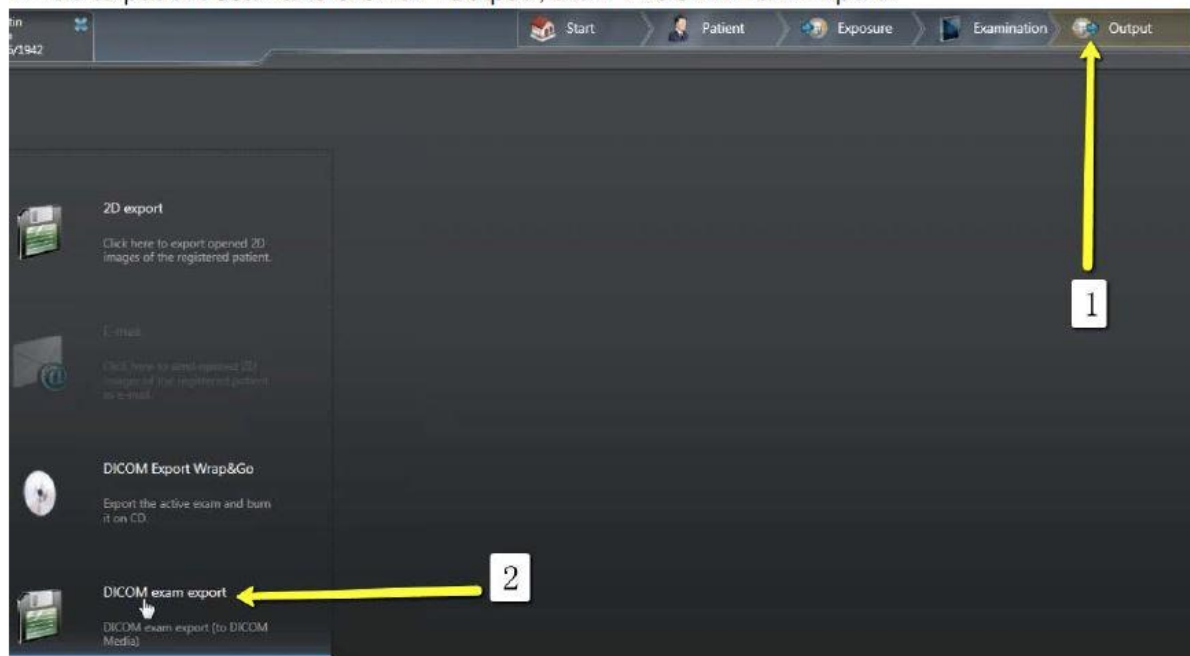



3. An additional window should pop up with the patient scan. Click on "Settings".




4. Please ensure that "Include Raw Data" is selected. No other options should have to change. Click "OK".

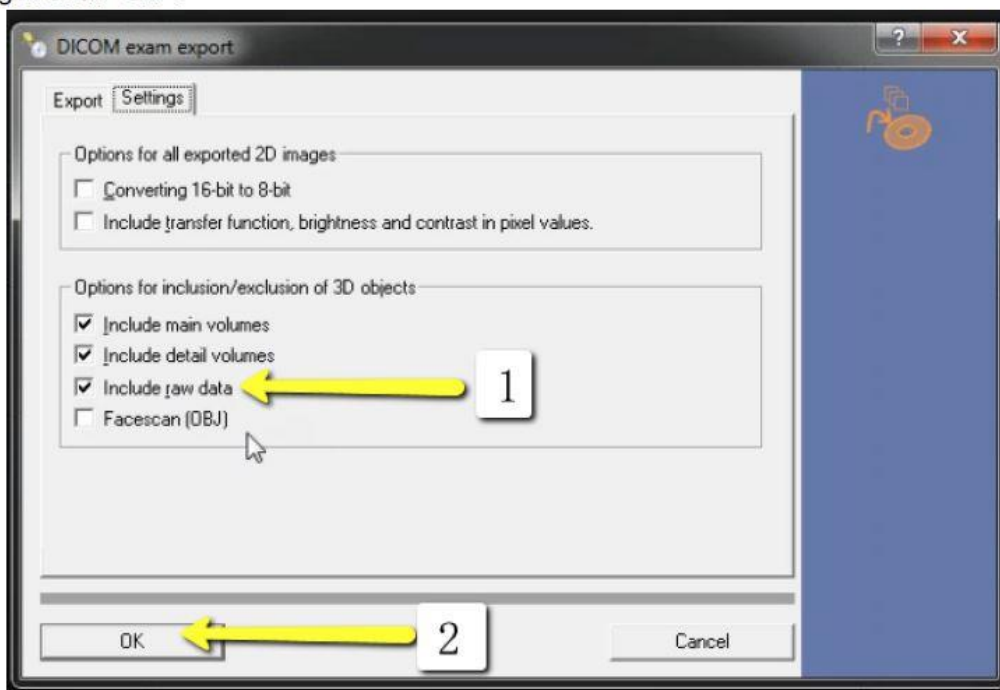


5. Another window should pop up for a save location. Select the folder you just created on the desktop and click “Open”, then “Save”.

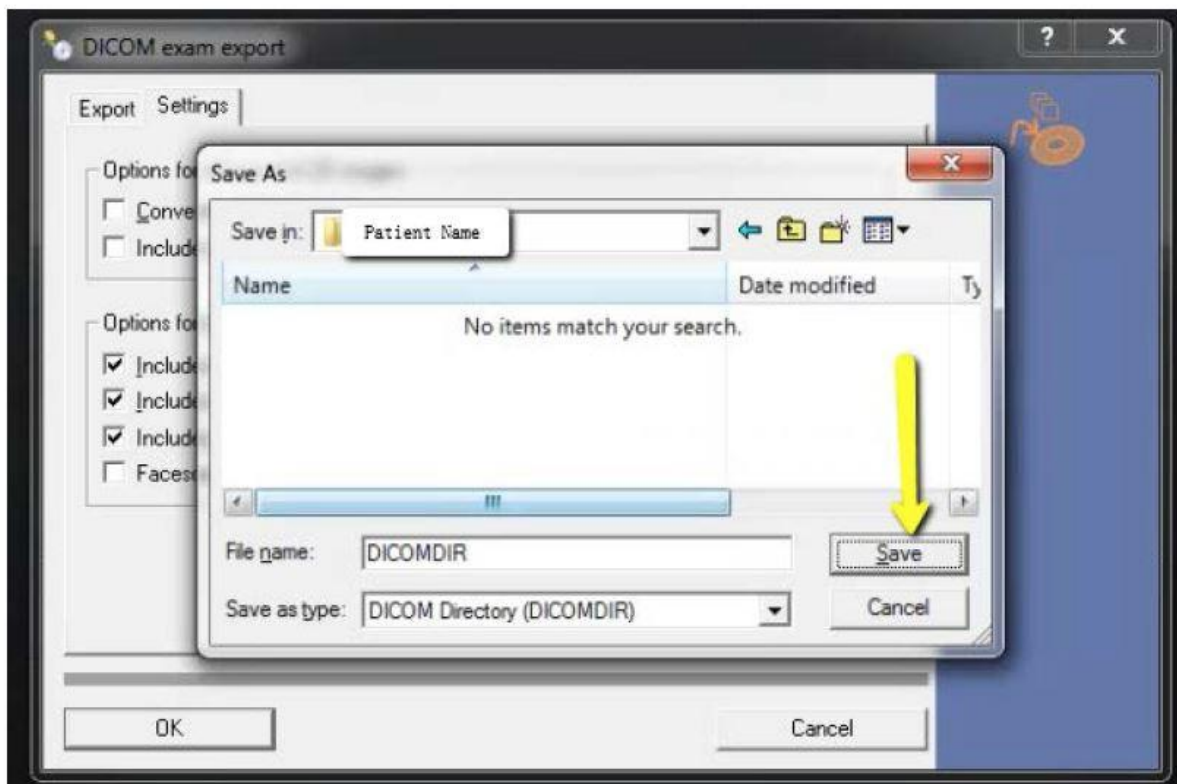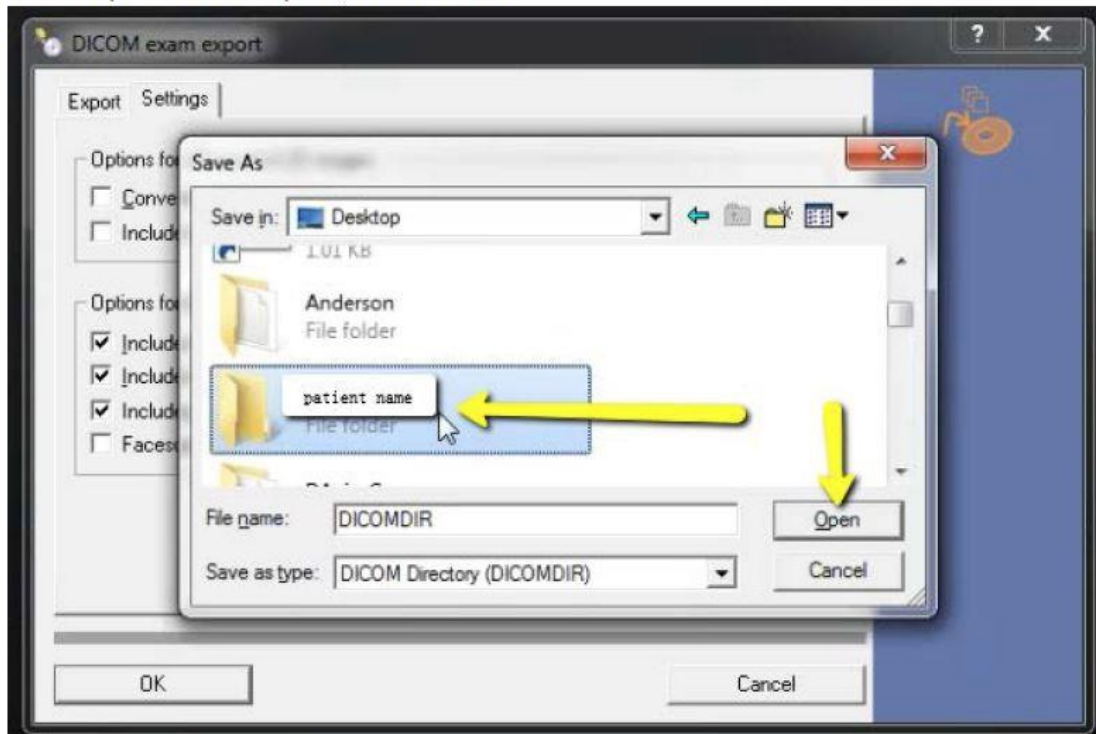

6. Your files will begin exporting.

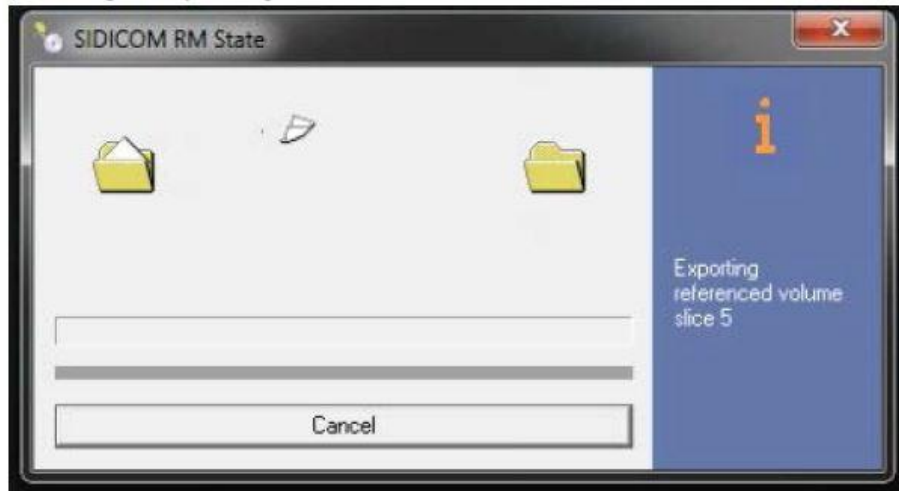


7. After the DICOM exports and saves, go to the folder and look for the sub folder named "CT3" (you may have to open a few folders before you see it). Verify the files were exported correctly; there should be several hundred files.

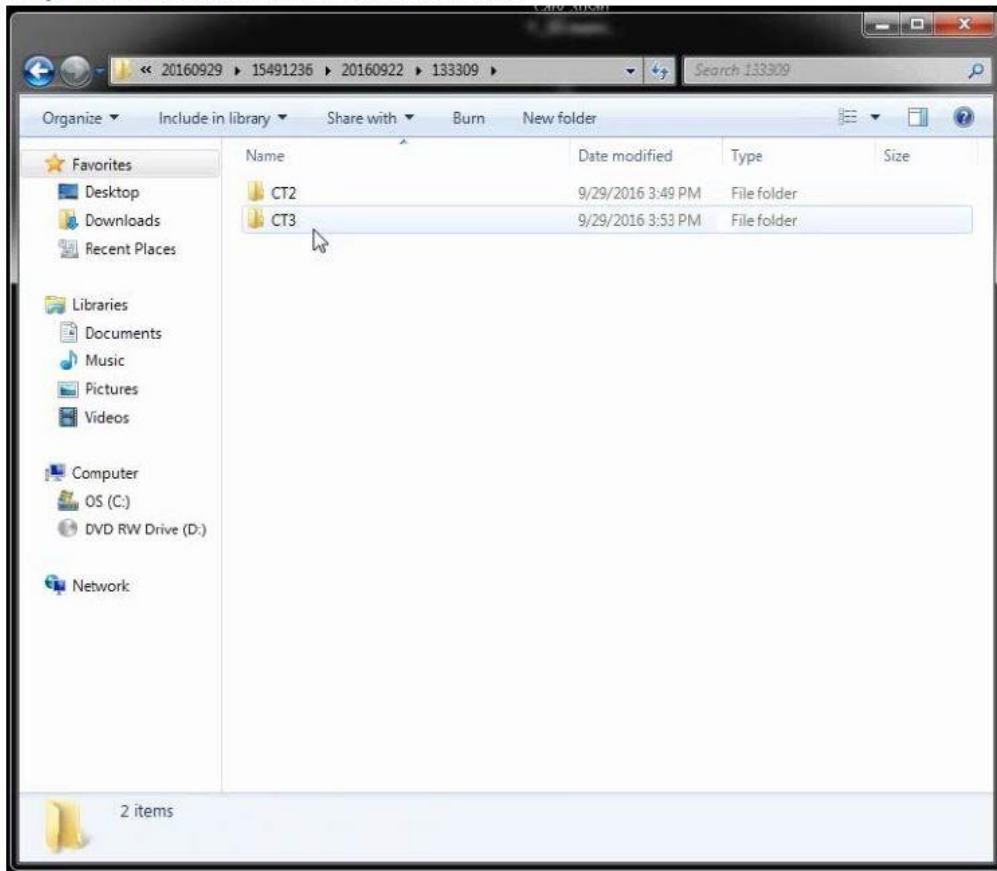


HYBRIDGE

LABORATORY

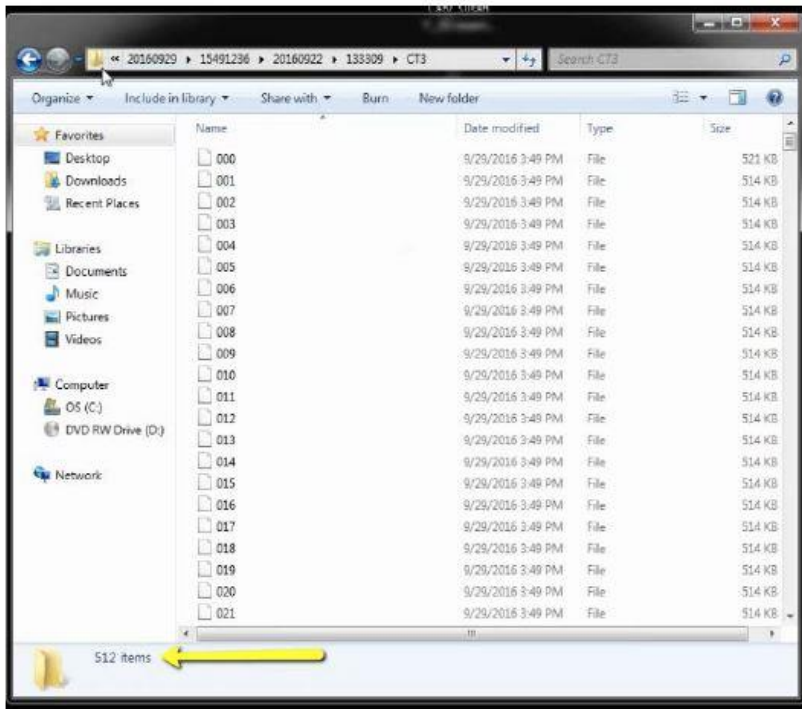

8. Go back to the desktop, right click on the patients folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

(*** If following a Dual Scan Protocol, please zip scans individually and label accordingly; i.e: “Patient Scan” and “Denture Scan” ***)

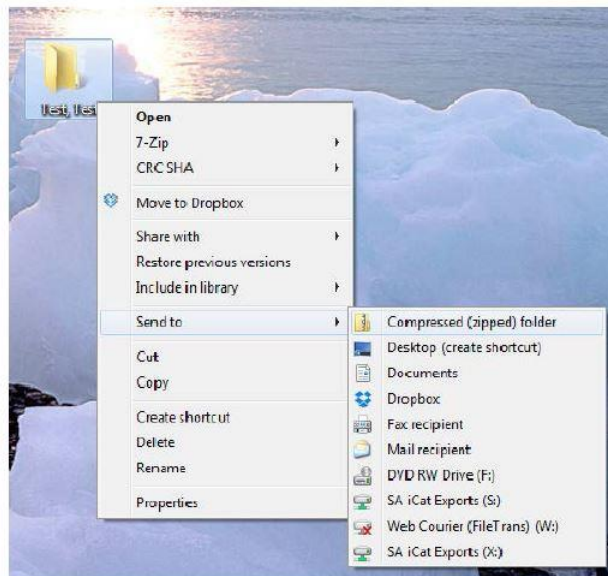

Now you can upload your folders to Hybridgerx.com.

Please call Christina at (585)319-5399 if you have any questions!