



- How to export scans as stl files

1. Select a patient and open Carestream Orthodontic/OMS Imaging software.
2. Right-click the 3D Model in the visit and select Export to STL to move the STL files to a designated location.
3. The Save As dialog box is displayed, enabling you to specify a destination and file name.

RECOMMENDED: Save the file as the patient's first initial and last name to the desktop, so you will be able to locate the files easily.

4. Browse to a location and click Save. The MeshConverter result dialog box is displayed, confirming that the file was exported successfully.
5. The STL files are exported automatically to the chosen location.
6. They can now be uploaded directly to our portal at:
www.HybridgeRx.com