



Dentsply
Sirona

- How to export scans as stl files

To Export STL Files with Omnicam or PrimeScan:

1. Set up a case in CEREC, Sirona Connect or Ortho Software. Capture the images, and advance to the point where the buccal bite is articulated in the Model Phase.
2. Click the arrow in the upper left corner of the screen. Select Export.
3. Highlight the location where you want to save the file on the left-hand side of the windows screen.
4. Use the drop-down menu in the 'Save as Type' field and select .STL from the list.

OPTIONAL: The user can rename the file by typing whatever they choose in the File Name field. (ex. Date, Patient, Arch, Bite, Facebow, etc.)

5. Click Save.

The file is now saved in the location as directed by the user. It can be copied to a flash drive if IOS is not connected to the internet or uploaded directly to our portal: www.HybridgeRx.com