


Finalizing the Immediate Provisional Assistant To Do While Doctor Sutures

HYBRIDGE[®]
LABORATORY

eXact[™]
Simply Guided System
HYBRIDGE


Preparation

1. Clean Provisional and ensure it is completely dry.
2. Screw analogs onto each temporary cylinder ensuring they are seated properly. Do not over-tighten.
3. Place plastic plugs into each screw channel, pushing them in so they are secure.
4. Prepare all surfaces with monomer to ensure a good chemical bond.
5. Starting with the occlusal, fill all access holes with Stellar acrylic & light cure. Once set, flip to the intaglio and fill all holes and voids, including the indexing holes and facial injection holes. Light cure.
6. Mark where you are going to reduce cantilevers and remove plugs.

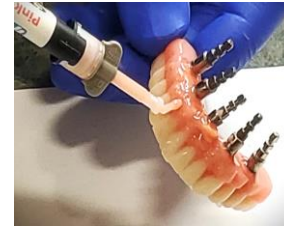

Finishing

1. Using a diamond disc, remove cantilevers.
2. Using a large carbide cutter, contour the occlusal, lingual, and facial surfaces for a smooth, rounded transition on the intaglio. Tissue surface must be convex with no concavities.
 - ☐ Reduce thickness of lingual wall
 - ☐ Create a tapered and rounded transition on both the facial & lingual toward intaglio
 - ☐ Smooth any contours where acrylic was added
 - ☐ Using a long, thin carbide, relieve between each analog to create a cleaning embrasure
 - ☐ Using an inverted cone carbide with a flat end, remove 1-2 mm of acrylic around each analog as needed to expose metal collars
3. Remove analogs & screws and put somewhere safe.
4. Continue to refine exposing of the collars using a flat-ended inverted cone carbide.
5. Using a long, tapered bur, widen the occlusal screw access holes, tapering toward the screw.

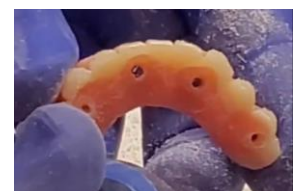

Finalizing the Immediate Provisional *Assistant To Do While Doctor Sutures*

Polishing

1. Use a soft, wet rag wheel with pumice to smooth all pink surfaces, especially lingual and between implants.
 - ☐ Make sure wheel is wet
 - ☐ Use a low setting
 - ☐ Focus on where tongue meets provisional
 - ☐ Smooth all irregularities
2. Rinse, dry well and inspect. Continue with pumice until smooth.
3. Using a dry, soft rag wheel and acrylic polish, high shine teeth and pink tissue.
4. Scrub Provisional under running water with soap and a toothbrush.
5. Dry and inspect.
6. Disinfect screws and Provisional.
7. Place screws with driver.
8. Deliver finished Provisional.

