Position for Excellence

Instructions for Use

925.PFGC-BX10

Paediatric Face Guard Cushion – Box of 10


Position for Excellence

Pediatric Face Guard Cushion - Box of 10

Product Code: 925.PFGC-BX10

*Recommended for patients up to age 8


The Face Guard Cushion offers support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes. **For use with the Face Guard Set (Helmet + Mirror) and other compatible head support systems.

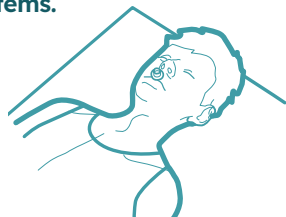
1 With patient supine, disconnect ET tubing


2 Place cushion onto patient's face*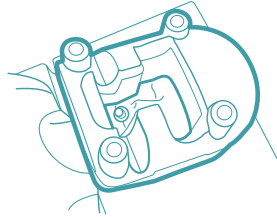


3 Place helmet over cushion

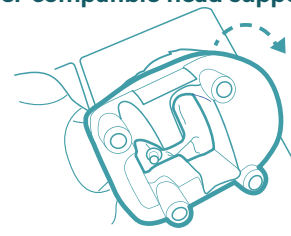
4 Turn patient to prone position

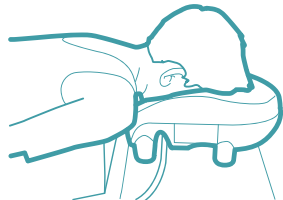
5 Re-connect ET Tubing

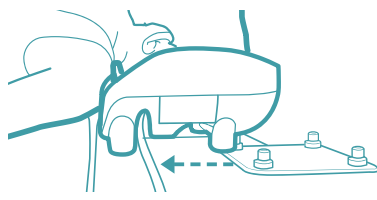6 Lift helmet & locate onto mirror's cogs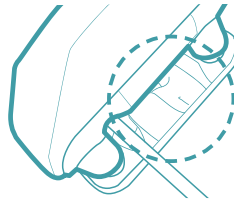

7 Check face correctly positioned in cushion*

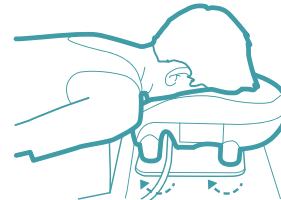8 Adjust cogs to obtain neutral neck line

Only to be used by trained clinicians *See overleaf for guidance on correct positioning **Warning: Do NOT remove cogs from mirror.**

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

! DO NOT REUSE! If reused injury to patient could occur due to issues with infection control and deterioration of the product.
! SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
! DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

www.bonefoam.com/eIFU
1 (877) 861-2663

The Forehead & Cheeks

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High-risk patient (includes the elderly, diabetics, smokers and those taking certain medications, e.g. steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

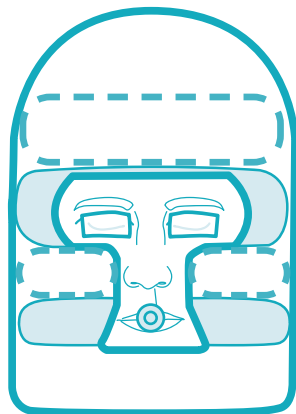www.bonefoam.com/eIFU
1 (877) 861-2663

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

Pediatric Face Guard Cushion - Box of 10

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