






Instructions for Use
925.FGS
Face Guard Set (Helmet + Mirror)

Face Guard Set (Helmet + Mirror)

Position for Excellence

*Suitable for patients 8 years+ To only be used with the Face Guard Cushion – Box of 10

The Face Guard Set (Helmet + Mirror) used with the Face Guard Cushion offers support to your patient's forehead and cheeks whilst ensuring a neutral neckline and assisting with patient monitoring. Ensure the eyebrows are visible to avoid direct pressure to the eyes.

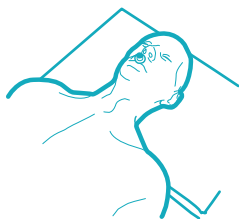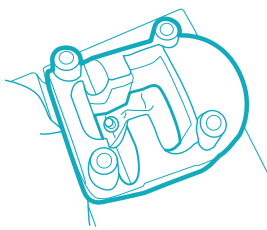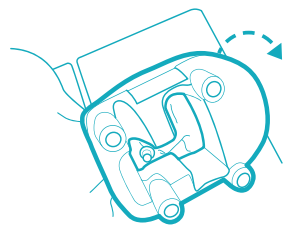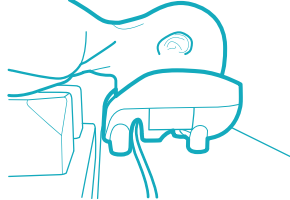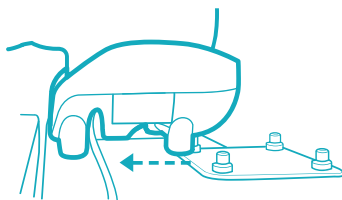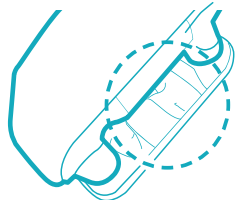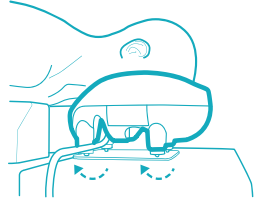- 
- 1** With patient supine, disconnect ET tubing
- 
- 2** Place cushion onto patient's face*
- 
- 3** Place helmet over cushion
- 
- 4** Turn patient to prone position
- 
- 5** Re-connect ET Tubing
- 
- 6** Lift helmet & locate onto mirror's cogs
- 
- 7** Check the face is correctly positioned in cushion*
- 
- 8** Adjust cogs to obtain neutral neck line

Only to be used by trained clinicians *See overleaf for guidance on correct positioning **Warning: Do NOT remove cogs from mirror.**

www.bonefoam.com/eIFU
1 (877) 861-2663

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.

DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

The Forehead & Cheeks

Possible problems

- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High-risk patient (includes the elderly, diabetics, smokers and those taking certain medications, e.g. steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement


Cleaning Instructions

Clean using standard hospital wipes

Do not immerse in water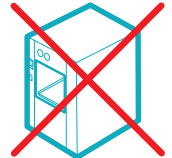

Do not autoclave

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

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