
▲BONEFOAM™

# HIPPY

DIRECT ANTERIOR HIP  
POSITIONING SYSTEM

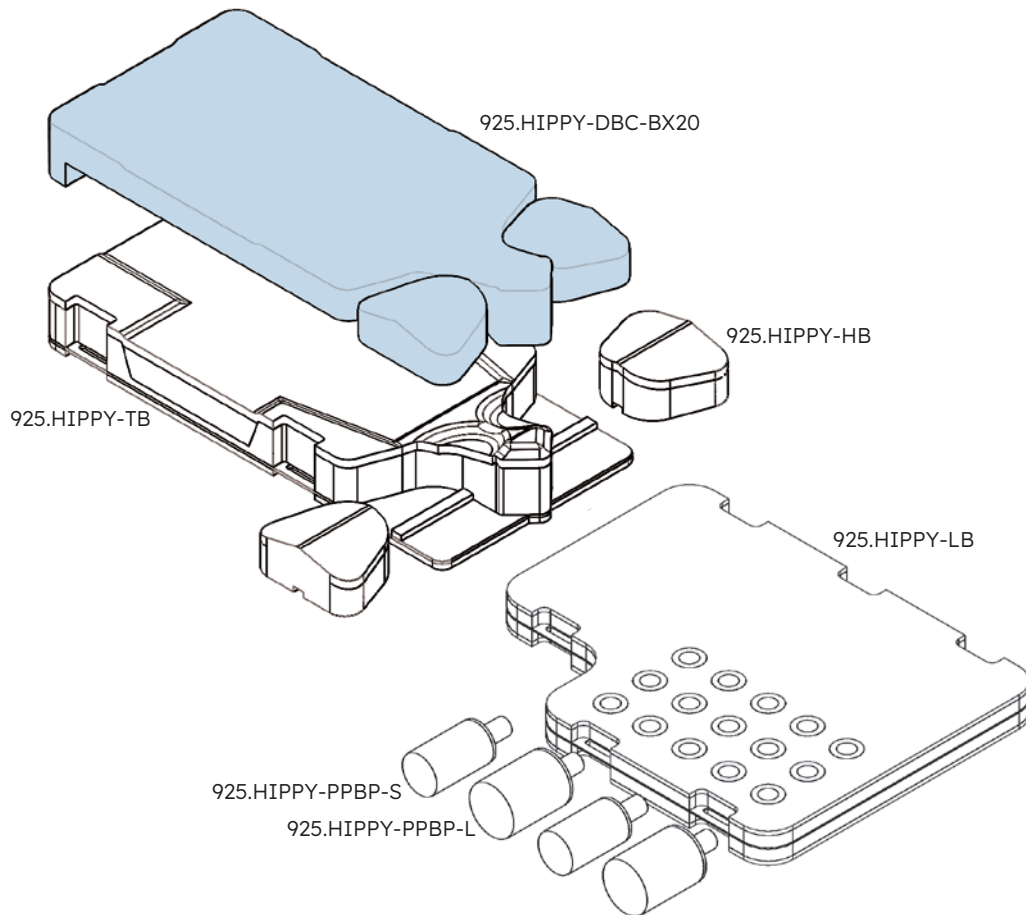## SELECT LANGUAGE:

ENGLISH

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SPANISH • ESPAÑOL

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GERMAN • DEUTSCH

FINNISH • SUOMI

SWEDISH • SVENSKA

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# HIPPY

## DIRECT ANTERIOR HIP POSITIONING SYSTEM

The Hippy Direct Anterior Hip Positioning System is a reusable tabletop positioning set designed to position a patient for off-table anterior hip procedures.

The following disposable products are recommended for use with the Hippy, but are not included:

- 925.SAS-WS-BX10 - Arm Guard - Box of 10
- 925.HIPPY-DBC-BX20 - Disposable Barrier Covers
- 925.HIPPY-SUS - Hippy Single-Use Sets, including 20 Arm Guards and 20 Barrier Covers

To order, please contact [sales@bonefoam.com](mailto:sales@bonefoam.com)

BoneFoam, Inc.  
2601 49th Ave N Suite 100  
Minneapolis, MN 55430  
USA

Email: [customerservice@bonefoam.com](mailto:customerservice@bonefoam.com)  
Phone: (763) 559-1830  
Toll-free: 1 (877) 861-2663  
Fax: (763) 559-1822


BoneFoam B.V.  
P.O. Box 10110, NL-5060 GA Oisterwijk  
Kantstraat 19, 5076 NP Haaren  
EAR@qnetbv.eu

[bonefoam.com](http://bonefoam.com)


HIPPY-eIFU Rev.1 2026-03-15

## INSTRUCTIONS FOR USE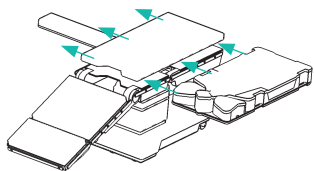


1

Replace table pads with Hippy torso base and secure with table straps.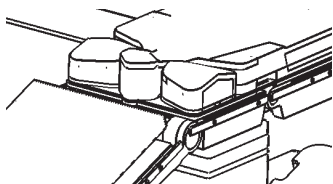



2

Drape torso base and Hip Bolsters with disposable covers.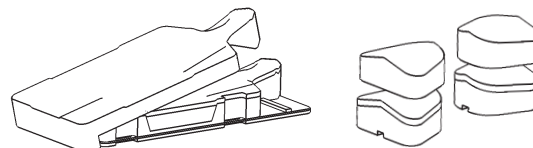

3

If using leg board, replace table pads with Hippy leg board and secure with table straps.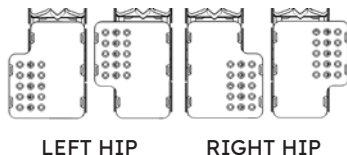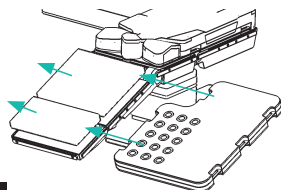


LEFT HIP

RIGHT HIP


4

Transfer patient to Hippy following facility transfer protocol. Center patient's pelvis, then pull patient by feet/legs until pelvis engages the pommel and counter traction is felt.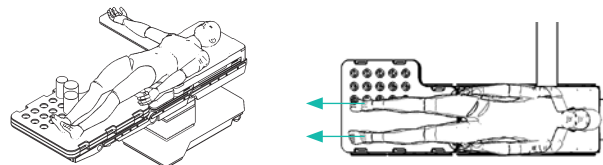


5

If using leg board, before sterile prep, place pegs in desired holes by positioning legs in anticipated location/configuration.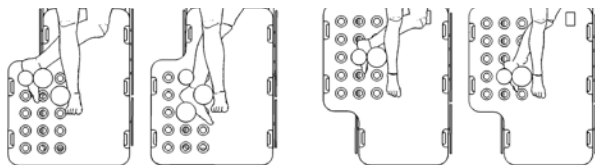

6

Remove Hip Bolster from surgical side.

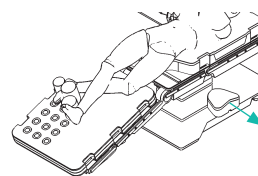## CLEANING AND DISINFECTING INSTRUCTIONS

- Do not sterilize the product. This product is not intended to be sterilized.
- For coated products, wipe the product with a clean wipe and disinfectants after every use. The following disinfecting materials are known to be compatible with coated BoneFoam products:
  - Rubbing alcohol (70% isopropyl alcohol)
  - General purpose/all purpose cleaner
    - Read and follow the cleaning product's instructions.
- Make sure that the coated product is dry before you store it or use it again.

## WARNINGS/PRECAUTIONS/CONTRAINDICATIONS

- Intended to contact intact skin only.
- For single-use accessories, risk of cross-contamination if not disposed of immediately after use.

- Incorrect positioning of the patient with this product can increase the risk of:
  - Pressure injury resulting in markings or ulcers
  - Peripheral nerve injury, which can lead to temporary or permanent paralysis
  - Venous pressure, which can lead to ischemia
  - Instability, which can lead to patient falls
- Product is to be used by licensed medical professionals only.
- Not intended for patient transport.
- Ensure device is compatible with work surface, procedural instruments, and equipment.
- This product has a vinyl coating. While this coating is intended to enhance product durability and usability and is safe for skin contact, it may cause allergic reactions in individuals sensitive to vinyl or its derivatives.
- Possibility of artifacts in imaging applications depending on setup and imaging settings.
- Risk of fire. Below are flammability ratings of the materials:
  - Coating: UL 94 HBF
  - Foam: Rated either UL 94 HBF or CAL 117

## CARE, HANDLING AND USEFUL LIFE

- Avoid using the product if package is damaged during transportation.
- Allow coated products to reach room temperature after storage or transportation in cold temperatures.
- Transport and store in dry conditions and out of direct light where possible.
- To prevent patient and/or user injury and/or equipment damage, examine the product for potential damage or wear prior to use. Do not use the product if damage is visible, if parts are missing, or if it does not function as expected.
- Avoid storing where sharp objects could pierce the coating of the product.
- Avoid stacking items on top of product when storing.
- Do not submerge the product in water. Equipment damage can occur.
- Prior to using this product, read the Instructions for Use.
- Discontinue use if foam has degraded or lost resilience, or if coating is torn, cracked, or loose and wrinkles, as this may lead to acute pressure points.

SYMBOL	DESCRIPTION
	Indicates the device is a medical device
	Indicates the medical device manufacturer
	Indicates the country where the medical device was manufactured and date of manufacture
	Indicates the manufacturer's lot number
	Indicates the medical device Global Trade Item Number
	Indicates the manufacturer's catalogue number
	Indicates the unique product identifier information for the device
 <small>bonefoam.com/eIFU</small>	Indicates the need for the user to consult the instruction for use
	Indicates the device does not contain natural rubber or dry natural rubber latex

## SAFE DISPOSAL INSTRUCTIONS

Customers should adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories.

## PACKAGING

The product is packaged in a clear polythene bag inside a corrugated cardboard box.

## SAFETY INFORMATION

The techniques detailed in this manual are only manufacturer's suggestions. The final responsibility for patient care with respect to this product remains with the attending physician. Any serious incident that has occurred in relation to this product should be reported to the manufacturer and regulatory authority, where reported. Keep this manual available for future reference.

SYMBOL	DESCRIPTION
	Indicates the device is intended for one single-use only
	Indicates the temperature limits to which the medical device can be stored, transported or used
	Indicates a medical device should not be used if the package has been damaged or opened
	Indicates quantity in package
	Indicates the European Authorized Representative name and address in the European Community
	Signifies European technical conformity
	Signifies Great Britain technical conformity
	Indicates a UK Responsible Person on medical device information supplied by the manufacturer
	Indicates an Authorized Representative in Switzerland on medical information supplied by the manufacturer

SKU	DESCRIPTION	INTENDED USE	INDICATIONS
<b>REUSABLE SETS</b>			
925.HIPPY	Hippy: Base Package and Lower Extremity Package	Nonsterile, reusable patient positioning system intended to stabilize the pelvis, provide proximal femur access and support, and stabilize the legs in a desired orientation in the supine position.	Indicated for use by healthcare professionals to position the patient's full body on standard OR tables for anterior hip procedures.
925.HIPPY-BP	Base Package: Torso Base and 2 Removable Hip Bolsters	Nonsterile, reusable patient positioning system intended to stabilize the pelvis and provide proximal femur access in the supine position.	Indicated for use by healthcare professionals to position the patient's upper body and pelvis for anterior hip procedures.
925.HIPPY-LEP	Lower Extremity Package: Leg Board, 2 Padded Peg Board Posts Large and 2 Padded Peg Board Posts Small	Nonsterile, reusable positioning system intended to support and stabilize the leg in a desired orientation in the supine position.	Indicated for use by healthcare professionals with the Hippy Base Package to position the patient's legs on standard OR tables for anterior hip procedures.
<b>SINGLE-USE ACCESSORIES</b>			
925.HIPPY-DBC-BX20*	Disposable Barrier Cover Box of 20	Nonsterile, single-use covers intended to provide a barrier between the patient and the positioner.	Indicated for use by healthcare professionals to cover the Hippy Torso Base and Hip Bolsters.
<b>REPLACEMENT COMPONENTS</b>			
925.HIPPY-TB	Torso Base	Nonsterile, reusable positioner intended to support and stabilize the torso and trunk in the supine position.	Indicated for use by healthcare professionals with the Hippy and Hippy Base Package to position the patient's upper body and pelvis for anterior hip procedures.
925.HIPPY-HB	Hip Bolster	Nonsterile, reusable, removable bolster intended to be removed from underneath the operative side hip to provide posterior access and movement of the hip in the supine position.	Indicated for use by healthcare professionals with the Hippy Torso Base (part of the Hippy and Hippy Base Package) to position the patient's upper body and pelvis for anterior hip procedures.
925.HIPPY-LB	Leg Board	Non-sterile, reusable, padded peg board with peg holes for inserting padded pegs for the purpose of placing and securing the lower extremities in the supine position.	Indicated for use by healthcare professionals with the Hippy, Hippy Base Package, and/or Hippy Lower Extremity Package to position the patient's legs on standard OR tables for anterior hip procedures.
925.HIPPY-PPBP-L	Padded Peg Board Post Large	Nonsterile, reusable, padded peg, designed for corresponding peg holes in the Hippy Leg Board, for the purpose of securing and stabilizing the leg in the desired arrangement and orientation in the supine position.	Indicated for use by healthcare professionals with the Hippy Leg Board (part of the Hippy and Hippy Lower Extremity Package) to position the patient's legs on standard OR tables for anterior hip procedures.
925.HIPPY-PPBP-S	Padded Peg Board Post Small		

\*  applies only to these products.  does not apply to these products.

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