Instructions for Use
925.FGC-CFLEX
Face Guard Cushion | C-Flex

Use with the Allen® C-Flex® Head Positioning System to support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes.


 Only to be used by trained clinicians.



1 With patient supine, disconnect ET tubing

2 Place cushion onto patient's face*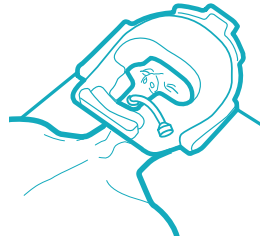

3 Turn the patient to the prone position and locate the face cushion in the C-Flex® Head Positioner. Position the ET tubes through the channel as shown

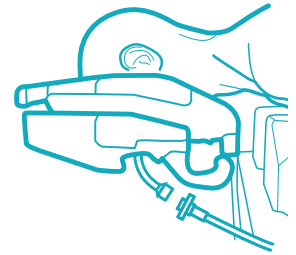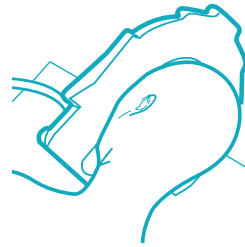4 Ensure the tubing is not trapped between the cushion and the patient's face

Follow the instructions supplied with the Allen® C-Flex Head Positioning System to achieve a suitable neutral neckline

*See overleaf for guidance on correct positioning

Allen® are trademarks or registered trademarks of Allen Medical Systems Inc

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

EU REP.

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

 **DO NOT REUSE!** if reused injury to patient could occur due to issues with infection control and deterioration of the product.
 **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
 **DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

www.bonefoam.com/eIFU
1 (877) 861-2663

5 060624 128325

The Forehead & Cheeks

www.bonefoam.com/eIFU
1 (877) 861-2663

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High-risk patient (includes the elderly, diabetics, smokers and those taking certain medications, e.g. steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long

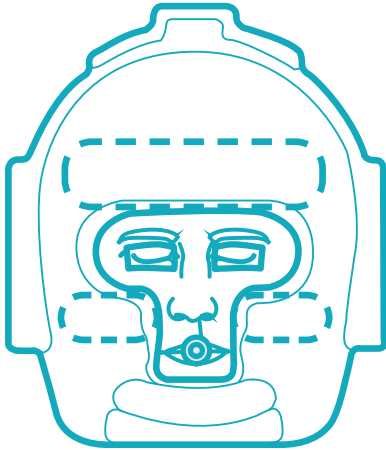Maximum
Patient Weight

Care Surgical Ltd
Unit 6 Ringtail Road
Burscough,
L40 8JY

Care Surgical Supplies Ltd
Office 2, 12a Lower Main Street,
Lucan Co.Dublin, K78 X5P8,
Ireland

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