



MYCO
MEDICAL

TheSharpChoice®

NMSDC Certified Diversity Supplier


RELI® SAFETY BLOOD COLLECTION NEEDLE

Instructions for Use

NOTE: Begin at STEP 3 if your tube holder is pre-attached.

STEP 1:

Twist and remove clear cap from valve section of multi-sample blood collection needle.

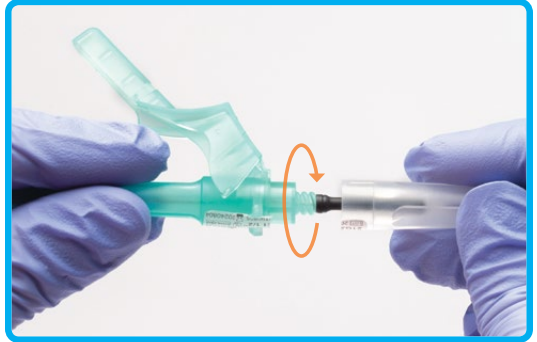

STEP 2:

Thread into the tube holder. Confirm needle is securely attached.



STEP 3:

Gently position the safety shield straight back toward the holder.



STEP 4:

Remove colored cap.

NOTE: Needle bevel is always in position for venipuncture when the safety shield is facing up. Do not twist or rotate the safety shield.



STEP 5:

Perform venipuncture per facility protocol. Insert tube fully into tube holder.



STEP 6:

Immediately after removing needle from vein, place thumb behind safety shield. Press shield onto exposed needle. An audible *click* will be heard.



STEP 7:

Needle is safely secured within the safety shield and is ready to be disposed of responsibly.

