


The Reli® Safety Slide™ Safety Blood Collection and Infusion Set is a sterile, single-use, winged blood collection device designed for safety and efficiency. The safety shield can be activated in-vein by sliding the shield over the needle during removal from skin helping to reduce the risk of accidental needle sticks.

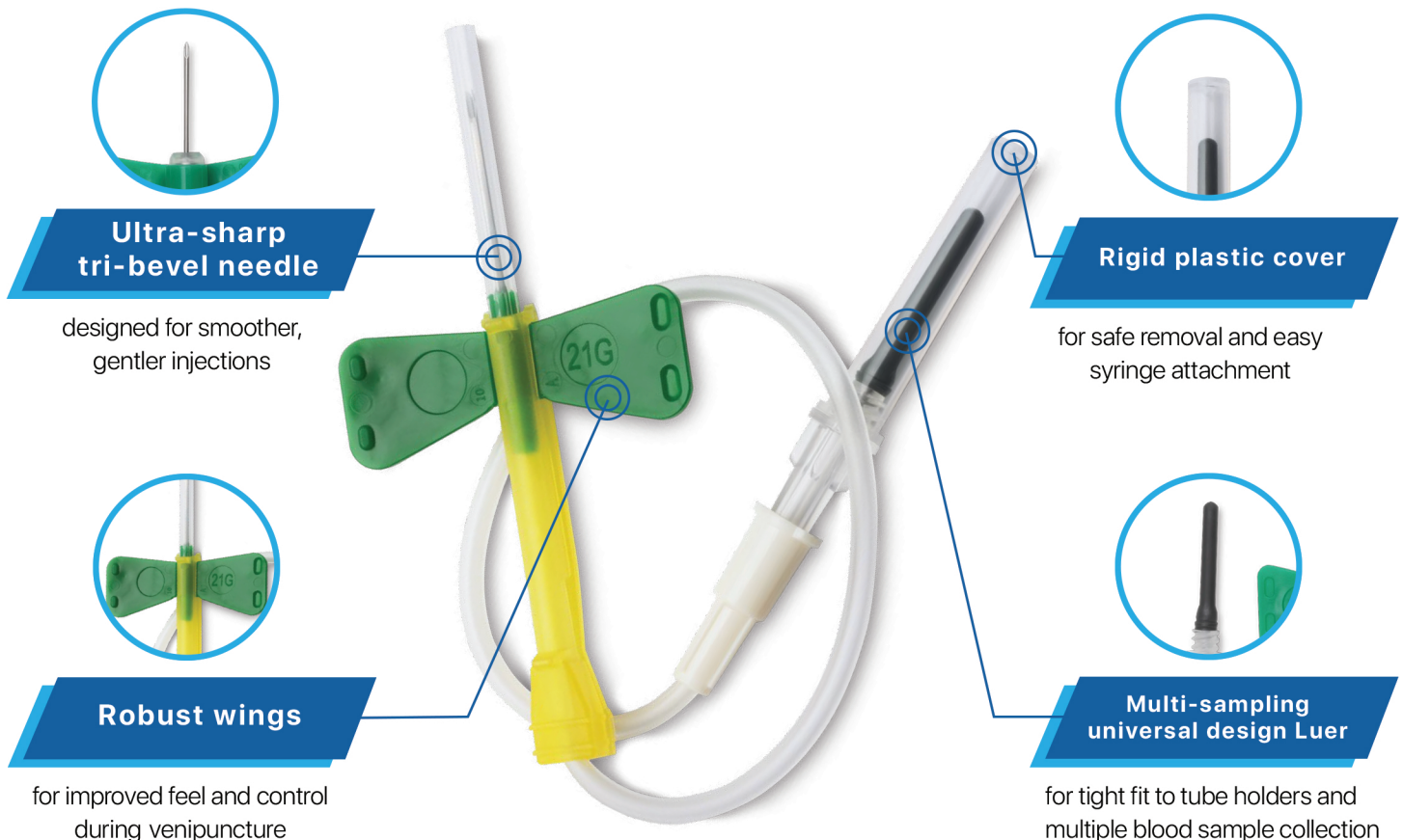

Pictured above: RELI® Safety Slide™ Blood Collection /Infusion Set 21G x 3/4" x 12" (SBCS21G-12")

Additional Features

- Not made with natural rubber latex
- Features single-handed safety shield activation for convenience and ease of use
- Sterile blister pack, single use only
- Siliconized needle for smooth insertion
- Cartridge sliding design for simple and safe use
- Audible "click" when needle is in the final lock position
- OSHA compliant
- One- or two-handed irreversible activation
- Flexible color-coded wing for improved tactile feel and easy gauge identification

INSTRUCTIONS FOR USE

STEP 1

Open package and ensure the multi luer adapter is securely attached.

NOTE: Wear gloves when using. For syringe blood collection or short-term IV use, see instructions in box.

STEP 2



Remove needle sheath and attach the tube holder

STEP 3



Prepare venipuncture site per your facility's protocol. Remove the protective needle sheath.

STEP 4



Perform venipuncture by holding the wings and per your facility's protocol. (Insert needle in the center of the tube when collecting the sample.)

STEP 5



Withdraw blood collection set from site by grasping the translucent yellow safety shield grip area with thumb and index finger while grasping the tubing. Apply pressure to site.

STEP 6



A. One-hand Safety Activation - Holding the tubing in hand and advance the translucent yellow safety shield with thumb and index finger until an audible click is heard indicating the needle is completely retracted and locked in place.



B1. Two-hand Safety Activation - Using dominant hand, place the thumb behind the tab below the wings with index finger underneath



B2. Using the other hand, grasp the tubing and pull back until an audible click is heard indicating the needle is completely retracted and locked in place.

STEP 7

Dispose of device in the nearest Sharp's container.

ORDER INFORMATION

MYCO CODE	NEEDLE GAUGE	TUBE LENGTH	COLOR	UNIT OF MEASURE
SBCS21G-12	21G X ¾"	30 cm (11.8")	Green	50 ea/bx; 200 ea/cs
SBCS23G-12	23G x ¾"	30 cm (11.8")	Blue	50 ea/bx; 200 ea/cs
SBCS25G-12	25G x ¾"	30 cm (11.8")	Orange	50 ea/bx; 200 ea/cs



**Scan to watch
Instructions for Use**

<https://mycomed.co/reli-sbcs-ifu>



MYCO
MEDICAL
TheSharpChoice®

NMSDC Certified Diversity Supplier



919 460 2535



2015 Production Drive
Apex, NC 27539 USA



www.mycomedical.com



info@mycomedical.com



Scan to learn more
or receive complimentary
evaluation samples

<https://mycomed.co/reli-sbcs>

Some medical devices are not licensed for sale in Canada.
© 2025 MYCO Medical, Inc. All rights reserved.
MYCO® is a registered trademark of MYCO Medical, Inc.
LIT-01045 Rev 1 Eff XX.XX.XXXX