

sweeth

Clear Aligner System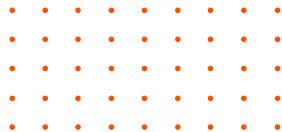

Your Sweeth Use & Care Guide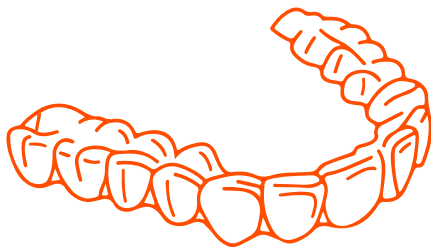

LET'S GET STARTED.

Welcome to the Aligner Use & Care Guide! In this comprehensive guide, you'll find all the essential information you need to ensure you get the most out of your aligners and maintain your oral health throughout your orthodontic journey.

Track your treatment progress by using the aligner tracker log provided at the end of the document.

Languages: English, Dutch, French, German

See symbols glossary at sweeth.pro/ifu

ALIGNER USE AND CARE GUIDE

CUSTOM-MADE DEVICE DESCRIPTION

The Sweeth Shape Memory Aligners involve a series of direct 3d printed, thin, clear, orthodontic appliances (aligners), prescribed by a doctor. These aligners gently guide the patient's teeth in small steps from their initial position to a more optimal, treated state.

INDICATIONS FOR USE

The Sweeth Shape Memory Aligners are intended for the alignment of teeth during orthodontic treatment of malocclusion.

CONTRA-INDICATIONS

If a patient has active periodontal disease, it is advisable to avoid or postpone treatment with clear aligners until the patient is free from symptoms of the active periodontal disease.

WARNINGS

- In rare cases, a small number of patients may exhibit allergic reactions to the material used for the aligners.
- In rare cases, patients may experience sudden localized swelling of subcutaneous tissues, including the larynx.
- In rare cases, hygiene-related stomatitis can occur. This can be accompanied by neck pain. It can be avoided by following the hygiene recommendations.
- In all these cases, cease usage immediately and seek immediate medical advice from

a healthcare professional. Sweeth must also be notified. Contact details are available at the bottom of the last page of this document.

- Orthodontic appliances or parts may be accidentally swallowed or inhaled, posing potential harm.

PRECAUTIONS

- Store aligners away from sunlight and harsh chemicals.
- Ensure aligners are stored in a location inaccessible to young children and pets.
- Always keep aligners stored in a cool, dry location.
- For sensitive patients it is possible that the aligner emits an odor.
- Overly bending or twisting the aligner can compromise its shape memory and overall effectiveness.
- Severe open bite, severe overjet, mixed dentition, and/or skeletally narrow jaw may necessitate supplementary treatment alongside aligner therapy.
- Certain situations may arise where the aligner might get contaminated or slightly change color.
- Skeletal expanders, bonded auxiliaries, elastics, and other fixed or removable orthodontic appliances may be necessary for complex treatment plans where aligners alone may not suffice to achieve the desired outcome.
- Oral surgery may be necessary to address severe crowding or jaw imbalances. If surgery is required, consider the risks associated with anesthesia and proper healing before proceeding with treatment.
- Teeth that are atypically shaped, erupting, or missing may impact the adaptation of aligners and could potentially hinder achieving the desired outcome.
- Existing dental restorations such as crowns or bridges may become dislodged, necessitating re-cementation or, in certain cases, replacement.

- Aligners are not suitable for moving dental implants.
- Specific medical conditions and certain medications can impact the results of orthodontic treatment.
- Dental tenderness may occur after the initial placement of aligners and when transitioning to each new aligner in the series.
- The product may temporarily affect speech, potentially causing a lisp. However, any speech impediment typically resolves within one to two weeks.
- Not wearing the appliances for the prescribed number of hours per day and/or not following the instructions provided by your doctor can prolong the treatment duration and impact the ability to achieve the desired results.
- The product may cause irritation or scratching to the gums, cheeks, or lips.
- Attachments may be temporarily bonded to one or more teeth during treatment to aid in aligner retention and/or tooth movement.
- Attachments may become dislodged and need to be replaced.
- All attachments must be removed upon completion of treatment.
- Consider the potential risks associated with interproximal reduction, which may be recommended to create space for tooth movement as part of the treatment.
- Inadequate brushing and flossing during treatment, as well as consuming sugary foods or beverages while wearing aligners, may lead to tooth decay, periodontal disease, and permanent stains or decalcification markings.
- The bite may undergo changes during treatment, potentially causing temporary discomfort for the patient.
- At the end of treatment, the doctor may need to adjust the bite.
- It's recommended to ensure that all teeth are at least partially covered to help prevent supra-eruption.

- After treatment, teeth may naturally shift positions. Consistently wearing retainers at the end of treatment can help minimize this tendency.
- It's recommended to ensure that all teeth are at least partially covered to help prevent supra-eruption.
- A temporary increase in salivation or dryness of the mouth may occur.
- A tooth that has undergone previous trauma or extensive restoration may experience aggravation. In rare cases, this may lead to a reduced useful life of the tooth, necessitate additional dental treatments such as endodontic or further restorative procedures, and in extreme cases, result in tooth loss.
- The health of the bone and gums supporting the teeth may be compromised or worsened.
- During orthodontic treatment, roots of the teeth may be shortened, posing a potential risk to the long-term health of the teeth.
- In rare instances, temporomandibular joint (jaw joint) issues may manifest symptoms such as joint pain, headaches, or ear problems.

WEARING AND USING YOUR SWEETH ALIGNERS

To ensure correct usage of your aligners, always:

1. Rinse your aligners with water upon removing them from the packaging.
2. Once a day, place the aligner in a bath of +80°C for 1 minute and clean the aligner 2X a day with the accompanying brush and mousse for one minute.
3. Wear it as much as possible, only removing it for eating and brushing teeth.
4. Each aligner must be worn for 10 days before switching to the next one.
5. If your aligner no longer fits, contact your doctor immediately.
6. Do not discard your previously worn aligners.

7. Handle your aligners with clean, freshly washed hands.

ALIGNER INSERTION

1. To help you identify the correct aligner, each plastic bag is marked with an aligner number with (U) for upper teeth and (L) for lower teeth.
2. Once a day, place the aligner in a bath of +80°C for 1 minute.
3. When inserting the aligner, press it firmly to ensure a good fit. You can soften the aligner with warm water to facilitate this process.
4. Um eine Beschädigung Ihrer Aligner zu vermeiden, vermeiden Sie es, sie festzubeißen.

NOTE: If you experience sharp pain or significant discomfort, stop using your aligners and contact your treating clinician immediately.

ALIGNER REMOVAL

1. To detach the aligner, use the hook provided. Do this carefully, ensuring the aligner does not bend excessively.
2. Repeat step 1 on the opposite side of the mouth.
3. Once the aligner is released from the molars on both sides of your mouth, gently begin to separate it from your teeth as you work your way forward.

REMOVAL TIPS:

- After removal, promptly rinse each aligner with water and shake off any excess before storing.
- To avoid damaging aligners, refrain from unnecessary removal.
- Be cautious not to use excessive force that might bend or twist the aligner when removing it, taking care to avoid stretching or breaking them.
- Avoid using sharp objects to remove your aligners, as this could damage them and

potentially cause injury to your teeth or gums.

- Consult your treating health care professional if removing aligners becomes difficult.

USE AND CARE OF ALIGNERS:

1. Clean aligners thoroughly 2X day using the provided mousse, along with a soft-bristle brush and a small amount of water, followed by a thorough rinse. If desired, clean the outside of the aligners while wearing them, then remove to clean the inside surfaces.
2. Avoid soaking aligners in mouthwash. Additionally, refrain from using alcohol, or bleach solutions, as these may damage the aligner's surface, making it less clear and more visible.
3. Once a day, place the aligner in a bath of +80°C for 1 minute.

DAILY ORAL HYGIENE:

1. Remove aligners before eating.
2. Remove aligners before drinking anything except cool water.
3. After each meal or snack, brush and floss your teeth before re-inserting your aligners. If a toothbrush isn't available, rinse your mouth and aligners with water. Clean your aligners thoroughly as soon as possible.
4. Seek advice from your treating clinician if you have any questions about hygiene techniques.
5. Regular dental checkups and cleanings are recommended to maintain the health of your teeth and gums.

STORING YOUR ALIGNERS:

1. Keep all your aligners in their original packaging, or follow the instructions provided by your treating health care professional.
2. When not in use, store your aligners in their original packaging or in a suitable container

to protect and maintain their cleanliness.

3. If your current aligner is lost or damaged, contact your treating doctor.
4. Keep aligners away from small children and pets.
5. Water is allowed to be consumed but other drinks are discouraged (e.g. soda, tea, and coffee).

NOTE: If you have additional questions after reading this Aligner Use and Care Guide, please consult your treating doctor.

WARNING:

In rare instances, some patients may be allergic to the material. If adverse reaction occurs (allergic or otherwise), discontinue use, and seek immediate medical attention; Sweeth must also be notified.

WARNING:

Orthodontic appliances may be accidentally swallowed or aspirated, which may be harmful.

USER AND PATIENT NOTICE:

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country. Also contact Sweeth.

Single-Patient Use. Risk of cross-contamination.

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Sweeth ist eine Marke von Plus-U GmbH.