

What You'll Find in Your Sweeth Kit:



THE GUIDE

1



SET OF ALIGNERS WITH ALIGNER CASE

2



ALIGNER HOOK & BRUSH

3



CLEANSING MOUSSE

4



Wear aligner as much as possible, only remove it for eating and brushing teeth.

Water is allowed to be consumed but other drinks are discouraged (e.g. soda, tea, and coffee).

Each aligner must be worn for 10 days before switching to the next one.

Questions?
visit www.sweeth.pro/ifu

sweeth
Clear Aligner System



Your Sweeth Quick
Start Guide





sweeth

Patient Name:

Patient ID:

Case Number:

Date:

REF

LOT



Welcome!

The Sweeth Shape Memory Aligners are intended for the alignment of teeth during orthodontic treatment of malocclusion.

80°F




01




Insert Aligners

Once a day, place the aligner in a bath of +80°C for 1 minute and clean the aligner twice a day with the accompanying brush and mousse.

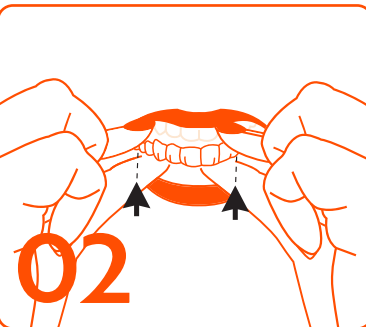

02


When inserting the aligner, press it firmly to ensure a good fit everywhere. You can soften the aligner with warm water to make this process easier.

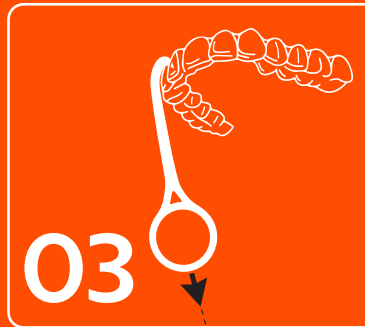

03

Remove Aligners

To detach the aligner, you can use the hook. Do this carefully to ensure the aligner does not bend too much.

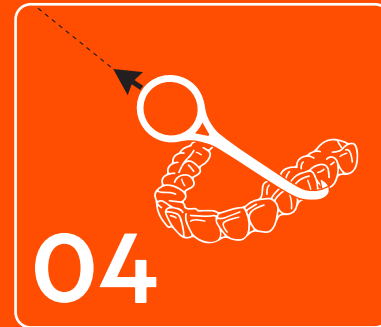

04

Repeat step 3 on the other side of the aligner.