

TREATMENT PROTOCOL



- **WARNING:** It is not absolute protocol and it will vary with the patients' condition. Also, it allows all the areas that have fat layer more than 2cm in the body to treat.

1 Abdominal Area

Cartridge	Power(J)	Spacing(mm)	Length(mm)	Treatment Zone(shot)
L4	1.5	1.5	25	10

Treatment Tip

- Depending on the abdominal area can vary the number of treatment zone
- It allows to treat in the area that had more than 2cm thick adipose tissue
- Treatment should be performed horizontally first for one entire section and then performed vertically.
- 10shot ~ 15shot, horizontal approximate interval is 5mm
- 10shot ~ 15shot, vertical approximate interval is 5mm
- The treatment is prohibited in the ribs.
- The treatment is prohibited around umbilical zone

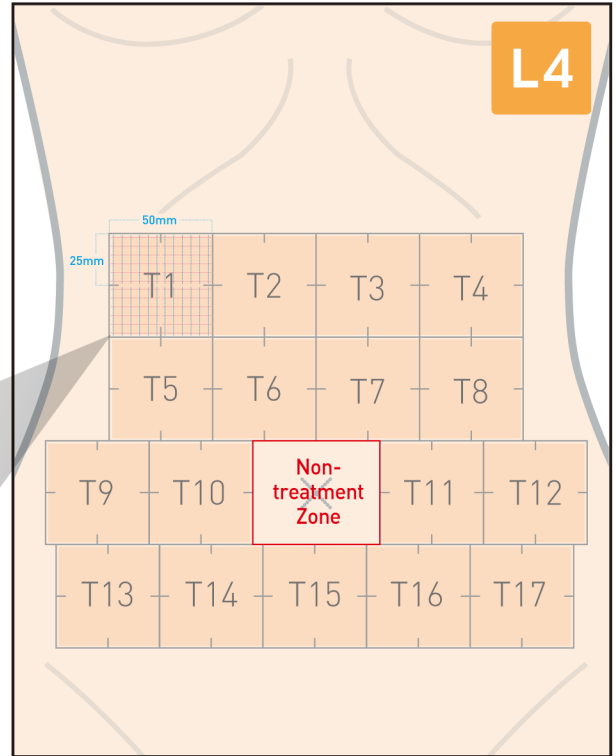
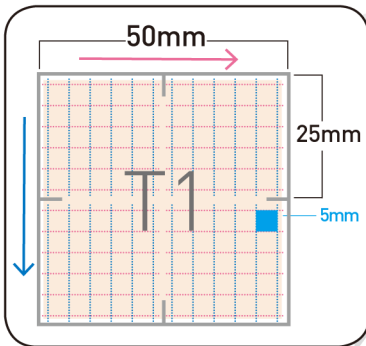


2 Arm

Cartridge	Power(J)	Spacing(mm)	Length(mm)	Treatment Zone(shot)
L4	1.5	1.5	25	10

Treatment Tip

- Depending on the area of arms can vary the number of treatment zone
- It allows to treat in the area that had more than 2cm thick adipose tissue
- Treatment should be performed horizontally first for one entire section and then performed vertically.
- 10shot ~ 15shot, horizontal approximate interval is 5mm
- 10shot ~ 15shot, vertical approximate interval is 5mm

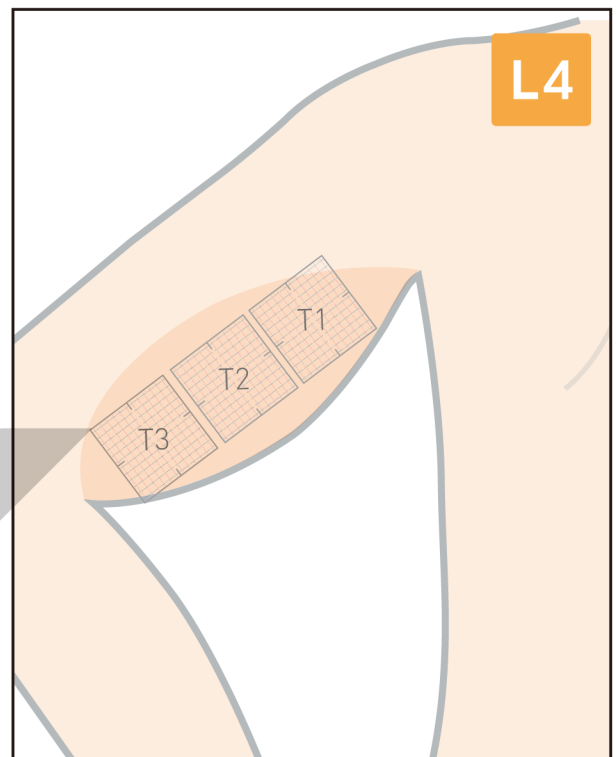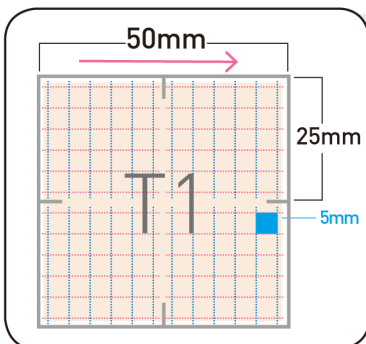



TREATMENT PROTOCOL

3 Outer Thigh

Cartridge	Power(J)	Spacing(mm)	Length(mm)	Treatment Zone(shot)
L4	1.5	1.5	25	10

Treatment Tip

- Depending on the area of outer thigh can vary the number of the treatment zone.
- It allows to treat in the area that had more than 2cm thick adipose tissue
- Treatment should be performed horizontally first for one entire section and then performed vertically.
- 10shot ~ 15shot, horizontal approximate interval is 5mm
- 10shot ~ 15shot, vertical approximate interval is 5mm

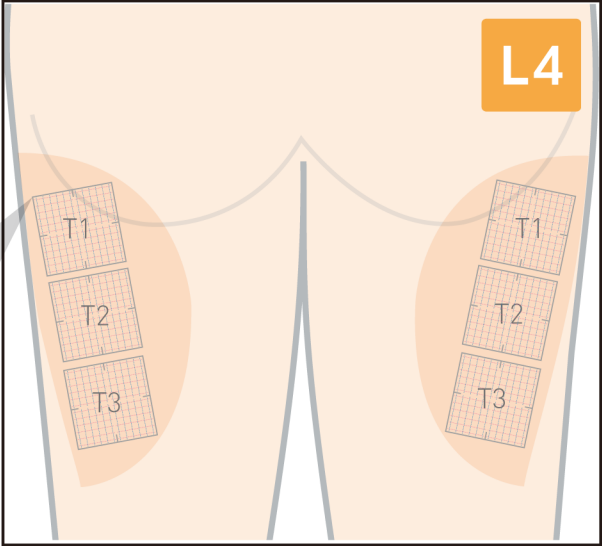
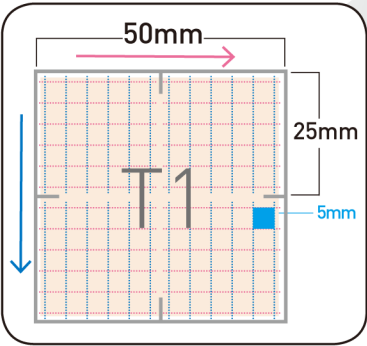


4 Inner Thigh

Cartridge	Power(J)	Spacing(mm)	Length(mm)	Treatment Zone(shot)
L4	1.5	1.5	25	10

Treatment Tip

- Depending on the area of inner thigh can vary the number of the treatment zone.
- It allows to treat in the area that had more than 2cm thick adipose tissue
- Treatment should be performed horizontally first for one entire section and then performed vertically.
- 10shot ~ 15shot, horizontal approximate interval is 5mm
- 10shot ~ 15shot, vertical approximate interval is 5mm

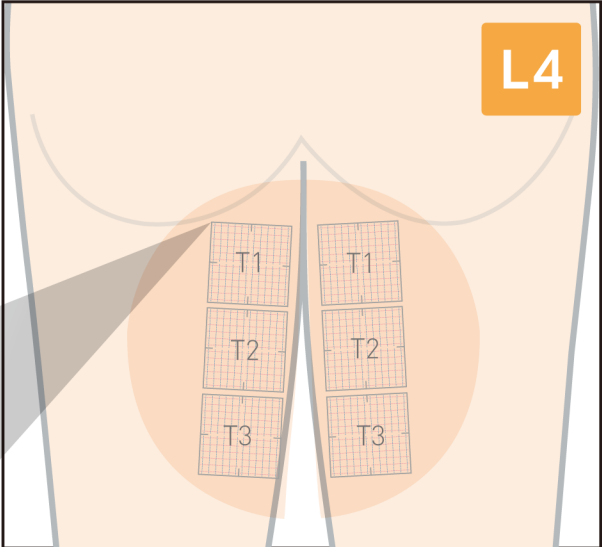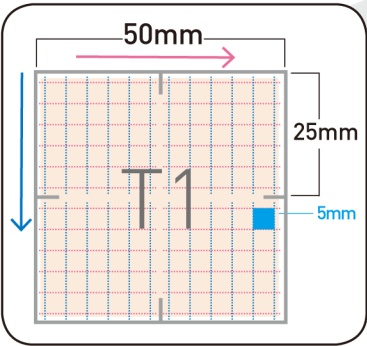

T R E A T M E N T P R O T O C O L B Y A R E A

dōublo · gold

LOCAL
TREATMENT
PROTOCOL


- ※ This device should only be used by physicians trained by Hironic. Read the User Manual before using this product.
- ※ The information provided herein is for reference only. This product is to be used by trained physicians as per patient condition.

Precautions Before Use

- The eye and the inside area of orbital wall must not be treated by this device.
- The area of thyroid must be avoided for the treatment.
- HIFU energy emission on the periosteum could accompany the pain and/or nerve damage because of the thin soft tissue nearby the bone.
- The perioral area would be avoided the treatment as it contains superficial sensory nerve system.


1 Cheek 1

Cartridge	Power[J]	Spacing[mm]	Length[mm]
D4	1.00~1.20	1.5~1.7	25

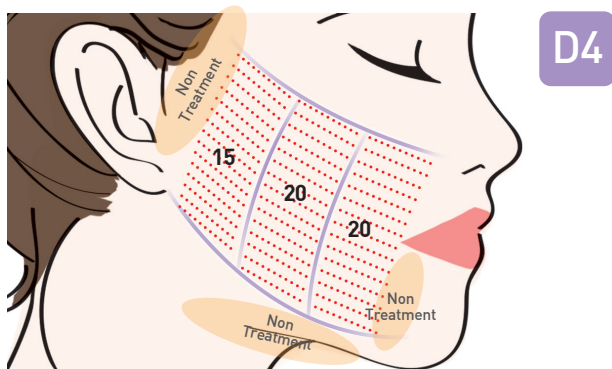


2 Cheek 2

Cartridge	Power[J]	Spacing[mm]	Length[mm]
D4	1.00~1.20	1.5~1.7	25
M7	0.35~0.45	1.0~1.2	25

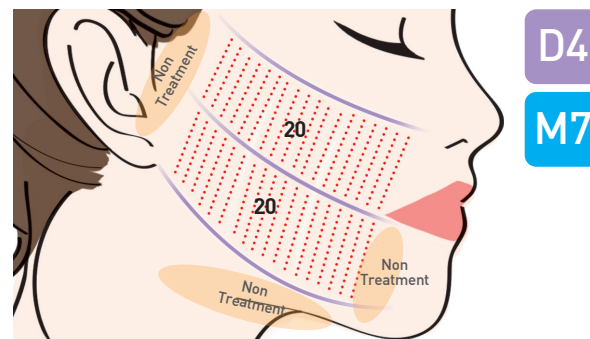


3 Submental

Cartridge	Power(J)	Spacing(mm)	Length(mm)
D4	1.00~1.20	1.5~1.7	25
M7	0.35~0.45	1.0~1.2	25

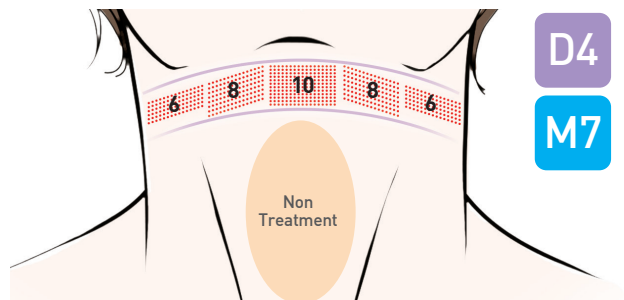


4 Infra Orbital

Cartridge	Power(J)	Spacing(mm)	Length(mm)
M7	0.35~0.45	1.0~1.2	25
S7	0.15~0.25	1.1~1.2	25

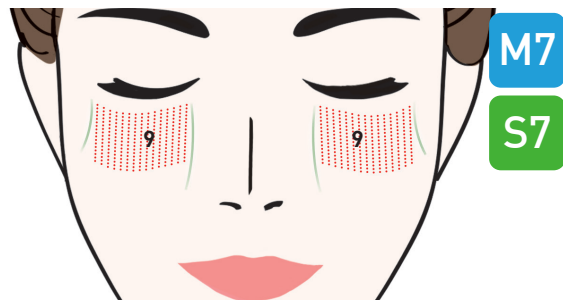


5 Forehead

Cartridge	Power(J)	Spacing(mm)	Length(mm)
M7	0.35~0.45	1.0~1.2	25
S7	0.15~0.25	1.1~1.2	25

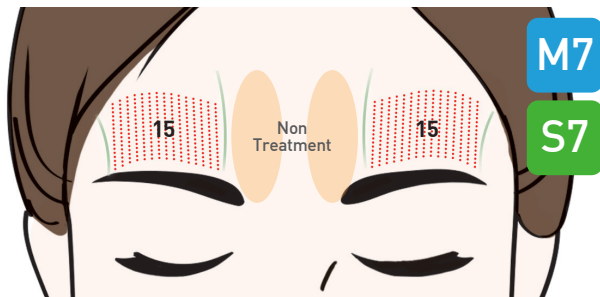


6 Lateral Orbit

Cartridge	Power(J)	Spacing(mm)	Length(mm)
M7	0.35~0.45	1.0~1.2	25
S7	0.15~0.25	1.1~1.2	25

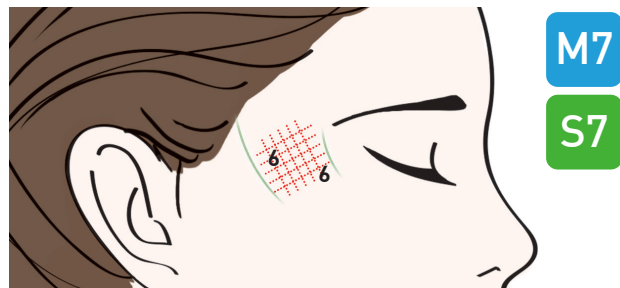


Precautions

- Press down gently to achieve complete contact of cartridge and skin.
- Accumulated heat sense may be relatively stronger due to the speed improvement.
- Avoid emitting energy directly on the bone.
- Be careful of blood vessel and nerve positions.

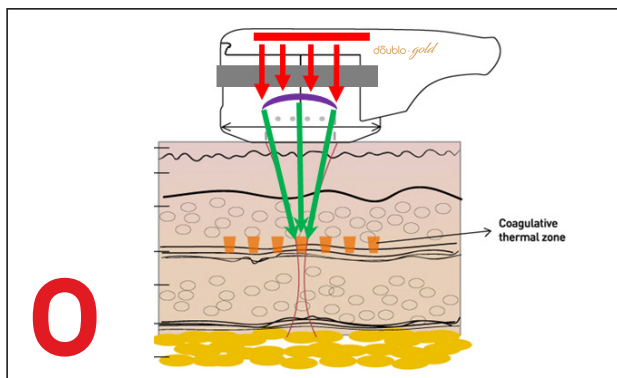

| Correct |

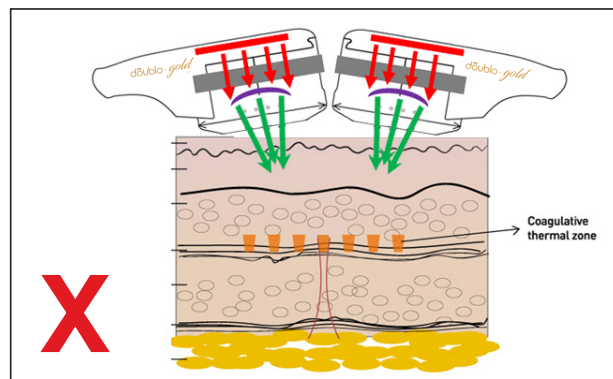

| Incorrect |

※ Failure to hold the cartridge **vertical** to the target skin area can cause epidermal damage accompanied by one of both of the following.

- 1) Red spots 2) Blisters

Precautions

- The patient should avoid saunas and drinking alcohol for 1 week after treatment.
- Any redness resulting from treatment will disappear within an hour. Instead of a cold pack, use a mineral pack or a nourishing (regenerative) pack to maximize treatment effect.
- Makeup can be worn immediately after treatment.
- Wear a sunscreen (SPF 50 or higher) when going outside after treatment.
- After treatment, use a piece of cotton wool dabbed in alcohol to remove the ultrasonic gel from the plastic coating of the cartridge without damage, and remove any remaining moisture with a dry towel or cotton wool.

dōublo · gold

