


Bench aid

for AST reading and interpretation

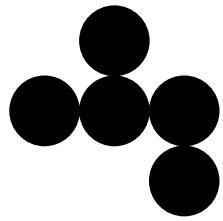

Antibiogo



LA FONDATION

V8 - 16/09/2025 for 2.4

Step 0

Setting up the photo booth

- 1 Background:** a rectangular piece of matte black felt, A4 size (21x30 cm)
 - 2 Stands:** two objects to support the upper cardboard. About 21 cm such as a bottle, or a becher.
 - 3 Upper cardboard:** a rigid sheet of matte black cardboard, A3 size (42x60 cm) or larger.
- ⚠ **Place the Petri dish in the middle of the Background sheet, face up and cover removed.**

