

# iStatis SYPHILIS

Antibody Test (CA)




## INTRODUCING THE iStatis Syphilis Antibody Test


- ✓ Accurate and easy-to-read results
- ✓ Results in just 15 minutes
- ✓ Simple-to-use, easy-to-store, and portable

# TEST PROCEDURE


## TEST SET UP

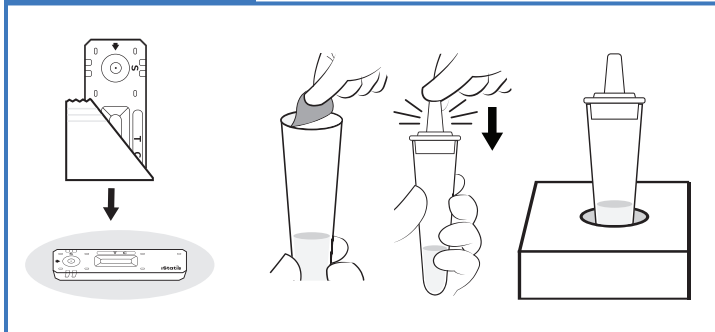


## BLOOD SAMPLE COLLECTION

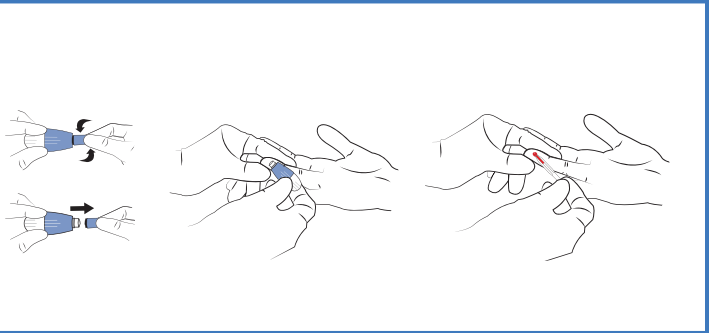

## TESTING OF SAMPLE

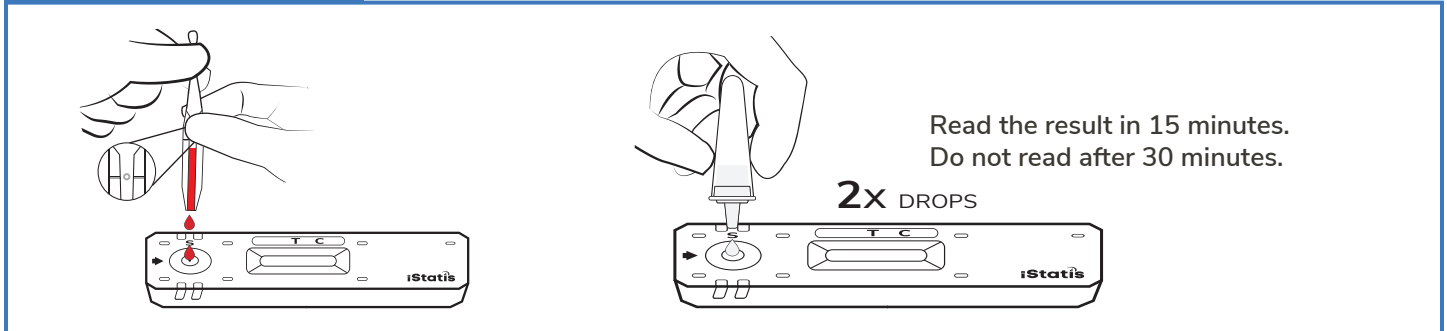

## READ TEST RESULTS

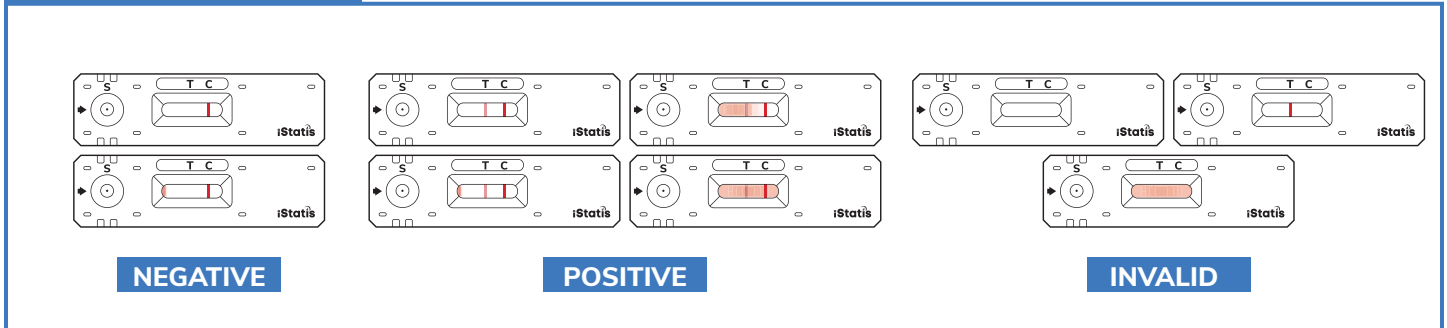

## PRODUCT INFORMATION

INFORMATION TYPE	PRODUCT DETAILS
Method	Lateral Flow
Time To Results	15 Minutes
Storage Conditions	2–30 °C (35.6–86 °F)
Shelf Life	14 Months <sup>1</sup>
Sample Type	Whole Blood, Fingerstick Whole Blood, Serum or Plasma

Notes: <sup>1</sup>Shelf-life to be extended.

## ACCURACY

SAMPLE TYPE	PPA	NPA
Fingerstick Capillary Blood	96.4% [94.4%-97.9%]	100.0% [99.6% - 100.0%]
Venous Whole Blood	98.8% [97.4%-99.6%]	100.0% [99.6% - 100.0%]
Serum	100.0% [98.5%-100.0%]	100.0% [99.3%-100.0%]
Plasma	99.0% [97.7%-99.7%]	100.0% [99.6% - 100.0%]

## ORDERING INFORMATION

PRODUCTS	
901144140011-50	iStatis Syphilis Antibody, Singles, CA, 50 (Individual tests)
901144140012-100	iStatis Syphilis Antibody, Singles, CA, 250 (Individual tests)
901150140015-50	iStatis Syphilis Antibody, Bulk, CA, EN, 50
901150140016-100	iStatis Syphilis Antibody, Bulk, CA, EN, 100

- ✓ Built-In Control
- ✓ All Support Materials Included
- ✓ Easy-To-Use and Clear Results
- ✓ High Accuracy and Reliability