

StomaDiscTM

StomaDiscTM Size: XX Fr

LOT

XXXXX



XX/XXXX

Product Description: The reusable StomaDiscTM is designed to help retain the StomaEaseTM powder underneath the external retention device of the catheter.

Indications and Intended Use: To be used with percutaneous enteral catheters. The device is placed over an enteral stoma, followed by application of StomaEaseTM powder to the surrounding skin. This encloses the powder in the StomaDiscTM, increasing the powder's efficacy to collect moisture along the stoma as well as streamlining the cleaning process. For use with gastrostomy (G-Tubes), jejunostomy (J-Tubes) and other percutaneous enteral catheters.

Contraindications: Not for use with colostomy, ileostomy, or urostomy stomas, or if blood is present.

Precautions: For external use only. Discontinue use if redness or irritation is observed. Contact a healthcare practitioner if experiencing skin irritation or infection. In the event of a serious incident during product use, please report it to the local distributor or manufacturer.



LumaClean LLC

INNOVATIVE SOLUTION SPECIALISTSTM

9724 Colts Neck LN

Concord, NC 28027 USA

LumaClean
CaresTM

www.LumaCleanCares.com

e-mail: info@stomaease.com




www.StomaEase.com




Instructions for Use

- 1 - Wash and dry hands well and/or wear medical grade gloves.
- 2 - Wash and dry skin around the stoma area.
- 3 - **IF** using the **StomaDisc™** fluid and StomaEase™ Powder retainer device: Insure that your catheter has sufficient space between its external retainer and your abdomen to accomodate the 6mm (1/4") thick StomaDisc™. Caution should be used with young children possibly removing the StomaDisc™ and placing it in their mouths.

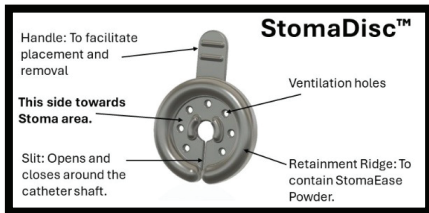

- 3.i - Place the StomaDisc™ with the smooth flat side facing up and raised ridge side facing towards the skin onto the shaft of the catheter and below the catheters' external retention device.

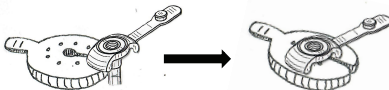

3.ii – After installing the StomaDisc™, gently lift-up the edges to create access to the stoma site.

3.iii - Apply a thin layer of StomaEase™ powder around the enteral feed tube and stoma site. Then release the edges of the StomaDisc™ to cover the StomaEase powder.

3.iv - Periodically check the stoma area for any signs of a gel plug or leakage. If you notice the powder has formed a gel plug or if leakage resumes, it is time to remove the gel plug, clean the area, and apply fresh StomaEase™ Powder.

3.v - To remove the gel plug, first remove the StomaDisc™. If the gel plug comes up with the disc, dispose of the gel plug and save the StomaDisc™ for future use (wash with soap and water before reusing). If the gel plug remains on the skin, use gauze to gently remove it and dispose of the used gauze. Rinse and dry the stoma site before reapplying as necessary.

3.vi - Keep the cap tightly closed to prevent moisture from entering the bottle.

StomaDisc™: Is made from medical grade material and is latex free.

For additional information please visit: www.stomaease.com

Proudly made in the USA.

StomaEase™ is patent pending and a trademark of LumaClean, LLC
StomaDisc™ is patent pending and a trademark of LumaClean, LLC.