



2850 Redhill Ave, Suite 210, Santa Ana, CA 92705 USA
Phone: 781-587-3242 • www.openimplants.com

Instructions For Use

Sherlock

Caution:







Federal law restricts this device to sale by or on the order of a dentist.

Small diameter implants and angled abutments are not recommended for the posterior region of the mouth.

Disclaimer of Liability:

The users of Open Implants, LLC. products must determine whether or not a particular product is suitable for a particular application and circumstance. Open Implants, LLC. disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages arising out of or in conjunction with any errors in professional judgment or practice in the use of Open Implants, LLC. products. Users are advised and obliged to study the latest news and developments in implant dentistry, and to frequently review www.openimplants.com for any updates to products and/or specifications. Open Implants, LLC. has no control over the use of its products, which are the responsibility of the user. Open Implants, LLC. assumes no liability whatsoever for damage arising thereof.

Prosthetic Components

					
Do Not Re-use	Consult Instructions For Use	Manufacturer	Non-Sterile	Federal law restricts this device to sale by or on the order of a licensed dentist	MR Conditional

MM_021_IFU_2.0

Users are urged to consult website for additional guidance on instructions for use

Indications for Use:

These implants, prosthetic and laboratory components are to be used for the reconstruction of partial and total edentulous patients. They are not intended for any other part of the body than the oral cavity. The implants are to be used as single or multiple units in the edentulous regions of the maxilla and mandible. Anterior and posterior locations can be utilized following the given surgical protocol. Prosthetic and laboratory components are to be utilized with this system following the standard protocol for the reconstruction of the dental prosthesis in the given indication. Indication range from single tooth application, implant supported partial denture, implant supported bar-overdenture, implant retained overdenture, fixed-detachable implant prosthesis, and totally edentulous implant retained prosthetic design. The diameter changes of the implant are to correspond with the available bone width in the treatment areas. If a larger diameter implant can be utilized, it is recommended to be used to engage the greatest amount of bone volume.