Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- The product must not be stored out
- The product must not be warmed more than 50°C (112 °F)
- No weight must be loaded when the bed table is in a 45° tilting position
- Do not scratch the paint.

2. Installation

Adjust the height of the bed table with the roller placed on the central tube. Once the appropriate height is found, screw the roller. Before use, check the stability of the table, and lock the castors.

You can place a book on the plate up to a 45° angle. The plastic surrounding will prevent books from falling down. The plastic surrounding is especially designed to prevent dust and water from coming into the wooden plate.

3. Cleaning

The plate and the legs must be cleaned frequently with non-abrasive cleaners.

4. Materials

The product is made up of a metal frame, a laminated plate, and 4 castors (with 2 breaks). It also has a PVC trim.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Weight: 9 kg
Maximal weight supported: 10 kg, in a vertical position
Dimensions of the plate: 40*80 cm

*This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.
This product is biocompatible.*

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