drive

DeVilbiss®
HEALTHCARE

Rollator

Road

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INDICATION

Walking device for indoor and outdoor use for compensation and improvement respectively of restricted ability of walking.

PACKAGE CONTENTS

1. Walking device
2. Basket
3. Tray
4. Cane holder (in 2 parts)
5. Operating instructions

INSTALLATION

Height adjustment handgrips


Insert the tube of the adjustable handgrip into the frame at least to the "Maximum" marking.

Fix it in the desired height with the fixing lever by turning it clockwise (see picture 1).

Adjust both handgrips on the same height/level. Handgrips must be adjusted parallel to the walking direction!

Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting.

For readjustment of the height of the handgrips unlock the fixing lever by turning it counter-clockwise. Then fix the handgrip how described above.

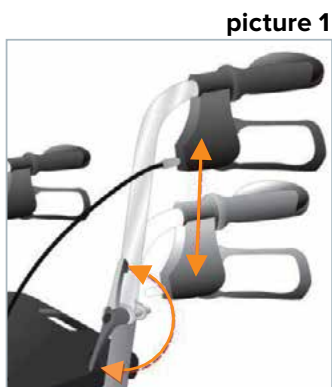picture 1

Unfold


Position the walking device on its wheels. Push the front frame forward by pressing on the handgrips down and pulling backwards until the walking device stands on four wheels. Then move the seat down until it touches the crossbar. (see picture 2)

picture 2

Fitting with basket and tray

Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture 3).

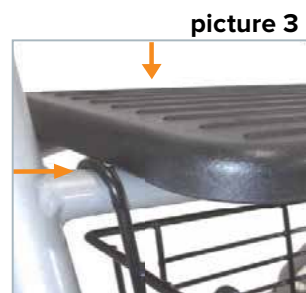picture 3

BRAKING / PARKING

For braking the walking device pull both brakes upwards to the handgrip (see picture 4). Always brake slowly and steady.

For parking the walking device press down the brakes until they click in the parking position (see picture 5).

For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

The effect of the brakes can be affected by abrasion of the wheels.

picture 4

picture 5

ADJUSTMENT OF BRAKES

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 7) or with the suitable tool (not included) on the brake wire (picture 6). The gap between brake plate and tire should be between 3-4 mm (see picture 8).

picture 6

picture 7

picture 10


picture 11


picture 8

picture 12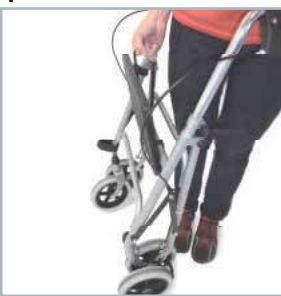

picture 13

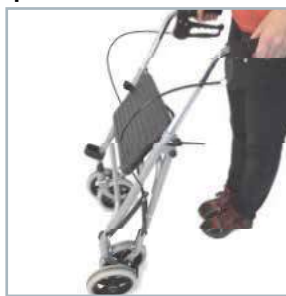SITTING

Position the walking device on an even and solid surface.

Then lock the brakes (parking position). Walking device must stand and must not slip. Sit straight up and do not lean backwards (see picture 9).

Seating position only as shown allowed (against the direction)!

picture 9

FOLDING AND STORAGE

For folding pull seat upwards (see pictures 10, 11 and 12). Walking device can now be transported by trailing or is now suitable for storage (picture 13).

SAFETY INSTRUCTIONS

Use only as a walking device.

Use only on even and solid surfaces.

Walking device must not be used for standing on or as a wheelchair.

It must not be used for transport of persons or animals.

Maximum load Walking device: 120 kg.

Use tray only for small, light, objects. Max. load: 5.0 kg.

Use basket only for transport of objects and do not over- load. Max. load: 5.0 kg.

Surface temperature

If the walking device is exposed to a heat source, such as a heater or the sunrays, either directly or through a windowpane, or is in a car boot, the temperature of the seat, and metallic and plastic parts may be very high and burn the user. Those elements should be cooled before using the rollator.

REPROCESSING

The rollator is suitable for reuse.

Ensure that the new user is given all of the technical documents required to ensure safe handling.

The rollator must be cleaned and disinfected prior to reuse and must be inspected for damage and approved by the retailer.

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