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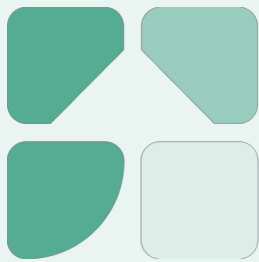
Patient log			
Care plan: ALL		Type: MEDICAL	
Showing 1 out of 2 total log messages.			
Date	Type	Log	User
16/01/2024, 14:43	MEDICAL	This patient has an inflammation	

Patient log			
Care plan: ALL		Type: ALL	
Showing 2 out of 424 total log messages.			
Date	Type	Log	
25/09/2023, 02:11	FLAG (red line)	TRIGGER: The patient provided an improvement suggestion in medical scan: Satisfaction @ 5 weeks	
16/04/2023, 14:26	FLAG (red line)	TRIGGER: The patient contacted for issue of app index	

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
Patient's report - 17/05/2022		Personal information																																																													
Surgery information Surgery type: Colon resection Reason: Colon cancer Preval Therapy: Colon Check Hospital: A-Zentrum Treatment: Multidisciplinary		Patient number: 17001 Age: 71 Gender: Female Surgery date: 20/10/2022																																																													
Patient-reported outcomes <table border="1"> <thead> <tr> <th>Score</th> <th>Pre op</th> <th>4-6w</th> <th>3-6m</th> <th>6-12m</th> <th>1Y</th> <th>2Y</th> </tr> </thead> <tbody> <tr> <td>Stoma-related QOL</td> <td>10</td> <td>10</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Function QOL</td> <td>10</td> <td>10</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Quality of Life QOL</td> <td>10</td> <td>10</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Health-related QOL</td> <td>10</td> <td>10</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Psychosocial QOL</td> <td>10</td> <td>10</td> <td></td> <td></td> <td></td> <td></td> </tr> </tbody> </table>		Score	Pre op	4-6w	3-6m	6-12m	1Y	2Y	Stoma-related QOL	10	10					Function QOL	10	10					Quality of Life QOL	10	10					Health-related QOL	10	10					Psychosocial QOL	10	10					Clinical milestones <table border="1"> <thead> <tr> <th>Time point</th> <th>Event</th> <th>Completed</th> </tr> </thead> <tbody> <tr> <td>20/10/2022</td> <td>Stoma-related</td> <td>20</td> </tr> <tr> <td>20/10/2022</td> <td>Function-related</td> <td>20</td> </tr> <tr> <td>20/10/2022</td> <td>Quality of Life-related</td> <td>20</td> </tr> <tr> <td>20/10/2022</td> <td>Health-related</td> <td>20</td> </tr> <tr> <td>20/10/2022</td> <td>Psychosocial-related</td> <td>20</td> </tr> </tbody> </table>		Time point	Event	Completed	20/10/2022	Stoma-related	20	20/10/2022	Function-related	20	20/10/2022	Quality of Life-related	20	20/10/2022	Health-related	20	20/10/2022	Psychosocial-related	20
Score	Pre op	4-6w	3-6m	6-12m	1Y	2Y																																																									
Stoma-related QOL	10	10																																																													
Function QOL	10	10																																																													
Quality of Life QOL	10	10																																																													
Health-related QOL	10	10																																																													
Psychosocial QOL	10	10																																																													
Time point	Event	Completed																																																													
20/10/2022	Stoma-related	20																																																													
20/10/2022	Function-related	20																																																													
20/10/2022	Quality of Life-related	20																																																													
20/10/2022	Health-related	20																																																													
20/10/2022	Psychosocial-related	20																																																													

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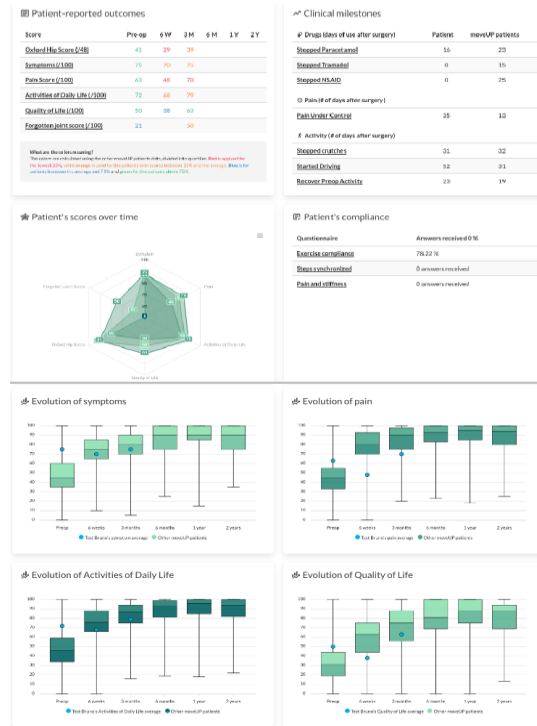Suggestions

- Two crutches outside
- Availability hometrainer
- Two crutches
- Swelling tips
- Stiffness tips
- Night pain tips

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
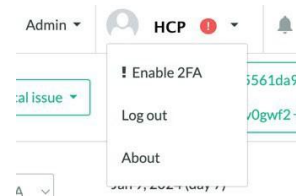
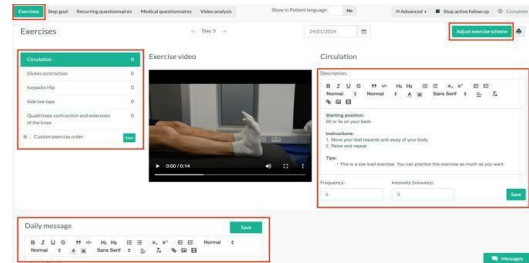
Clear view	Active exercises	Day	Keypack4 Hip	Circulation	Heel raises	Glutes contraction
		-1				
		0		6x 20 minutes	6x 3 minutes	3x 10 repetitions
		1		6x 20 minutes	6x 3 minutes	3x 10 repetitions

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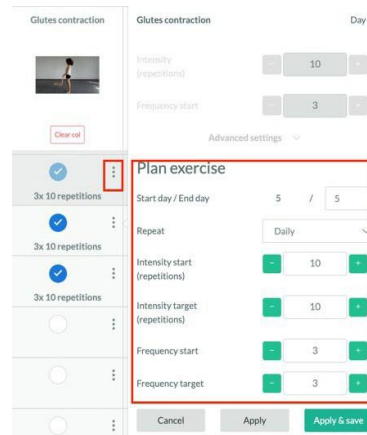The screenshot shows the 'Recurring questionnaires' section of the software. At the top, there are tabs for 'Exercises', 'Step goal', 'Recurring questionnaires', 'Medical questionnaires', and 'Video analysis'. Below the tabs, there is a 'Questionnaires' section with a 'Personalize' button. A table lists existing questionnaires with columns for Name, Start day, End day, Frequency (times), Active from, Reminders, and Order. Below the table is a modal titled 'Add questionnaire to personalized care plan'. This modal contains a dropdown menu for 'Questionnaire', input fields for 'Start date' and 'End date', a 'Frequency' dropdown set to 'Daily', and input fields for 'Activation hour' (set to 16) and 'Activation minutes' (set to 0). 'Save' and 'Cancel' buttons are at the bottom.

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The screenshot shows the 'Medical questionnaires' section. It has tabs for 'Exercises', 'Step goal', 'Recurring questionnaires', 'Medical questionnaires', and 'Video analysis'. Below the tabs is a 'Medical scores scheme' section with an 'Add' button. A table lists medical scores with columns for Name, Start day, and End day. Below the table is a modal titled 'Add medical questionnaire to care plan'. This modal contains a dropdown menu for 'Medical score', a dropdown for 'Activation type' set to 'Day in treatment', and input fields for 'Start Date' and 'End Date'. 'Save' and 'Cancel' buttons are at the bottom.

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
The screenshot shows the 'Video analysis' section. It has tabs for 'Exercises', 'Step goal', 'Recurring questionnaires', 'Medical questionnaires', and 'Video analysis'. Below the tabs is a 'Video analysis' section with a checkbox for 'Analysis enabled for this patient' and an 'Add request' button. Below this is a 'Video requests' section with a table listing requests with columns for Name, Start day, and End day. Below the table is a modal titled 'Add video request to personalized care plan'. This modal contains a dropdown menu for 'Video request', input fields for 'Start date' and 'End date', and 'Save' and 'Cancel' buttons. Below the modal is another modal titled 'Request the patient to record a video for analysis' with a dropdown menu for 'Video type' and 'Save' and 'Cancel' buttons.

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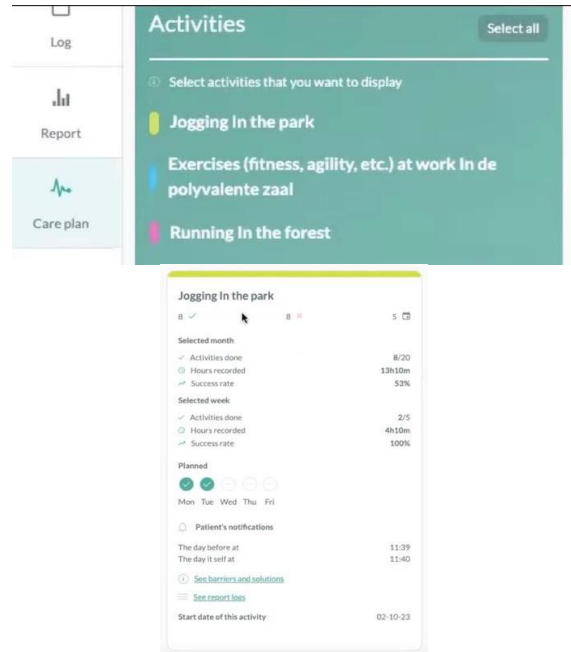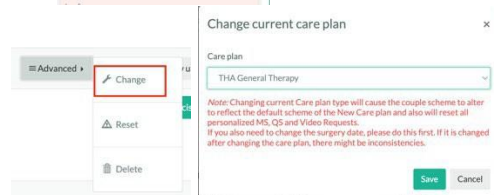
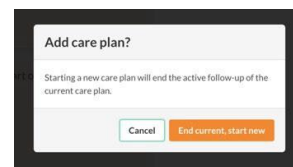
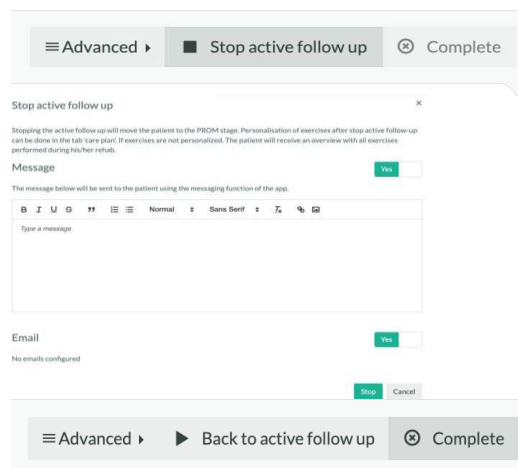
Stop treatment?

Are you sure you want to stop the treatment? Stopping a treatment will mark the patient as being finished. Medical scores in the treatment will no longer be activated.

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The screenshot shows a user profile editing page for 'Laurens Baes'. The 'Wellness module' is currently disabled (checkbox is unchecked). The 'Enable wellness data' checkbox is checked. The 'Save' button is highlighted in green.

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The screenshot shows a 'Platform notice' dialog box titled 'Wellness module terms of use'. The text explains that the wellness module provides additional data about general wellbeing and activity levels, but this data is not for clinical decision-making. It also states that the data is optional and not linked to medical data. At the bottom, there is an 'I agree' checkbox and 'Sign' and 'Cancel' buttons.

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Day	Reported at	Q1: Pain(1) Indicate how much pain you had in your affected joint during the night (0=none, 1=light, 2=moderate, 3=severe)	Q2: Pain(2) Indicate how much pain you had in your affected joint during the day (0=none, 1=light, 2=moderate, 3=severe)	Q3: Pain(3) Did you experience any pain other than in the affected joint?	Q4: Pain(4) Indicate where and when you experienced pain.
242	Jan 23, 2023, 9:26:45 AM	0.52	0.81	Yes	Night, Back, Exercise - Call, No pain - Green

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
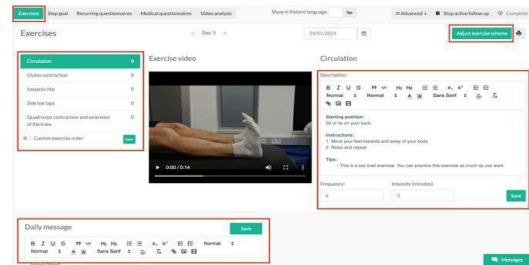
Clear row	Active exercises	Day	Icepacks Hip	Circulation	Heel raises	Glutes contraction
		-1				
		0	6x 20 minutes	6x 3 minutes		3x 15 repetitions
		1	6x 20 minutes	6x 3 minutes		3x 10 repetitions

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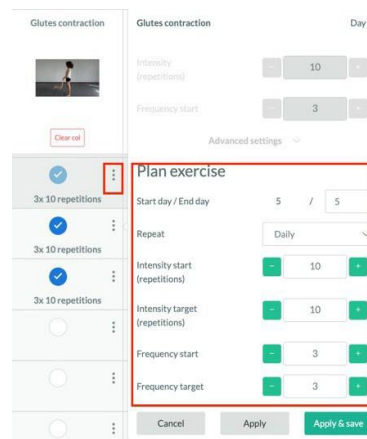Name	Start day	End day	Frequency (times)	Active from	Reminders	Order
Discharge	3	30	1	14:00	No reminder set last	1
Postoperative status	0	3	1	14:00	No reminder set last	2
Activities	15	100	1	14:00	No reminder set last	3
Exercises	1	100	1	14:00	No reminder set last	4

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Care plan: ALL | Type: MEDICAL

Showing 1 out of 2 total log messages.

Date	Type	Log	User
16/01/2024, 14:43	MEDICAL	This patient has an inflammation	

Patient log

Care plan: ALL | Type: MEDICAL

Showing 2 out of 404 total log messages.

Date	Type	Log
25/09/2023, 02:11	FLAG (red line)	TRIGGER: The patient provided an improvement suggestion in medical score: Satisfaction @ 1 week.
25/09/2023, 14:26	FLAG (red line)	TRIGGER: The patient contacted for issue of app index.

Patient's report - 17/03/2022

Surgery information

Surgery type: Abdominal surgery

Region: Abdominal surgery

Procedure: Abdominal surgery

Procedure date: 17/03/2022

Procedure status: Completed

Personal information

Patient number: 12345

Age: 65

Gender: Male

Surgeon name: Dr. J. Doe

Patient-reported outcomes

Score	Pre op	4 Wk	12 Wk	1 Yr	2 Yr
Overall patient-reported outcomes	100	100	100	100	100
Pain (0-100)	100	100	100	100	100
Quality of life (0-100)	100	100	100	100	100
Death (0-100)	100	100	100	100	100
Postoperative complications	100	100	100	100	100

Clinical milestones

Of these milestones	Patients	Completed
Of these milestones	10	10
Of these milestones	10	10
Of these milestones	10	10
Of these milestones	10	10

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
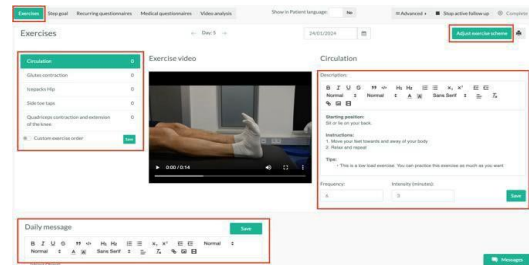
Clear rate	Active exercises	Day	Icepacks Hip	Circulation	Heel raises	Glutes contraction
		-1				
		0	6x 20 minutes	6x 3 minutes		3x 15 repetitions
		1	6x 20 minutes	6x 3 minutes		3x 10 repetitions

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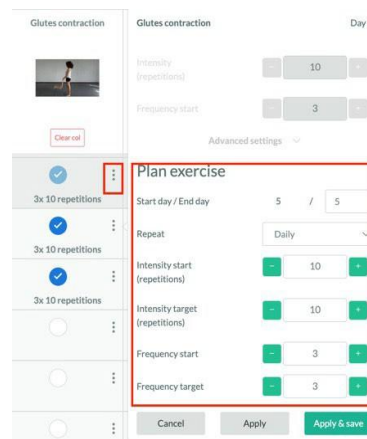Name	Start day	End day	Frequency (times)	Active from	Reminders	Order
Discharge	3	30	5	16:00	No reminder set last	1
Postoperative status	0	3	5	16:00	No reminder set last	2
Activities	15	100	5	16:00	No reminder set last	3
Exercises	1	100	5	16:00	No reminder set last	4

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Activities Select all

Select activities that you want to display

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Advanced **Stop active follow up** **Complete**

Stop active follow up

Stopping the active follow up will move the patient to the PROMM stage. Personalisation of exercises after stop active follow up can be done in the tab 'care plan'. If exercises are not personalized, the patient will receive an overview with all exercises performed during his/her rehab.

Message Yes

The message below will be sent to the patient using the messaging function of the app.

Type a message

Email Yes

No emails configured

Advanced **Back to active follow up** **Complete**

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Platform notice

Wellness module terms of use

Dear User,
You are about to activate the Wellness Module.

What is the Wellness Module?

The wellness module provides you with additional data about the general wellbeing and activity levels of your patients. The data in the Wellness Module can come from:

- imported data from general-purpose devices such as e.g. activity trackers
- manual entry by patients

This data does not come from medical devices. This means that neither the devices from which the data is imported, nor the manual entries by the patients have been verified or validated by moveUP, and therefore, it should not be used for clinical decision-making.

What is the purpose of this data?

We provide the option to see wellness data to give you a broader idea about the general wellbeing of the patients and to give you the opportunity to discuss and give further guidance regarding wellbeing.

This module is an optional module and is not linked to the medical data presented in the "Medical data" tab of the dashboard.

The data in the wellness module is not intended to be used for clinical decisions.

How to recognize the wellness data?

Wellness data is displayed in a separate tab in the dashboard or in a separate report.

Whenever wellness data is displayed in a column, together with other data it has a grey background to visually indicate that this does not concern medical data. Furthermore, it has a warning sign next to it.

Enabling/Disabling the Wellness Module

At any time, a user can enable/disable the Wellness Module in the user settings.

Acknowledgement

By clicking on the button below, you acknowledge that you have read and understood that the Wellness Module is an optional module that provides you with additional data regarding the general wellbeing of the patient. This data is not validated by moveUP and is not intended for clinical decision making.

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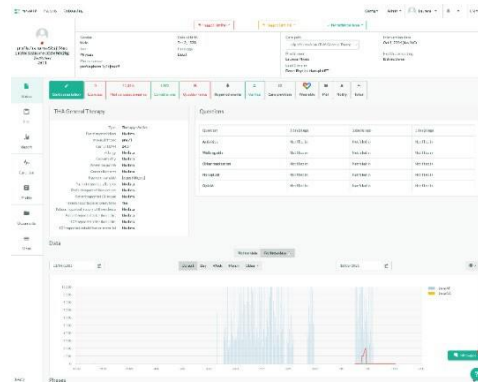
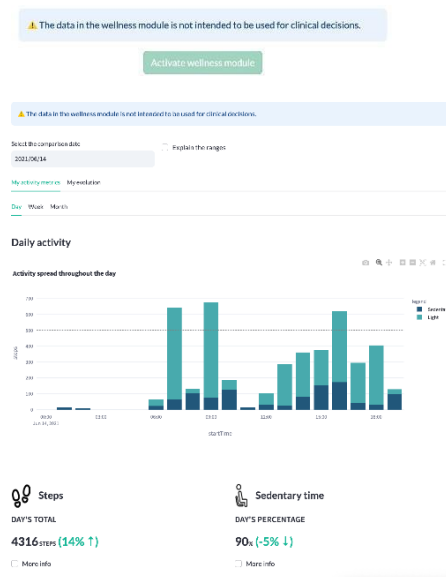
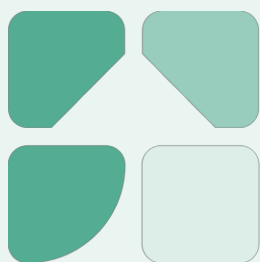
User manual for the Health Care Provider(HCP)

moveUP

A personalized treatment, with more insight and involvement
in the care pathway.

App version : 2.3.0 and later

Tradename: moveUP

Manual version : 2.0

Data of release manual : 06/03/2026

Table of Contents

Table of Contents	1
1. Product description	2
1. Intended Use	2
2. Intended Users	2
3. Target population / indication for use	3
4. Claims	3
5. moveUP app & devices	3
6. Warnings and precautions	4
7. Contra-indications and potential side effects	4
8. Notification of serious incidents	4
9. Human body contact	5
10. Accessories/products used in combination	5
11. Device lifetime	5
12. Company details	5
2. Patient app	6
3. Medical module	6
1. Healthcare platform account creation	6
2. Onboarding of patients	7
3. Healthcare platform general functions	8
3.1. Patient overview	8
3.2. Patient details	10
3.3. Log out	14
3.4. Support	14
3.5. Care plan management	14
4. Healthcare platform care plan functions	15
4.1. Exercise management	15
4.2. Recurring questionnaire	15
4.3. Medical questionnaire	16
4.4. Video analysis	16
4.5. Goal manager	16
4.6. Change of care plan	17
4.7. Stop active follow up	17
4. WELLNESS MODULE	18
1. WELLNESS MODULE ACTIVATION	18
2. WELLNESS MODULE OVERVIEW	19

1. Product description

1. Intended Use

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With the use of moveUP, valuable insights are given to the HCP/care team about the status of the patient condition and its progression. The HCP/care team can provide targeted therapy advice & adapt treatment (exercise) schemes based on the needs of the patient through the use of the moveUP app.

For patients, moveUP gives insights about their progress, provides instructions related to their disease or treatment and helps them manage their symptoms. Based on the patients' needs and timepoint in the patient pathway, patients can be followed up intensively and receive active therapy advice through the moveUP app.

moveUP is intended for the clinical follow-up on adult patients who underwent (postoperative) or will undergo (preoperative) a Total Knee Arthroplasty (TKA), Total Hip Arthroplasty (THA), Chronic obstructive pulmonary disease (COPD) or Bariatric surgery.

2. Intended Users

moveUP is intended to be used by patients and healthcare providers.

Patient

The main user of the mobile app and patient website is the patient. Patient users should be alert, mentally competent and capable of operating a tablet or smartphone.

Inclusion:

- Age: minimum 18 years / maximum no limit
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

Exclusion:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

Health Care Provider

The main user of the medical web interface is a healthcare practitioner (group) or clinical researcher (group), named the care team. The care team is able to operate a web interface via web browser on PC/tablet/smartphone. The healthcare practitioner needs to understand one of the available languages of the web interface (today only available in English)

Only qualified HCP with a diploma or certificate in the healthcare domain or qualified clinical researchers can access the platform and complete the moveUP e-learning trainings.

3. Target population / indication for use

moveUP is used by patients:

- Who underwent or will undergo a Knee (Resurfacing) arthroplasty
- Who underwent or will undergo a Hip (Resurfacing) arthroplasty
- Who underwent or will undergo a gastric sleeve or gastric bypass surgery
- Patients with COPD.

4. Claims

- Patients using moveUP are highly satisfied.
- Patients using moveUP show a high adherence/compliance level to their therapy.
- The use of moveUP is more cost-effective than standard of care for hip & knee arthroplasty
- The length of stay for Knee & hip arthroplasty can be reduced with the use of moveUP
- Patients can rehabilitate without the need to leave the home environment & without receiving F2F physical therapy session for TKA and THA
- Using moveUP is as effective as standard of care
- Using moveUP is as safe as standard of care

5. moveUP app & devices

The moveUP app runs on:

- iPhones/iPads (iOS 11.0 or newer)
- Android systems (Android 7.0.0 and newer).

If the patient has no compatible tablet/smartphone, moveUP can lease these devices to the patient.

- Lenovo Tab 3 - Android 6.0.1 or Higher
- Huawei MediaPad T3 - Android 7.0.0 or higher

The moveUP application needs an internet connection when in use.

The moveUP application is compatible with the following browsers:

- Chrome
- Safari
- Edge
- Firefox

We strongly recommend to use the latest version of the browser.

For your data safety please enable the 2-factor authentication(2FA) and use a safe network

6. Warnings and precautions

moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.

moveUP is not intended to monitor/treat vital parameters of critical diseases.

Patients need to be aware if questionnaires are not filled out on frequent basis or not filled out in a trustworthy, the care team has more difficulty to know their health situation & evolution.

moveUP is not a replacement of any treatment you need to follow, but is used as an addition.

The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

The platform includes modules displaying wellness data. These data are not part of the medical device and are not intended to be used for clinical decision making

* This data originates from a non-medical device product & may not be validated as such and the data shown in the wellness module cannot be used for medical purposes. The wellness module information is not part of the CE certificate of moveUP.

7. Contra-indications and potential side effects

There are no contra-indications or known side effects

8. Notification of serious incidents

A serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or another person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health that resulted in any of the following:
 - life-threatening illness or injury,
 - permanent impairment of a body structure or a body function, hospitalization
 - prolongation of patient hospitalization, medical or surgical intervention to prevent life-threatening illness or injury
 - permanent impairment to a body structure or a body function,
 - chronic disease,
- a serious public health threat;

If you detect any of these cases please send an email to info@moveUP.care and your competent authority:

- Netherlands: meldpunt@igj.nl
- Belgium: meddev@fagg.be
- France: dedim.ugsv@ansm.sante.fr
- Germany: zlg@zlg.nrw.de

9. Human body contact

No human body contact with the patient or user, due to the nature of the product (software).

10. Accessories/products used in combination

There are no accessories.

11. Device lifetime

2-years, depending on the willingness of the user to update the app. We sent out a notification with the recommendation to update the app, to make sure the app will keep the performance & recently included features.

12. Company details

Tel: +32 800 88 008

UDI: +G166MOVEUP230MDR0P

(01) G166MOVEUP230MDR0P

(10) +\$7230MDR

(11) /16D2026030

(21) P-

moveUP NV/SA

moveUP

Head office Brussel:
Cantersteen 47, B-1000
Brussels, Belgium

Email: info@moveUP.care
www.moveup.care

Secondary office:
Kliniekstraat 27A
9050; Ghent, Belgium

If you have any questions about the App or the tablet, you can contact moveUP via the following channels
- in order of priority: Via the message system of the App. Via email to support@moveup.care - Via phone during office hours: +32 800 88 008

moveUP guarantees delivery of the eIFU within seven (7) days upon request. Please contact info@moveUP.care if you want to obtain a free copy.

2. Patient app

Please read the user manual for patients on www.moveup.care/appinfo to view the functionalities of the patient app.

3. Medical module

1. Healthcare platform account creation

Step 1: The HCP registers on the healthcare platform on <https://v1.medical.moveup.care/>

Login

Username or Email
Username

Password
Password

Login

Request password reset

Login using an electronic identity

No account? [Register here](#)

Step 2: moveUP will contact the HCP to add his/her profile to the correct organization.

Step 3: Security measures: It is important to enable 2FA as you have access to sensitive health data.

NAME !

! Enable 2FA

[Click on course catalog in the top bar to see some example courses and get started. Subscribe for free.](#)

Orthopaedics

Belgium
English (English)
Français (French)

The Netherlands
English (English)

France
Français (French)

Germany
Deutsch (German)

E-learning: For HCPs who will provide care through the use of the healthcare platform it is recommended to follow the e-learning courses for the indications listed on <https://moveup.talentlms.com>. We ask the HCPs to register for the e-learning course in their applicable language.

2. Onboarding of patients

Onboarding is the process of registering a new patient in the healthcare platform.

A crucial part is adding a care plan to the patient's profile. A care plan determines what will be visible for the patient in the application, e.g.: type of exercises, information modules, message functionality, etc. A care plan usually consists out of an intervention, a treatment and (a) care team(s).

The onboarding dashboard is easy retrievable in the health care platform in the upper left corner when clicking 'Onboarding'. It is also accessible through <https://patient-onboarding-admin-moveup.care> using the HCP credentials of the health care platform.

Click on 'new patient' to onboard a new patient and fill out the patient information.

After the patient information is filled out, press 'Save'.

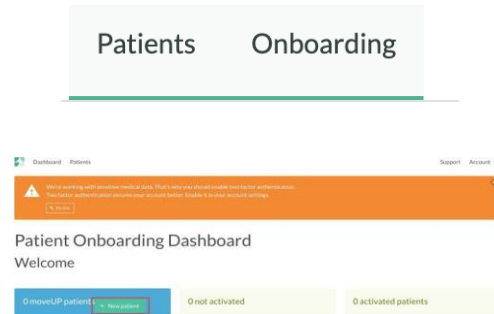
Important note: Only one account can be coupled to an e-mail address. It is not possible to reuse an e-mail address that is already known within our database, e.g.: reusing the same e-mail address for the account of a partner or parent.

Next up, the care plan needs to be selected: press 'Start new care plan'. Drag and drop an intervention and treatment to the care plan and fill out the dates, name of the treating doctor and care team.


Drag and drop a study if applicable. Click Save.

A mail template is shown. The HCP reviews the e-mail variables and clicks on 'send selected mails'.

An invitation mail will be sent to the patient with all the needed information to install and activate the application.

New patient

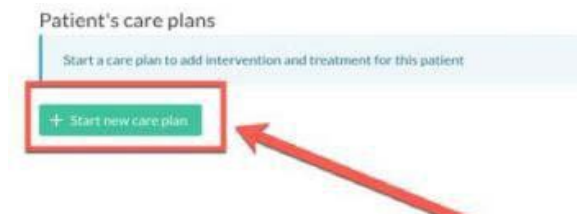3. Healthcare platform general functions

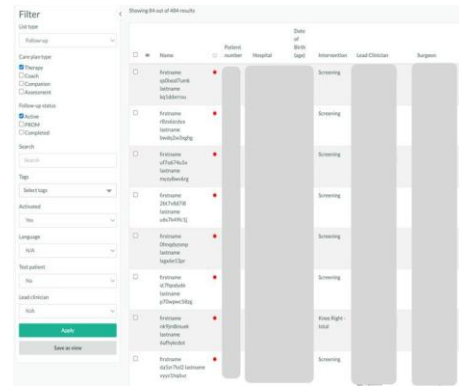
Below the HCP can retrieve all functions of the healthcare platform. The applicable functions will be shown based on the HCPs subscription. The extent of the available patient information is based on the patients subscription & his/her compliancy rate.

3.1. Patient overview

Patient overview


The health care platform gives a clear overview of the HCPs patients.

In the top bar the relevant categories are shown that identify the patient and show relevant health parameters, unread messages, the lead clinician and patient compliance. By default, when opening the healthcare platform, the patient overview is sorted based on intervention date


Tags

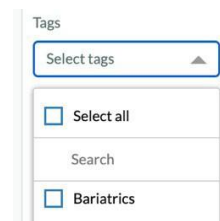Select patients related to a specific pathology by using tags.

Filters

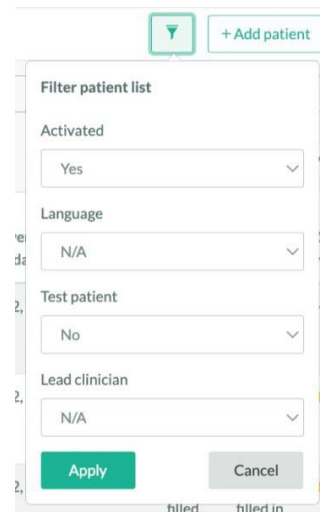Select patients based on language, test patient, lead clinician or being activated or not. Activated can mean 2 things: either the patient registered him/herself with a code through the app or he has been marked as immediately activated in onboarding. Non activated: patients who haven't registered themselves through the app yet and haven't been marked as immediately activated.

Color codes

Color codes indicate which patients need closer attention.


Pain	Feeling ^	Pain v	Feeling
1.3	1	8.7	2
1.6	1.4	8.5	7.7
3	1.9	8.2	4.1

VS


Messaging

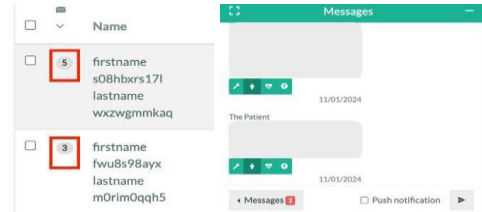The number of unread messages are displayed in the overview


Review / Flags

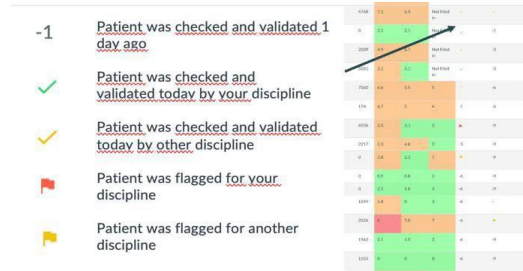With the use of review flags it is known when the patient was checked or who needs to check the patient. See more info in Patient details – Review / Flags.

Notifications

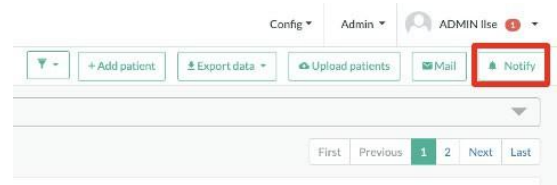
Notifications can be sent directly to the patient's device home screen.

User management

Only limited HCPs of an organization have the permission to add users. In case a HCP has this permission, they can add users by clicking the 'New' button, fill out the credentials, select

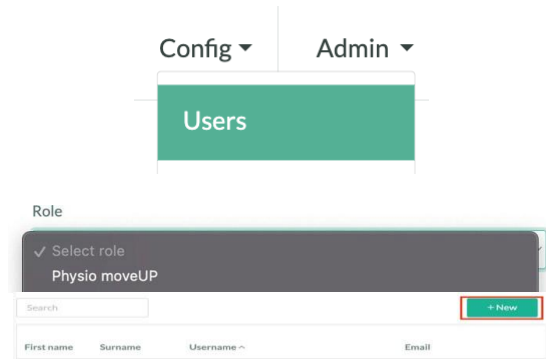the applicable role and press 'Register'. A mail will be sent to the newly created user to follow the account creation steps, see above.


3.2. Patient details

Overview

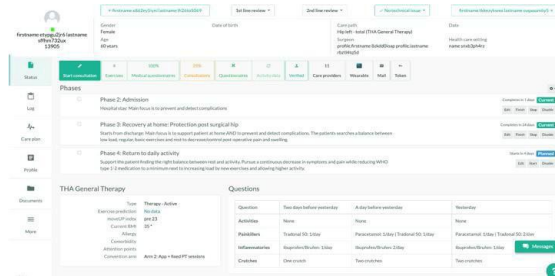
In the patient details overview all relevant (health) information of the individual patient can be retrieved. With the use of the previous and next patient buttons the HCP can easily switch to another patient. In the left side bar, the HCP can navigate between status, log, care plan, profile, documents and more.

Status

The reported healthcare data are retrievable in the 'Status' section. From there it is possible to navigate to the recurring and medical questionnaire results, exercises and many more.

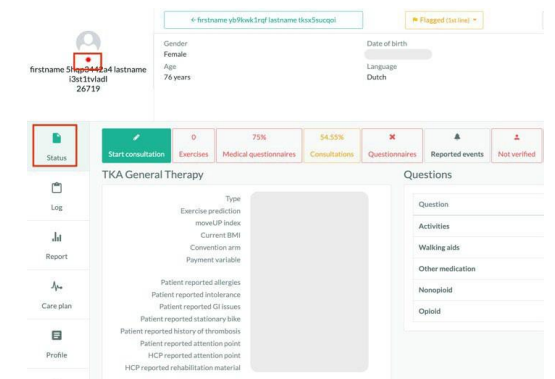A GREEN, YELLOW or RED status dot shows how compliant the patient is to the care plan, for example filling out all the suggested recurring and medical questionnaires and performing the suggested exercises will result in a GREEN status dot.

- Compliance OK for today or yesterday
- Non compliant > 2 days
- Non compliant > 7 days

Medical questionnaires

Via the 'medical questionnaire' button it is shown:

Which medical questionnaires are active for a patient.

How many of these are completed or not.

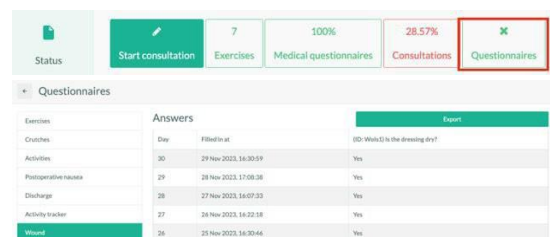
What the reported results are on the individual questions within the medical questionnaire.

What the sub scores and total scores are to be able to see a quick indication on how the patient is progressing.

Recurring questionnaires


With this patient reported answers it is possible to follow the progress of important health parameters frequently

Reported events

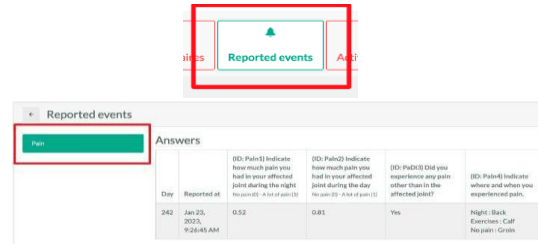The reported events are categorized per event. For each reported event you can view: (1) The date when the event was reported (+day in care plan) (2) The answers to all questions asked to the patient.


Review / Flags

1st line review is for the HCP that is not the lead clinician (e.g. physical therapist), 2nd line review is for the lead clinician (e.g. surgeon)

Click validate when a patient has been monitored. Monitoring can mean:

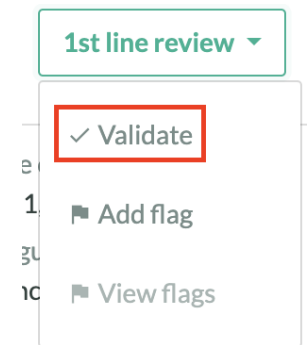- Questions answered – Advice given
- Adjusted or checked exercises
- Adjusted or checked step goal
- Checked daily parameters: pain, swelling, activity....
- Checked postoperative protocol: use of crutches/walker, medication....

Same principle for 2nd line review.

Flag someone from first line

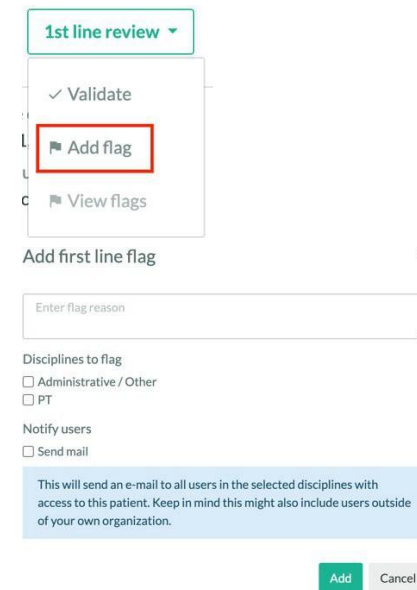
Patient needs to be checked by someone from first line (e.g. a physical therapist)

Give a reason for flag

Select the discipline to flag

Optional: add an e-mail notification to flag. Flagged discipline will receive an email about the flag.


A flag will add a new line in the patient log with the reason for flagging (see further).

Flag someone from second line

- Patient needs to be checked by someone from second line (e.g. a surgeon)
- Give a reason to flag.
- Select the discipline to flag.

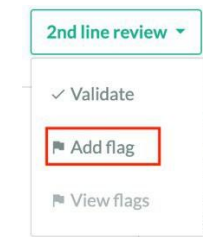Optional: add an e-mail notification to flag. The flagged discipline will receive an email about that flag.

Add second line flag ✕

Enter flag reason

Disciplines to flag

PT 2nd line

Surgeon

Notify users

Send mail

This will send an e-mail to all users in the selected disciplines with access to this patient. Keep in mind this might also include users outside of your own organization.

Add
Cancel

Report a technical issue.

✓ No technical issue
▼

✓ No technical issue

🚩 Technical issue

🔍 Under investigation

Log

Flags and technical updates automatically appear in the log of the patient.

In case of flagging:

- GREEN : solved flag
- RED : Unsolved flag for discipline of logged in user (e.g. PT)
- YELLOW : Unsolved flag for other discipline than logged in user (e.g. Surgeon)

Other important information can be added to the log manually. The applicable tags are added to the log in view of easy log searches

Patient log

Care plan: ALL | Type: MEDICAL

Showing 1 out of 2 total log messages.

Date	Type	Log	User
16-01-2024, 14:43	MEDICAL	This patient has an inflammation	

Patient log

Care plan: ALL | Type: MEDICAL

Showing 2 out of 424 total log messages.

Date	Type	Log
25-09-2023, 02:11	FLAG (red line)	TRIGGER: The patient provided an improvement suggestion in medical score: Satisfaction @ 1 week
16-04-2023, 14:26	FLAG (red line)	TRIGGER: The patient contacted for issue of app index

Report

This report is used to visualize the main milestones and medical questionnaires of the patient, as well as the progression of the patient.

Stream open

Patient's report - 17/03/2022

📄 Surgery information

Region: CH44 - Hout

Region: CH44 - Hout

Medical treatment: CH44 - Hout

Responsible: A.J. van der Vliet

👤 Personal information

Patient number: 1701

Age: 37

Gender: Female

Report date: 17-03-2022

📊 Patient-reported outcomes

Score	Per op	1W	3W	6W	1Y	2Y
Satisfaction (CSQ)	10	10				
Health-related QoL (EQ-5D)	10	10				
Pain (VAS)	10	10				
Health-related QoL (EQ-5D)	10	10				
Health-related QoL (EQ-5D)	10	10				

📈 Clinical milestones


of Group (days of use after surgery)	Return	medical pathway
Return of Consciousness	10	10
Return of Mobility	10	10
Return of Pain	10	10
Return of Health-related QoL	10	10
Return of Health-related QoL	10	10

Page 12 / 19

App version : 2.3.0 and later / Manual version : 2.0 / Data of release manual : 06/03/2026

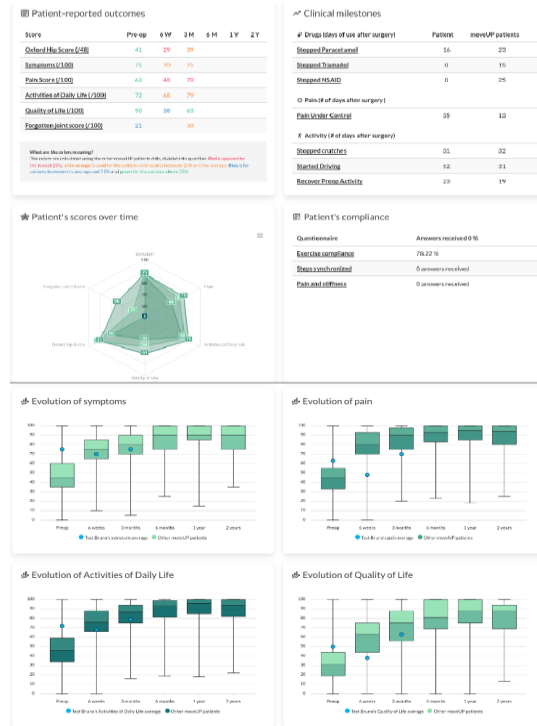The report can be downloaded in pdf form by clicking the 'Download report' button. This way the HCP can send the report to the patient. It's also possible to share the link directly to the report, using the button 'Copy report link'.


Profile


The HCP can view, edit or add all relevant basic, clinical, administrative or care plan specific data in the patient profile

Messaging

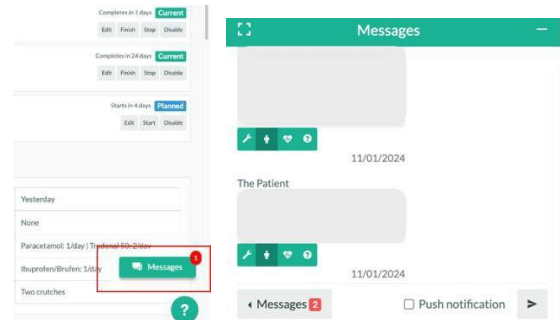The message popup (which can be enlarged) is closed by default and the number of new messages is visible in the floating button. This functionality is used to give advice to patients and answer their questions. It can also be used to ask for pictures, for example to follow-up on wound healing, swelling, redness etc...

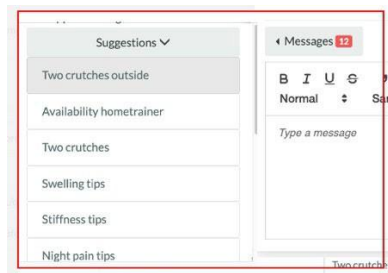When clicking 'messages', a set of prescribed personal messages is shown wherefrom the HCP can select an applicable message for that particular patient.

Click on the "1st line symbol" to mark the question as read.

Messages that are submitted by a HCP can be edited or removed. When hovering over a message, a button to edit or delete the message will become visible.

Documents

Documents can be uploaded in the 'Documents' section. Chat images and video recordings are automatically collected in this section in view of easy access to these materials.

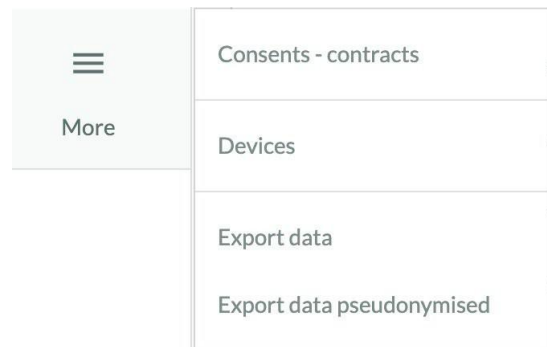
More

See which documents / contracts the patient has signed.


See on which device the patient is using the application and when the application was first and last seen.

Export data is only possible for limited HCPs due to security reasons and to protect data integrity.

3.3. Log out

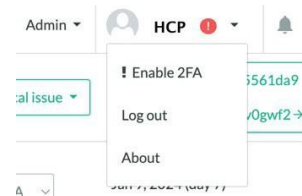In the upper right corner the HCP can log out. Know that the automatic log out time is 14 days, therefore it is recommended to log out more frequently to avoid unauthorized access.

3.4. Support

Reach out via support@moveup.care for questions or use the '?' button on the platform.

3.5. Care plan management

In care plan management it is possible to navigate between multiple functions that are applicable to the indication of the patient, such as exercises, step goal, recurring questionnaires, video analysis, automation, etc... For more information see further.

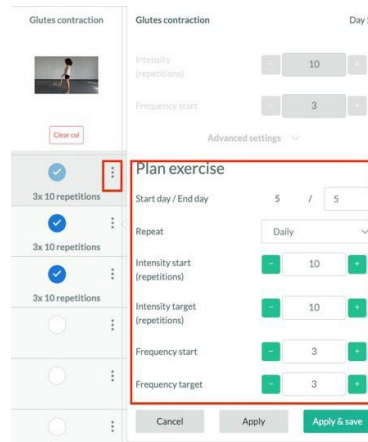
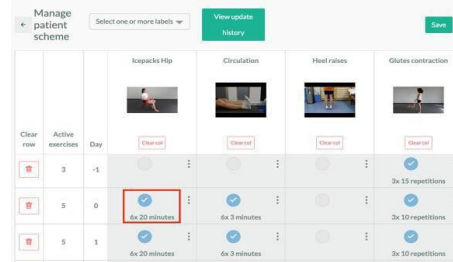
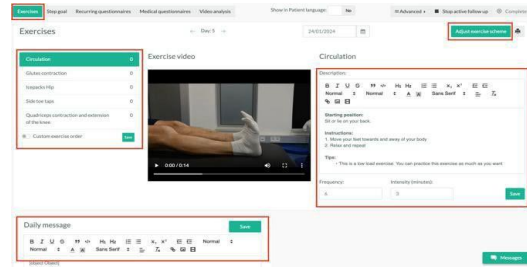
4. Healthcare platform care plan functions

4.1. Exercise management

The exercise management module includes the following possibilities of adaptation:


- Adapt description, frequency, and intensity of the exercises of the day.
- Adapt the daily message.
- Adapt the exercise scheme:
 - Blue check marks indicate active exercises, accompanied with frequency and intensity of these active exercises.
 - Schedule exercises over a longer period of time.

Extend the standard exercise scheme with a set of exercises that are not part of the exercises defined in the care plan

4.2. Recurring questionnaire

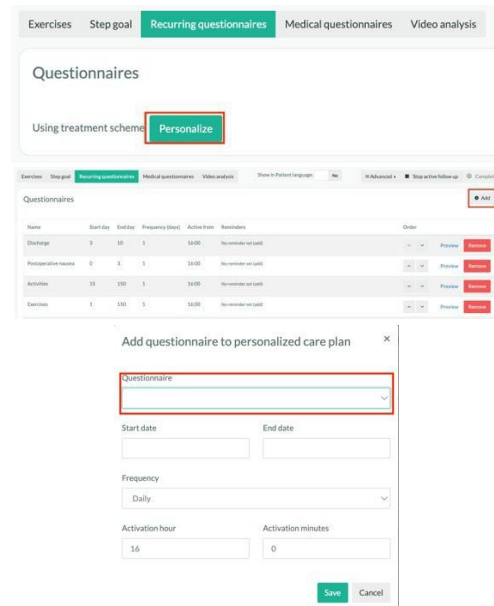Recurring questionnaires can be removed, reordered or a new recurring questionnaire can be added. The default list of recurring questionnaires, including their start day, end day, frequency and reminders is predetermined in the care plan.

4.3. Medical questionnaire

Medical questionnaires can also be removed or added to the default scheme. The default list of medical questionnaires, including their start and end day is predetermined in the care plan.

Name	Start day	End day	Preview	Remove
HIP PROM collection month 6	170	250	Preview	Remove
HIP PROM collection Year 1	340	500	Preview	Remove
Discharge Questionnaire HIP	0	15	Preview	Remove

4.4. Video analysis

With the use of video requests, it is possible to gain more objective information, for example about the gait analysis or range of motion of an operated joint. The default list of video requests is predetermined in the care plan. A video request can be added directly by using the 'Add request' button or a video request can be scheduled for a longer period of time by using the 'Add' button and by adding a start and end day to put the video request active.

Name	Start day	End day	Remove	Message
Gait analysis	-7	-1	Remove	
Gait analysis	21	30	Remove	Message
Gait analysis	-40	45	Remove	Message

4.5. Goal manager

The HCP can enable and disable a behavioral change tool which enables patients to set their personal goal, define barriers and solutions in order to reach a healthy lifestyle. The HCP can see the progress on the patient's goals that are defined by the patient.

Activities [Select all]

- Jogging In the park
- Exercises (fitness, agility, etc.) at work In de polyvalente zaal
- Running In the forest

Jogging In the park

Selected month: 8/20
Activities done: 8/20
Hours recorded: 13h10m
Success rate: 53%

Selected week: 2/5
Activities done: 2/5
Hours recorded: 4h10m
Success rate: 100%

Planned: Mon, Tue, Wed, Thu, Fri

Patient's notifications: The day before at 11:39, The day it self at 11:40

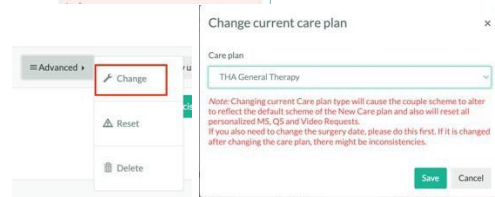
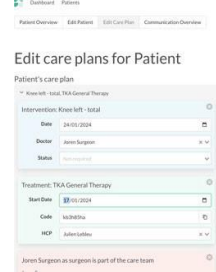
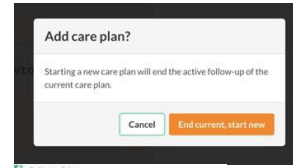
See barriers and solutions, See report logs

Start date of this activity: 02-10-23



4.6. Change of care plan

The care plan can be adapted based on the patient's interventions, needs and wishes. This can be done via the onboarding tool or via the healthcare platform via the button 'Advanced'. Instructions how to change a care plan are clearly indicated within the tools.


4.7. Stop active follow up

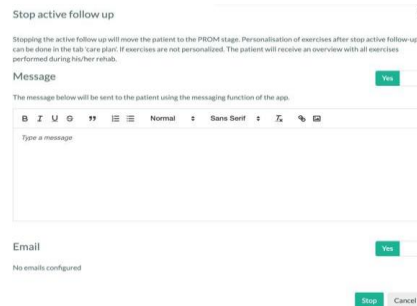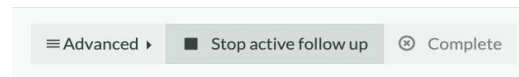When the active follow-up is finished (what falls under active follow-up is predetermined in the care plan), press 'Stop active follow-up'. This will redirect the patient to a passive long term follow-up if applicable (also predetermined in the care plan if long term follow-up is needed).

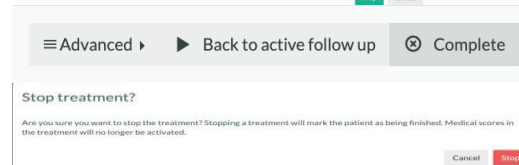When no actions are required anymore from the HCP or the patient, the patient can be redirected to the 'Completed' status.

4. WELLNESS MODULE

The wellness module is an optional non-medical device module and therefore the data shown is not intended to be used for clinical decision making.

This data gives an overview of the general wellbeing of the patient.

The wellness module by default is blurred and needs to be activated to check the patient's wellness data.

1. WELLNESS MODULE ACTIVATION

To activate the wellness module you need to enter your profile settings and activate the option:

Enable wellness data

Upon activation, a mandatory acknowledgment pop-up is displayed. It informs you that the data from the Wellness Module must not be used to support clinical decision-making.

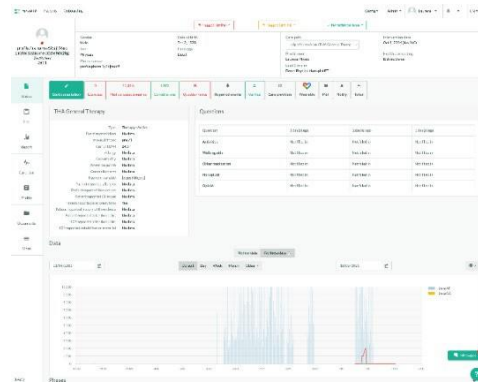
Once acknowledged, you gain access to the wellness data. This activation allows you to view the wellness data and export the wellness reports.

2. WELLNESS MODULE OVERVIEW

Wellness module overview

In this tab you'll be able to see the step data from the patient and see their evolution.

Wellness report

You can see the wellness reports when needed.