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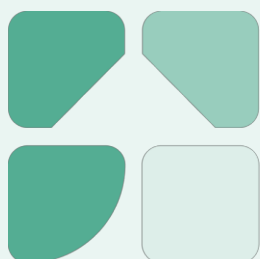
The image shows a series of screenshots from the MoveUP Patient Onboarding interface. At the top, there are tabs for 'Patients' and 'Onboarding'. Below this is the 'Patient Onboarding Dashboard' with a 'Welcome' message and statistics for 'Onboarded patients', 'Not activated', and 'Activated patients'. A 'New patient' button is highlighted. The next screenshot shows the 'New patient' form with fields for 'Managing Organization', 'Test Patient', 'Given names', 'Family name', 'Gender', 'Date of Birth', 'Language', 'Address Lines', 'Postal Code', 'City', 'Country', 'Email', and 'Phone'. A red box highlights the '+ Start new care plan' button. The final screenshot shows the 'Patient's care plans' section with a red box around the '+ Start new care plan' button and a red arrow pointing to it. Below this is a list of 'Email variables' including {{ firstname }}, {{ lastname }}, {{ code }}, {{ registrationlink }}, {{ organizationName }}, {{ surgeonName }}, {{ hosp_spec_par_start_HL }}, {{ organizationName }}, and {{ bodyStructure }}. A 'Send selected' button is highlighted, and a 'Mails were sent' confirmation message is shown in a red box.

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Day	Reported at	(ID-Pain1) Indicate how much pain you had in your affected joint during the night (0=not at all, 10=worst)	(ID-Pain2) Indicate how much pain you had in your affected joint during the day (0=not at all, 10=worst)	(ID-Pain3) Did you experience any pain other than in the affected joint?	(ID-Pain4) Indicate where and when you experienced pain.
242	Jan 25, 2023, 9:28:45 AM	0.52	0.81	Yes	Night: Back Exercises - Call No pain - Green

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The screenshot shows the 'Recurring questionnaires' tab in the software. At the top, there's a 'Questionnaires' section with a 'Personalize' button. Below it is a table of questionnaires:

Name	Start day	End day	Frequency (days)	Active from	Reminders	Order	Preview	Remove
Discharge	3	30	1	14:00	No reminder set (add)	-	Preview	Remove
Postoperative progress	0	3	1	14:00	No reminder set (add)	-	Preview	Remove
Activities	15	180	1	14:00	No reminder set (add)	-	Preview	Remove
Exercises	1	180	1	14:00	No reminder set (add)	-	Preview	Remove

Below the table is a modal 'Add questionnaire to personalized care plan'. It contains a dropdown for 'Questionnaire', fields for 'Start date' and 'End date', a 'Frequency' dropdown (set to 'Daily'), and 'Activation hour' (16) and 'Activation minutes' (0) fields. 'Save' and 'Cancel' buttons are at the bottom.

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The screenshot shows the 'Medical questionnaires' tab. At the top, there's a 'Medical scores scheme' section with an 'Add' button. Below it is a table of medical scores:

Name	Start day	End day	Preview	Remove
HIP PROM collection month 6	170	210	Preview	Remove
HIP PROM collection Year 1	340	500	Preview	Remove
Discharge Questionnaire HSP	0	15	Preview	Remove

Below the table is a modal 'Add medical questionnaire to care plan'. It contains a dropdown for 'Medical score', an 'Activation type' dropdown (set to 'Day in treatment'), and 'Start Date' and 'End Date' fields. 'Save' and 'Cancel' buttons are at the bottom.

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The screenshot shows the 'Video analysis' tab. At the top, there's a 'Video analysis' section with checkboxes for 'Analysis enabled for this patient' and 'Results NOT shown to this patient'. Below it is a table of video requests:

Name	Start day	End day	Remove	Messages
Gait analysis	-7	-1	Remove	Messages
Gait analysis	15	30	Remove	Messages
Gait analysis	-40	45	Remove	Messages

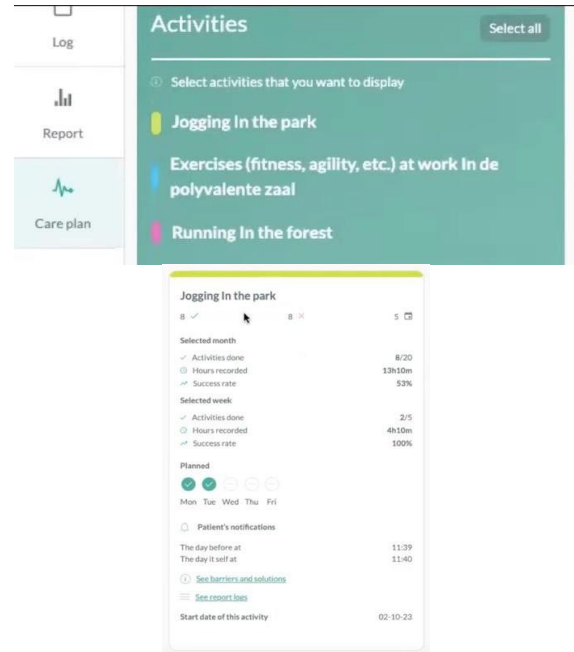
Below the table is a modal 'Add video request to personalized care plan'. It contains a dropdown for 'Video request', 'Start date' and 'End date' fields, and 'Save' and 'Cancel' buttons.

Below the modal is another modal 'Request the patient to record a video for analysis'. It contains a dropdown for 'Video type' and 'Save' and 'Cancel' buttons.

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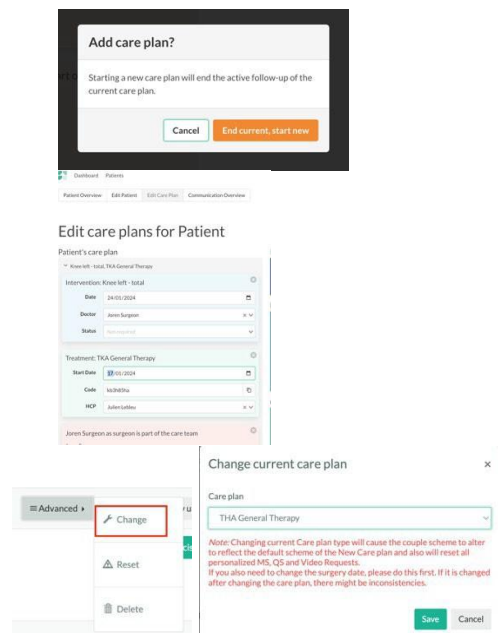
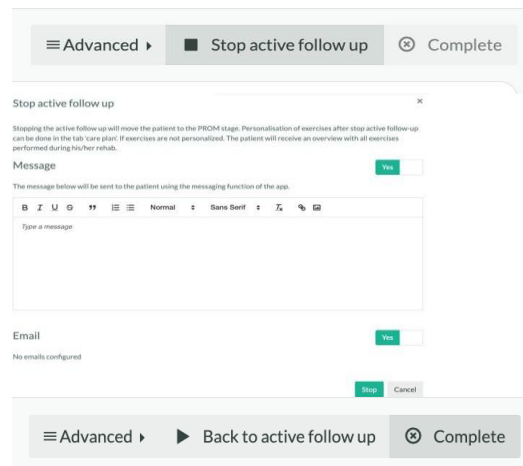
Stop treatment?

Are you sure you want to stop the treatment? Stopping a treatment will mark the patient as being finished. Medical scores in the treatment will no longer be activated.

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The screenshot shows the 'Edit Laurens Baes' profile page. The 'Wellness module' checkbox is checked, and the 'Save' button is highlighted in green. The page includes fields for personal information, contact details, and a 'Wellness module' section with a checkbox and a 'Save' button.

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
The screenshot shows a 'Platform notice' dialog box titled 'Wellness module terms of use'. It contains detailed information about the module's purpose, data sources, and usage restrictions. The text includes: 'Dear User, You are about to activate the Wellness Module.', 'What is the Wellness Module?', 'The wellness module provides you with additional data about the general wellbeing and activity levels of your patients.', 'The data in the Wellness Module can come from:', 'Imported data from general-purpose devices such as e.g. activity trackers', 'Manual entry by patients', 'This data does not come from medical devices. This means that neither the devices from which the data is imported, nor the manual entries by the patients have been verified or validated by moveUP, and therefore, it should not be used for clinical decision-making.', 'What is the purpose of this data?', 'We provide the option to see wellness data to give you a broader idea about the general wellbeing of the patients and to give you the opportunity to discuss and give further guidance regarding wellbeing.', 'This module is an optional module and is not linked to the medical data presented in the "Medical data" tab of the dashboard.', 'The data in the wellness module is not intended to be used for clinical decisions.', 'How to recognize the wellness data?', 'Wellness data is displayed in a separate tab in the dashboard or in a separate report.', 'Whenever wellness data is displayed in a column, together with other data it has a grey background to visually indicate that this does not concern medical data. Furthermore, it has a warning sign next to it.', 'Enabling/Disabling the Wellness Module', 'At any time, a user can enable/disable the Wellness Module in the user settings.', 'Acknowledgement', 'By clicking on the button below, you acknowledge that you have read and understood that the Wellness Module is an optional module that provides you with additional data regarding the general wellbeing of the patient. This data is not validated by moveUP and is not intended for clinical decision making.', and a 'Sign' button.

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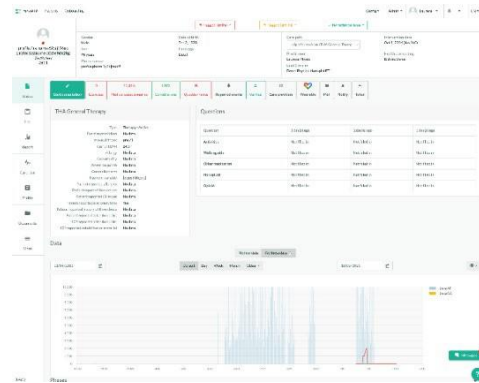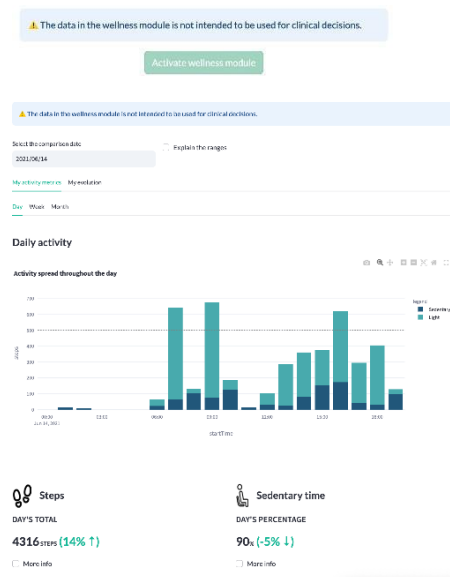
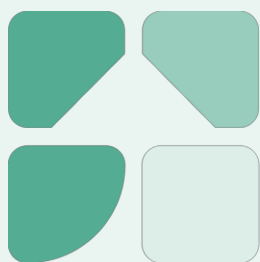
User manual for the Health Care Provider(HCP)

moveUP

A personalized treatment, with more insight and involvement
in the care pathway.

App version : 2.4.0 and later

Tradename: moveUP

Manual version : 2.1

Data of release manual : 28/04/2026

Table of Contents

Table of Contents	1
1. Product description	2
1. Intended Use	2
2. Intended Users	2
3. Target population / indication for use	3
4. Claims	3
5. moveUP app & devices	3
6. Warnings and precautions	4
7. Contra-indications and potential side effects	4
8. Notification of serious incidents	4
9. Human body contact	5
10. Accessories/products used in combination	5
11. Device lifetime	5
12. Company details	5
2. Patient app	6
3. Medical module	6
1. Healthcare platform account creation	6
2. Onboarding of patients	7
3. Healthcare platform general functions	8
3.1. Patient overview	8
3.2. Patient details	10
3.3. Log out	14
3.4. Support	14
3.5. Care plan management	14
4. Healthcare platform care plan functions	15
4.1. Exercise management	15
4.2. Recurring questionnaire	15
4.3. Medical questionnaire	16
4.4. Video analysis	16
4.5. Goal manager	16
4.6. Change of care plan	17
4.7. Stop active follow up	17
4. WELLNESS MODULE	18
1. WELLNESS MODULE ACTIVATION	18
2. WELLNESS MODULE OVERVIEW	19

1. Product description

1. Intended Use

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With the use of moveUP, valuable insights are given to the HCP/care team about the status of the patient condition and its progression. The HCP/care team can provide targeted therapy advice & adapt treatment (exercise) schemes based on the needs of the patient through the use of the moveUP app.

For patients, moveUP gives insights about their progress, provides instructions related to their disease or treatment and helps them manage their symptoms. Based on the patients' needs and timepoint in the patient pathway, patients can be followed up intensively and receive active therapy advice through the moveUP app.

moveUP is intended for the clinical follow-up on adult patients who underwent (postoperative) or will undergo (preoperative) a Total Knee Arthroplasty (TKA), Total Hip Arthroplasty (THA), Chronic obstructive pulmonary disease (COPD) or Bariatric surgery.

2. Intended Users

moveUP is intended to be used by patients and healthcare providers.

Patient

The main user of the mobile app and patient website is the patient. Patient users should be alert, mentally competent and capable of operating a tablet or smartphone.

Inclusion:

- Age: minimum 18 years / maximum no limit
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

Exclusion:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

Health Care Provider

The main user of the medical web interface is a healthcare practitioner (group) or clinical researcher (group), named the care team. The care team is able to operate a web interface via web browser on PC/tablet/smartphone. The healthcare practitioner needs to understand one of the available languages of the web interface (today only available in English)

Only qualified HCP with a diploma or certificate in the healthcare domain or qualified clinical researchers can access the platform and complete the moveUP e-learning trainings.

3. Target population / indication for use

moveUP is used by patients:

- Who underwent or will undergo a Knee (Resurfacing) arthroplasty
- Who underwent or will undergo a Hip (Resurfacing) arthroplasty
- Who underwent or will undergo a gastric sleeve or gastric bypass surgery
- Patients with COPD.

4. Claims

- Patients using moveUP are highly satisfied.
- Patients using moveUP show a high adherence/compliance level to their therapy.
- The use of moveUP is more cost-effective than standard of care for hip & knee arthroplasty
- The length of stay for Knee & hip arthroplasty can be reduced with the use of moveUP
- Patients can rehabilitate without the need to leave the home environment & without receiving F2F physical therapy session for TKA and THA
- Using moveUP is as effective as standard of care
- Using moveUP is as safe as standard of care

5. moveUP app & devices

The moveUP app runs on:

- iPhones/iPads (iOS 11.0 or newer)
- Android systems (Android 7.0.0 and newer).

If the patient has no compatible tablet/smartphone, moveUP can lease these devices to the patient.

- Lenovo Tab 3 - Android 6.0.1 or Higher
- Huawei MediaPad T3 - Android 7.0.0 or higher

The moveUP application needs an internet connection when in use.

The moveUP application is compatible with the following browsers:

- Chrome
- Safari
- Edge
- Firefox

We strongly recommend to use the latest version of the browser.

For your data safety please enable the 2-factor authentication(2FA) and use a safe network

6. Warnings and precautions

moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.

moveUP is not intended to monitor/treat vital parameters of critical diseases.

Patients need to be aware if questionnaires are not filled out on frequent basis or not filled out in a trustworthy, the care team has more difficulty to know their health situation & evolution.

moveUP is not a replacement of any treatment you need to follow, but is used as an addition.

The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

The platform includes modules displaying wellness data. These data are not part of the medical device and are not intended to be used for clinical decision making

* This data originates from a non-medical device product & may not be validated as such and the data shown in the wellness module cannot be used for medical purposes. The wellness module information is not part of the CE certificate of moveUP.

7. Contra-indications and potential side effects

There are no contra-indications or known side effects

8. Notification of serious incidents

A serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or another person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health that resulted in any of the following:
 - life-threatening illness or injury,
 - permanent impairment of a body structure or a body function, hospitalization
 - prolongation of patient hospitalization, medical or surgical intervention to prevent life-threatening illness or injury
 - permanent impairment to a body structure or a body function,
 - chronic disease,
- a serious public health threat;

If you detect any of these cases please send an email to info@moveUP.care and your competent authority:

- Netherlands: meldpunt@igj.nl
- Belgium: meddev@fagg.be
- France: dedim.ugsv@ansm.sante.fr
- Germany: zlg@zlg.nrw.de

9. Human body contact

No human body contact with the patient or user, due to the nature of the product (software).

10. Accessories/products used in combination

There are no accessories.

11. Device lifetime

2-years, depending on the willingness of the user to update the app. We sent out a notification with the recommendation to update the app, to make sure the app will keep the performance & recently included features.

12. Company details

Tel: +32 800 88 008

UDI: +G166240MDR09

+G166240MDR0/\$\$7240MDR/16D20260428S

moveUP NV/SA

moveUP

Head office Brussel:
Cantersteen 47, B-1000
Brussels, Belgium

Email: info@moveUP.care
www.moveup.care

Secondary office:
Kliniekstraat 27A
9050; Ghent, Belgium

If you have any questions about the App or the tablet, you can contact moveUP via the following channels
- in order of priority: Via the message system of the App. Via email to support@moveup.care - Via phone
during office hours: +32 800 88 008

moveUP guarantees delivery of the eIFU within seven (7) days upon request. Please contact info@moveUP.care if you want to obtain a free copy.

2. Patient app

Please read the user manual for patients on www.moveup.care/appinfo to view the functionalities of the patient app.

3. Medical module

1. Healthcare platform account creation

Step 1: The HCP registers on the healthcare platform on <https://v1.medical.moveup.care/>

Login

Username or Email
Username

Password
Password

Login

Request password reset

Login using an electronic identity

No account? [Register here](#)

Step 2: moveUP will contact the HCP to add his/her profile to the correct organization.

Step 3: Security measures: It is important to enable 2FA as you have access to sensitive health data.

NAME

! Enable 2FA

[Click on course catalog in the top bar to see some example courses and get started. Subscribe for free.](#)

Orthopaedics

Belgium
English (English)
Français (French)

The Netherlands
English (English)

France
Français (French)

Germany
Deutsch (German)

E-learning: For HCPs who will provide care through the use of the healthcare platform it is recommended to follow the e-learning courses for the indications listed on <https://moveup.talentlms.com>. We ask the HCPs to register for the e- learning course in their applicable language.

2. Onboarding of patients

Onboarding is the process of registering a new patient in the healthcare platform.

A crucial part is adding a care plan to the patient's profile. A care plan determines what will be visible for the patient in the application, e.g.: type of exercises, information modules, message functionality, etc. A care plan usually consists out of an intervention, a treatment and (a) care team(s).

The onboarding dashboard is easy retrievable in the health care platform in the upper left corner when clicking 'Onboarding'. It is also accessible through <https://patient-onboarding-admin-moveup.care> using the HCP credentials of the health care platform.

Click on 'new patient' to onboard a new patient and fill out the patient information.

After the patient information is filled out, press 'Save'.

Important note: Only one account can be coupled to an e-mail address. It is not possible to reuse an e-mail address that is already known within our database, e.g.: reusing the same e-mail address for the account of a partner or parent.

Next up, the care plan needs to be selected: press 'Start new care plan'. Drag and drop an intervention and treatment to the care plan and fill out the dates, name of the treating doctor and care team.

Drag and drop a study if applicable. Click Save.

A mail template is shown. The HCP reviews the e-mail variables and clicks on 'send selected mails'.

An invitation mail will be sent to the patient with all the needed information to install and activate the application.

Patients Onboarding

New patient

3. Healthcare platform general functions

Below the HCP can retrieve all functions of the healthcare platform. The applicable functions will be shown based on the HCPs subscription. The extent of the available patient information is based on the patients subscription & his/her compliancy rate.

3.1. Patient overview

Patient overview

The health care platform gives a clear overview of the HCPs patients.

In the top bar the relevant categories are shown that identify the patient and show relevant health parameters, unread messages, the lead clinician and patient compliance. By default, when opening the healthcare platform, the patient overview is sorted based on intervention date

Filter	Name	Patient number	Hospital	Date of Birth	Intervention	Lead Clinician	Surgeon
☐ All patients	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Active	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Completed	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Incomplete	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Not patient	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Lead clinician	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Not patient	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Lead clinician	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Not patient	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Lead clinician	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Not patient	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones
☐ Lead clinician	Antonieta	123456789	St. Mary's	1980-01-01	Screening	Dr. Smith	Dr. Jones

Tags

Select patients related to a specific pathology by using tags.

Tags

Select tags

Select all

Search

Bariatrics

Filters

Select patients based on language, test patient, lead clinician or being activated or not. Activated can mean 2 things: either the patient registered him/herself with a code through the app or he has been marked as immediately activated in onboarding. Non activated: patients who haven't registered themselves through the app yet and haven't been marked as immediately activated.

Filter patient list

Activated

Yes

Language

N/A

Test patient

No

Lead clinician

N/A

Apply

Cancel

Color codes


Color codes indicate which patients need closer attention.

Pain	Feeling	Pain	Feeling
1.3	1	8.7	2
1.6	1.4	8.5	7.7
3	1.9	8.2	4.1


Messaging

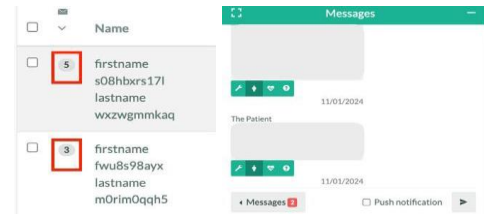The number of unread messages are displayed in the overview


Review / Flags

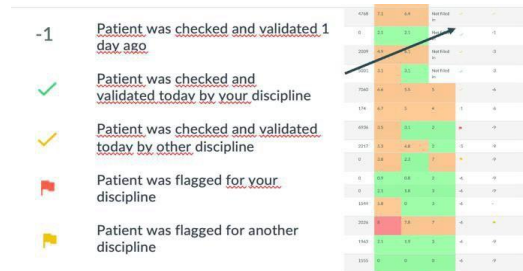With the use of review flags it is known when the patient was checked or who needs to check the patient. See more info in Patient details – Review / Flags.

Notifications

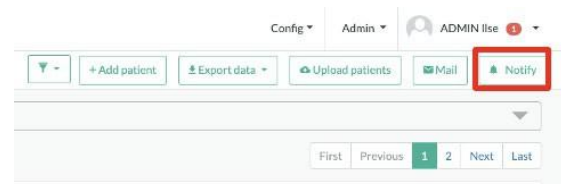
Notifications can be sent directly to the patient's device home screen.

User management

Only limited HCPs of an organization have the permission to add users. In case a HCP has this permission, they can add users by clicking the 'New' button, fill out the credentials, select

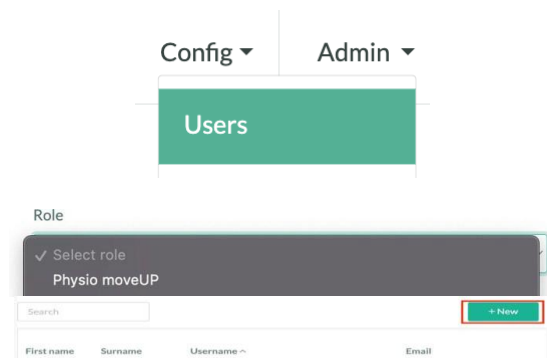the applicable role and press 'Register'. A mail will be sent to the newly created user to follow the account creation steps, see above.

3.2. Patient details

Overview

In the patient details overview all relevant (health) information of the individual patient can be retrieved. With the use of the previous and next patient buttons the HCP can easily switch to another patient. In the left side bar, the HCP can navigate between status, log, care plan, profile, documents and more.

Status

The reported healthcare data are retrievable in the 'Status' section. From there it is possible to navigate to the recurring and medical questionnaire results, exercises and many more.

A GREEN, YELLOW or RED status dot shows how compliant the patient is to the care plan, for example filling out all the suggested recurring and medical questionnaires and performing the suggested exercises will result in a GREEN status dot.

Medical questionnaires

Via the 'medical questionnaire' button it is shown:

Which medical questionnaires are active for a patient.

How many of these are completed or not.

What the reported results are on the individual questions within the medical questionnaire.

What the sub scores and total scores are to be able to see a quick indication on how the patient is progressing.

Recurring questionnaires

With this patient reported answers it is possible to follow the progress of important health parameters frequently

Reported events

The reported events are categorized per event. For each reported event you can view: (1) The date when the event was reported (+day in care plan) (2) The answers to all questions asked to the patient.

Day	Reported at	(ID: Pain1) Indicate how much pain you had in your affected joint during the night (0=not at all, 1=not at all, 2=not at all)	(ID: Pain2) Indicate how much pain you had in your affected joint during the day (0=not at all, 1=not at all, 2=not at all)	(ID: Pain3) Did you experience any pain other than in the affected joint?	(ID: Pain4) Indicate where and when you experienced pain.
242	Jan 25, 2023, 9:28:45 AM	0.52	0.81	Yes	Night: Back Exercise - Call No pain - Green

Review / Flags

1st line review is for the HCP that is not the lead clinician (e.g. physical therapist), 2nd line review is for the lead clinician (e.g. surgeon)

Click validate when a patient has been monitored. Monitoring can mean:

- Questions answered – Advice given
- Adjusted or checked exercises
- Adjusted or checked step goal
- Checked daily parameters: pain, swelling, activity....
- Checked postoperative protocol: use of crutches/walker, medication....

Same principle for 2nd line review.

Flag someone from first line

Patient needs to be checked by someone from first line (e.g. a physical therapist)

Give a reason for flag

Select the discipline to flag

Optional: add an e-mail notification to flag. Flagged discipline will receive an email about the flag.

A flag will add a new line in the patient log with the reason for flagging (see further).

Enter flag reason

Disciplines to flag

☐ Administrative / Other

☐ PT

Notify users

☐ Send mail

This will send an e-mail to all users in the selected disciplines with access to this patient. Keep in mind this might also include users outside of your own organization.

Add Cancel

Flag someone from second line

- Patient needs to be checked by someone from second line (e.g. a surgeon)
- Give a reason to flag.
- Select the discipline to flag.

Optional: add an e-mail notification to flag. The flagged discipline will receive an email about that flag.

Add second line flag ✕

Enter flag reason

Disciplines to flag

☐ PT 2nd line

☐ Surgeon

Notify users

☐ Send mail

This will send an e-mail to all users in the selected disciplines with access to this patient. Keep in mind this might also include users outside of your own organization.

Add **Cancel**

Report a technical issue.

✓ No technical issue

✓ No technical issue

Technical issue

Under investigation

Log

Flags and technical updates automatically appear in the log of the patient.

In case of flagging:

- GREEN : solved flag
- RED : Unsolved flag for discipline of logged in user (e.g. PT)
- YELLOW : Unsolved flag for other discipline than logged in user (e.g. Surgeon)

Other important information can be added to the log manually. The applicable tags are added to the log in view of easy log searches

Patient log

Care plan: ALL Type: MEDICAL

Showing 1 out of 2 total log messages.

Date	Type	Log	User
24/01/2024, 14:43	MEDICAL	This patient has an inflammation	

Patient log

Care plan: ALL Type: MEDICAL

Showing 2 out of 404 total log messages.

Date	Type	Log
25/09/2023, 02:51	FLAG (red line)	TRIGGER: The patient provided an improvement suggestion in medical score: Satisfaction @1 week.
15/04/2023, 14:36	FLAG (red line)	TRIGGER: The patient provided the lowest SF-36 score.

Report

This report is used to visualize the main milestones and medical questionnaires of the patient, as well as the progression of the patient.

Patient report

Patient's report - 17/03/2023

Surgery information

Surgeon: Kees de Boer

Region: CHIRURGIE

Prevalent Therapy: CHIRURGIE

Hospital: Afd. Chirurgie

Responsible: Med. Adv. Therapie

Personal information

Patient number: 9700

Age: 57

Gender: Male

Surgeon date: 20/11/2022

Patient-reported outcomes


Score	Pre-op	1 W	3 W	6 W	1 Y	2 Y
Health-related QoL	60	55	55	55	55	55
Physical QoL	60	55	55	55	55	55
Psychological QoL	60	55	55	55	55	55
Health-related QoL	60	55	55	55	55	55
Physical QoL	60	55	55	55	55	55
Psychological QoL	60	55	55	55	55	55

Clinical milestones

Group (date of onset after surgery)	Patient	most of patients
Group 1 (date of onset after surgery)	100%	100%
Group 2 (date of onset after surgery)	100%	100%
Group 3 (date of onset after surgery)	100%	100%

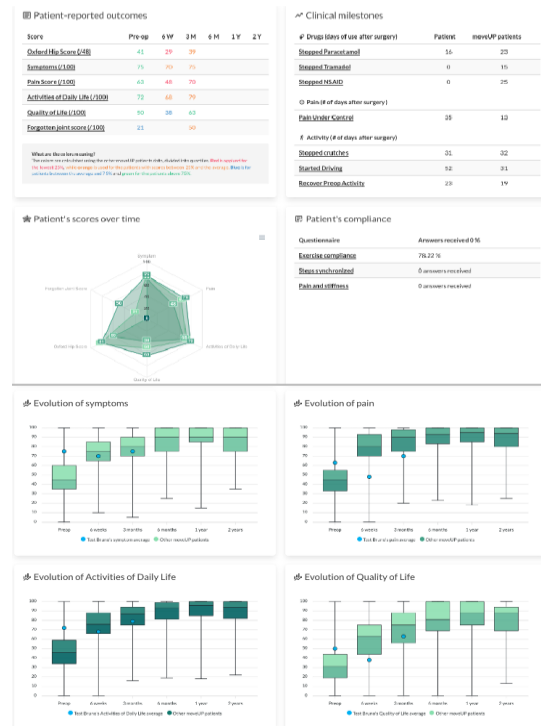The report can be downloaded in pdf form by clicking the 'Download report' button. This way the HCP can send the report to the patient. It's also possible to share the link directly to the report, using the button 'Copy report link'.

Profile

The HCP can view, edit or add all relevant basic, clinical, administrative or care plan specific data in the patient profile

Basic information

First Name *

firstname znetoao9sk

Surname *

lastname ix0mvl25fh

Messaging

The message popup (which can be enlarged) is closed by default and the number of new messages is visible in the floating button. This functionality is used to give advice to patients and answer their questions. It can also be used to ask for pictures, for example to follow-up on wound healing, swelling, redness etc...

Messages

11/01/2024

The Patient

11/01/2024

Messages 2

Push notification

When clicking 'messages', a set of prescribed personal messages is shown wherefrom the HCP can select an applicable message for that particular patient.

Suggestions

- Two crutches outside
- Availability hometrainer
- Two crutches
- Swelling tips
- Stiffness tips
- Night pain tips

Messages 12

Normal

Type a message

Click on the "1st line symbol" to mark the question as read.

1st line symbol

Messages

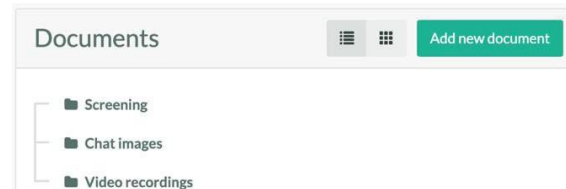
Messages that are submitted by a HCP can be edited or removed. When hovering over a message, a button to edit or delete the message will become visible.

Documents

Documents can be uploaded in the 'Documents' section. Chat images and video recordings are automatically collected in this section in view of easy access to these materials.


More

See which documents / contracts the patient has signed.


See on which device the patient is using the application and when the application was first and last seen.

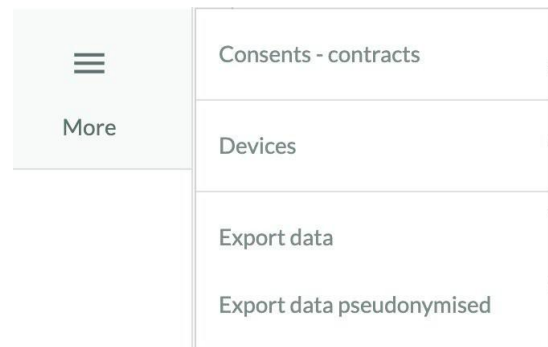Export data is only possible for limited HCPs due to security reasons and to protect data integrity.

3.3. Log out

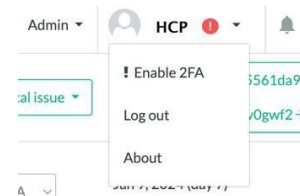In the upper right corner the HCP can log out. Know that the automatic log out time is 14 days, therefor it is recommended to log out more frequently to avoid unauthorized access.


3.4. Support

Reach out via support@moveup.care for questions or use the '?' button on the platform.

3.5. Care plan management

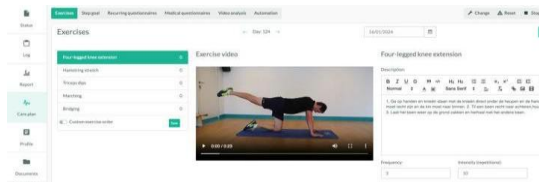In care plan management it is possible to navigate between multiple functions that are applicable to the indication of the patient, such as exercises, step goal, recurring questionnaires, video analysis, automation, etc... For more information see further.

4. Healthcare platform care plan functions

4.1. Exercise management

The exercise management module includes the following possibilities of adaptation:

- Adapt description, frequency, and intensity of the exercises of the day.
- Adapt the daily message.
- Adapt the exercise scheme:
 - Blue check marks indicate active exercises, accompanied with frequency and intensity of these active exercises.
 - Schedule exercises over a longer period of time.

Extend the standard exercise scheme with a set of exercises that are not part of the exercises defined in the care plan

The screenshot shows the 'Exercise' tab in the healthcare platform. It includes a video player for 'Exercise video', a 'Daily message' section, and a table of exercises. The table has columns for 'Exercise', 'Frequency', and 'Intensity'. A red box highlights the 'Daily message' section, and another red box highlights the 'Exercise' table.

The screenshot shows the 'Manage patient scheme' interface. It includes a 'Select one or more labels' dropdown, a 'View update history' button, and a table of exercises. The table has columns for 'Exercise', 'Frequency', and 'Intensity'. A red box highlights the 'Exercise' table.

The screenshot shows the 'Plan exercise' dialog box. It includes fields for 'Start day / End day', 'Repeat', 'Intensity start (repetitions)', 'Intensity target (repetitions)', 'Frequency start', and 'Frequency target'. A red box highlights the 'Plan exercise' dialog box.

4.2. Recurring questionnaire

Recurring questionnaires can be removed, reordered or a new recurring questionnaire can be added. The default list of recurring questionnaires, including their start day, end day, frequency and reminders is predetermined in the care plan.

The screenshot shows the 'Recurring questionnaire' interface. It includes a 'Questionnaires' section with a 'Personalize' button, a table of questionnaires, and a dialog box to add a new questionnaire. The dialog box has fields for 'Questionnaire', 'Start date', 'End date', 'Frequency', 'Activation hour', and 'Activation minutes'. A red box highlights the 'Personalize' button, and another red box highlights the 'Add questionnaire to personalized care plan' dialog box.

4.3. Medical questionnaire

Medical questionnaires can also be removed or added to the default scheme. The default list of medical questionnaires, including their start and end day is predetermined in the care plan.

The screenshot shows the 'Medical questionnaires' tab in the software interface. It displays a table titled 'Medical scores scheme' with columns for Name, Start day, and End day. The table lists three questionnaires: 'HPP PROM collection month 6', 'HPP PROM collection Year 1', and 'Discharge Questionnaire HPP'. Each row has 'Preview' and 'Remove' buttons. Below the table is a dialog box 'Add medical questionnaire to care plan' with a dropdown for 'Medical score', a dropdown for 'Activation type', and input fields for 'Start Date' and 'End Date'. There are 'Save' and 'Cancel' buttons at the bottom of the dialog.

4.4. Video analysis

With the use of video requests, it is possible to gain more objective information, for example about the gait analysis or range of motion of an operated joint. The default list of video requests is predetermined in the care plan. A video request can be added directly by using the 'Add request' button or a video request can be scheduled for a longer period of time by using the 'Add' button and by adding a start and end day to put the video request active.

The screenshot shows the 'Video analysis' tab. It includes a section for 'Video analysis' with checkboxes for 'Analysis enabled for this patient' and 'Results NOT shown to this patient', and an 'Add request' button. Below is a table titled 'Video requests' with columns for Name, Start day, and End day, listing three 'Gait analysis' requests. To the right of the table are 'Add', 'Remove', 'Messages', and 'Reschedule' buttons. Below the table are two dialog boxes. The first, 'Add video request to personalized care plan', has a dropdown for 'Video request', input fields for 'Start date' and 'End date', and 'Save'/'Cancel' buttons. The second, 'Request the patient to record a video for analysis', has a dropdown for 'Video type' and 'Save'/'Cancel' buttons.

4.5. Goal manager

The HCP can enable and disable a behavioral change tool which enables patients to set their personal goal, define barriers and solutions in order to reach a healthy lifestyle. The HCP can see the progress on the patient's goals that are defined by the patient.

The screenshot shows the 'Goal manager' interface. On the left is a sidebar with 'Log', 'Report', and 'Care plan' options. The main area is titled 'Activities' and lists three activities: 'Jogging In the park', 'Exercises (fitness, agility, etc.) at work In de polyvalente zaal', and 'Running In the forest'. A 'Select all' button is in the top right. Below the list is a detailed view for 'Jogging In the park'. This view shows progress for 'Selected month' and 'Selected week', a 'Planned' section with a calendar, 'Patient's notifications' with a timeline, and a 'Start date of this activity'.

4.6. Change of care plan

The care plan can be adapted based on the patient's interventions, needs and wishes. This can be done via the onboarding tool or via the healthcare platform via the button 'Advanced'. Instructions how to change a care plan are clearly indicated within the tools.

The first screenshot shows a modal titled "Add care plan?". It contains the text: "Starting a new care plan will end the active follow-up of the current care plan." Below this text are two buttons: "Cancel" and "End current, start new".

The second screenshot shows the "Edit care plans for Patient" interface. It displays a list of care plans for a patient named "Kees-101: TKA General Therapy". The first care plan is "Intervention: Knee left: total" with a date of "24-01-2024", doctor "Jeroen Surgeon", and status "Active". The second care plan is "Treatment: TKA General Therapy" with a start date of "14-01-2024", code "10-04010", and HCP "Jeroen Surgeon". Below the list, there is a note: "Jeroen Surgeon as surgeon is part of the care team".

4.7. Stop active follow up

When the active follow-up is finished (what falls under active follow-up is predetermined in the care plan), press 'Stop active follow-up'. This will redirect the patient to a passive long term follow-up if applicable (also predetermined in the care plan if long term follow-up is needed).

The first screenshot shows the "Stop active follow up" modal. It has a header with "Advanced", "Stop active follow up", and "Complete" buttons. The main text says: "Stopping the active follow up will move the patient to the PROM stage. Personalisation of exercises after stop active follow up can be done in the tab Care plan. If exercises are not personalised, the patient will receive an overview with all exercises performed during his/her rehab." Below this is a "Message" section with a text area and a "Yes" button. There is also an "Email" section with a "Yes" button and a note "No emails configured". At the bottom are "Stop" and "Cancel" buttons.

The second screenshot shows the "Stop treatment?" modal. It has a header with "Advanced", "Back to active follow up", and "Complete" buttons. The main text asks: "Are you sure you want to stop the treatment? Stopping a treatment will mark the patient as being finished. Medical scores in the treatment will no longer be activated." Below this are "Cancel" and "Stop" buttons.

When no actions are required anymore from the HCP or the patient, the patient can be redirected to the 'Completed' status.

4. WELLNESS MODULE

The wellness module is an optional non-medical device module and therefore the data shown is not intended to be used for clinical decision making.

This data gives an overview of the general wellbeing of the patient.

The wellness module by default is blurred and needs to be activated to check the patient's wellness data.

1. WELLNESS MODULE ACTIVATION

To activate the wellness module you need to enter your profile settings and activate the option:

Enable wellness data

Upon activation, a mandatory acknowledgment pop-up is displayed. It informs you that the data from the Wellness Module must not be used to support clinical decision-making.

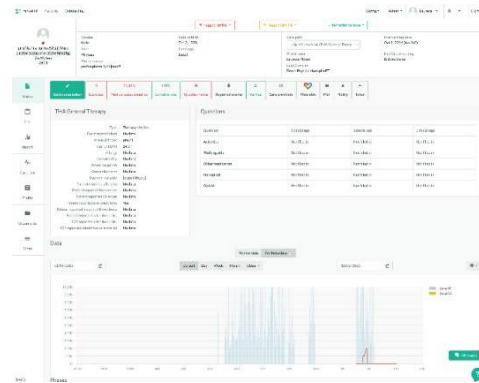
Once acknowledged, you gain access to the wellness data. This activation allows you to view the wellness data and export the wellness reports.

2. WELLNESS MODULE OVERVIEW


Wellness module overview

In this tab you'll be able to see the step data from the patient and see their evolution.

Wellness report

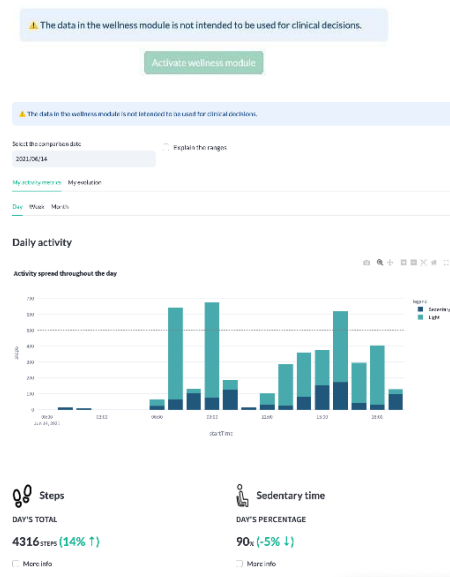You can see the wellness reports when needed.

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The screenshot shows a table with columns: Name, Patient number, Hospital, Date of birth, Intervention, Lead clinician, and Surgeon. The table lists several patients with their respective details. A filter sidebar is visible on the left, and a 'Showing 60 out of 400 results' message is at the top.

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The 'Tags' interface includes a 'Select tags' dropdown, a 'Select all' checkbox, a 'Search' input field, and a 'Bariatrics' checkbox.

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The 'Filter patient list' dialog box contains filters for 'Activated' (Yes), 'Language' (N/A), 'Test patient' (No), and 'Lead clinician' (N/A). It includes 'Apply' and 'Cancel' buttons at the bottom.

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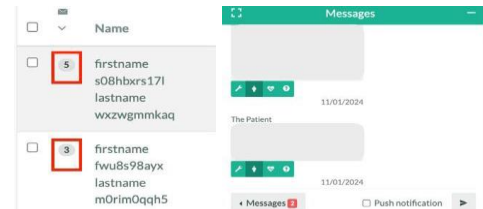
-1	Patient was checked and validated 1 day ago	11/01/2024
✓	Patient was checked and validated today by your discipline	11/01/2024
✓	Patient was checked and validated today by other discipline	11/01/2024
⚠	Patient was flagged for your discipline	11/01/2024
⚠	Patient was flagged for another discipline	11/01/2024

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Type: MEDICAL

Showing 1 out of 2 total log messages.

Date	Type	Log	User
14/01/2024, 14:43	MEDICAL	This patient has an inflammation	

Patient log

Care plan: ALL

Type: MEDICAL

Showing 2 out of 404 total log messages.

Date	Type	Log
20/09/2023, 02:51	FLAG (red line)	TRIGGER: The patient provided an improvement suggestion in medical room. Satisfaction @1 week.
15/04/2023, 14:36	FLAG (red line)	TRIGGER: The patient contacted the medical team today.

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Patient's report - 17/03/2023

Surgery information

Surgery type	Prostatectomy
Surgeon	Dr. John Doe
Procedure	Prostatectomy
Location	St. Mary's Hospital

Personal information

Patient name	John Doe
Age	65
Gender	Male
Surgeon	Dr. John Doe

Postoperative outcomes

Score	Pre-op	Post-op	1st	2nd	3rd	4th	5th	6th	7th	8th	9th	10th
Overall Satisfaction	10	10	10	10	10	10	10	10	10	10	10	10
Postoperative Pain	10	10	10	10	10	10	10	10	10	10	10	10
Postoperative Urinary Control	10	10	10	10	10	10	10	10	10	10	10	10
Postoperative Sexual Function	10	10	10	10	10	10	10	10	10	10	10	10
Postoperative Bowel Function	10	10	10	10	10	10	10	10	10	10	10	10
Postoperative Quality of Life	10	10	10	10	10	10	10	10	10	10	10	10

Clinical information

1st Postoperative Visit	10
2nd Postoperative Visit	10
3rd Postoperative Visit	10
4th Postoperative Visit	10
5th Postoperative Visit	10
6th Postoperative Visit	10
7th Postoperative Visit	10
8th Postoperative Visit	10
9th Postoperative Visit	10
10th Postoperative Visit	10

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The screenshot displays the moveUP platform interface for managing patient exercise plans. The top section shows a patient's exercise plan with a video of a person performing a glute contraction exercise. The bottom section shows a 'Manage patient scheme' table with columns for exercises and their frequency/intensity. A 'Plan exercise' modal is open, allowing users to adjust the start/end day, repeat frequency, intensity, and frequency for a specific exercise.

Exercise	Frequency	Intensity
Glutes contraction	3x 10 repetitions	10
Legs hip	3x 10 repetitions	10
Shoulder	3x 10 repetitions	10
Glutes contraction and extension of the knee	3x 10 repetitions	10

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The screenshot shows a user profile editing page for 'Laurent Baes'. Under the 'Activation' section, the 'Wellness module' checkbox is checked, while 'Medical data' is unchecked. The 'Save' button is highlighted in green.

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The screenshot shows a 'Platform notice' dialog box titled 'Wellness module terms of use'. It contains the following text:

Wellness module terms of use

Dear User,
You are about to activate the Wellness Module.

What is the Wellness Module?

The wellness module provides you with additional data about the general wellbeing and activity levels of your patients. The data in the Wellness Module can come from:

- imported data from general-purpose devices such as e.g. activity trackers
- manual entry by patients

This data does not come from medical devices. This means that neither the devices from which the data is imported, nor the manual entries by the patients have been verified or validated by moveUP, and therefore, it should not be used for clinical decision-making.

What is the purpose of this data?

We provide the option to see wellness data to give you a broader idea about the general wellbeing of the patients and to give you the opportunity to discuss and give further guidance regarding wellbeing.

This module is an optional module and is not linked to the medical data presented in the "Medical data" tab of the dashboard.

The data in the wellness module is not intended to be used for clinical decisions.

How to recognize the wellness data?

Wellness data is displayed in a separate tab in the dashboard or in a separate report.

Whenever wellness data is displayed in a column, together with other data it has a grey background to visually indicate that this does not concern medical data. Furthermore, it has a warning sign next to it.

Enabling/Disabling the Wellness Module

At any time, a user can enable/disable the Wellness Module in the user settings.

Acknowledgement

By clicking on the button below, you acknowledge that you have read and understood that the Wellness Module is an optional module that provides you with additional data regarding the general wellbeing of the patient. This data is not validated by moveUP and is not intended for clinical decision making.

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New patient

Patient's care plans

Start a care plan to add intervention and treatment for this patient

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The screenshot shows a table with columns: Name, Patient number, Hospital, Date of birth, Intervention, Lead clinician, and Surgery. The table lists several patients with their respective details. A filter sidebar is visible on the left, and a 'Select all' button is at the bottom.

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The 'Tags' interface shows a 'Select tags' dropdown menu. Below it, there is a 'Select all' checkbox and a 'Search' input field. A 'Bariatrics' tag is shown as selected.

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The 'Filter patient list' dialog box contains filters for 'Activated' (Yes), 'Language' (N/A), 'Test patient' (No), and 'Lead clinician' (N/A). It has 'Apply' and 'Cancel' buttons at the bottom.

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