


PROSTAGRIP™

Transurethral Prostatic Manipulator Device

The only transurethral access prostatic
manipulator that enables dissection
during minimally invasive
prostatectomy procedures

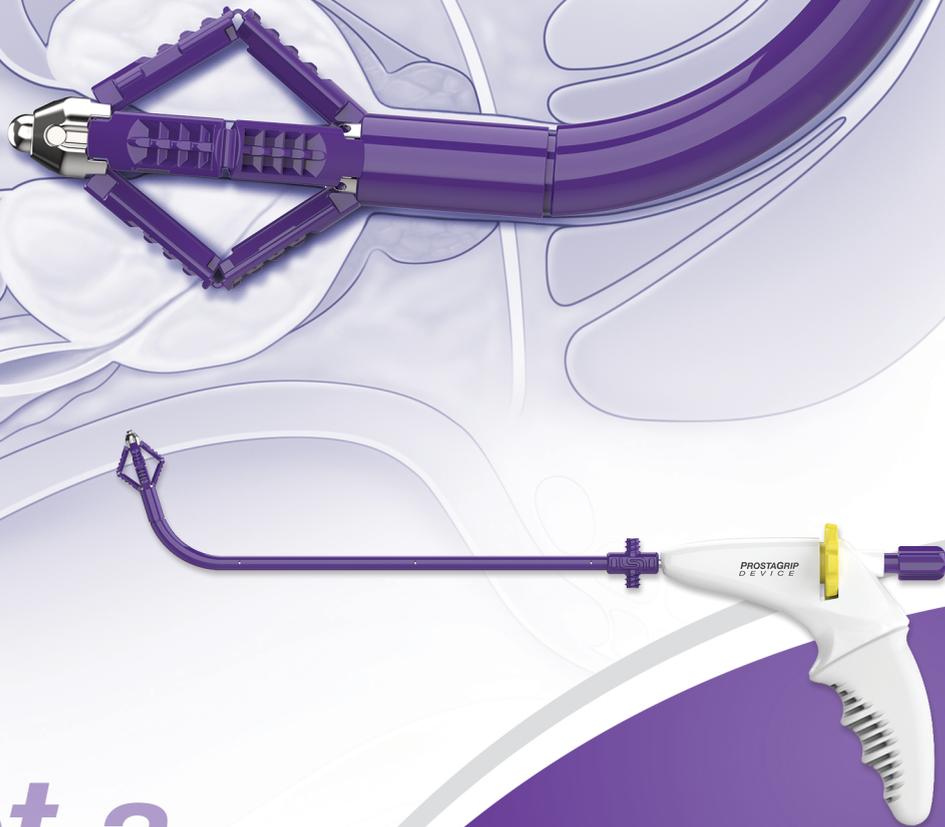


Get a
GRIP™

LS SOLUTIONS®

PROSTAGRIP™

Transurethral Prostatic Manipulator Device

The only transurethral access prostatic manipulator that enables dissection during minimally invasive prostatectomy procedures

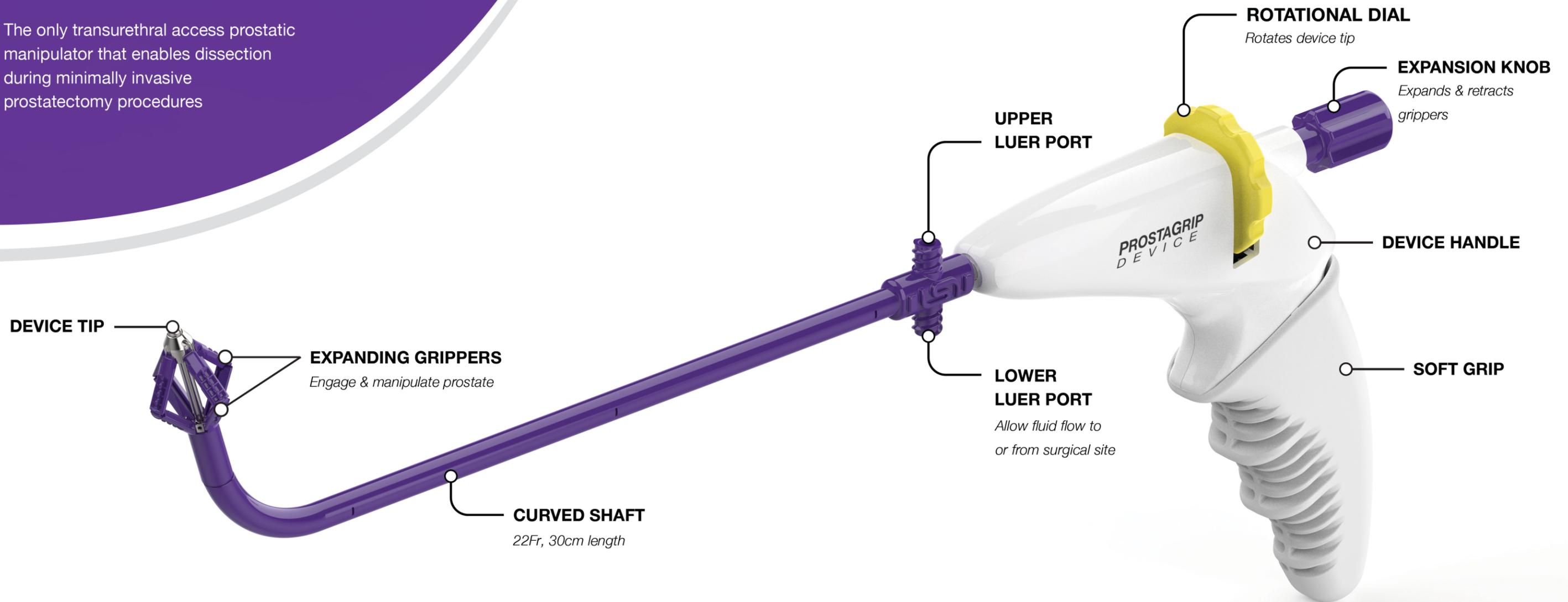

The **PROSTAGRIP™** Device facilitates the manipulation of the prostate via transurethral access during minimally invasive prostatectomy. The patient end of the device has a rotating tip that contains 4 radially expandable grippers to engage and manipulate the prostate. The surgeon's end of the device has a soft grip handle with an expansion knob, which expands the 4 grippers on the device tip, and a rotational dial, which rotates the tip of the device about the shaft for precise gripper positioning.

Upper and lower luer ports allow for bladder drainage or infusion of sterile fluid into the surgical site, as necessary. When the **PROSTAGRIP™** Device is appropriately inserted into the urethra and the prostate is engaged, the device allows the surgeon to ergonomically lift, angle, pull, and push the prostate to facilitate dissection.



PROSTAGRIP™ PRODUCT ORDERING			SUPPLIED: STERILE
	REORDER	PRODUCT	DESCRIPTION
 x 6	REF 081358	PROSTAGRIP™ DEVICE	Box of (6) Devices

PN081627B 01.08.26

LSI SOLUTIONS®

7796 VICTOR-MENDON RD | VICTOR, NY 14564 USA | CUSTOMER SERVICE: 866.575.3493 | TECH SUPPORT: 866.428.9092 | FAX: 585.742.8086 | WWW.LSISOLUTIONS.COM

The LSI logo, LSI SOLUTIONS, Prostagrip, and Get a Grip are trademarks and registered trademarks of LSI SOLUTIONS, Inc. Copyright © 2024, LSI SOLUTIONS®. All Rights Reserved.