



DRUG PRECURSOR TESTING KITS

Procedure Card for IDT0290

APAAN IDENTIFICATION

Substance Test




Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1

Remove the Smart-Tip™ probe from test unit.

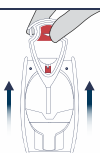

2

Sample suspected substance.


Substance

Touch Smart-Tip™ probe to sample.



3

Lock Smart-Tip™ probe into test unit. Press firmly.

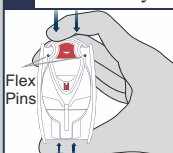


STOP

Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Identification

4

With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

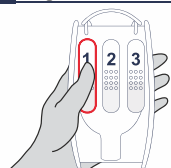

5


Gently shake test unit for 2-3 seconds.



Identification


6

With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

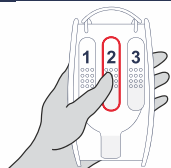

7


Gently shake test unit for 2-3 seconds.

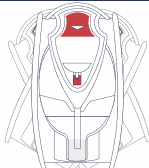

Identification


8

With **thumb and forefinger**, squeeze chamber 3 until capsule breaks.

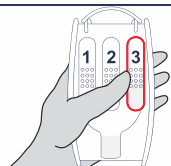

9

Tilt test unit sideways allowing the solution to flow into **chamber 3**. Then gently shake test unit for 2-3 seconds.

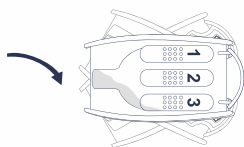

Results



The appearance of a red-purple (wine) color indicates the presence of **APAAN** (α -Phenylacetoacetonitrile, 2-Phenylacetoacetonitrile).



If no red-purple (wine) color develops, APPAN is **not** present.



Disposal

The test kit is safe to dispose of in the trash.

01-19