



Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the probe from test unit.	2	Sample suspected substance or surface. Touch or Swipe	3	Secure probe into test unit. Press firmly.	Do not continue unless probe is secured properly in place.

Chamber 1 Identification	4	From both sides of the kit, squeeze on the circle of Chamber 1 until the capsule breaks.	5	Gently shake test unit for 2-3 seconds.

Chamber 1 Results		If a brown powder is present, the appearance of a purple-red color strongly indicates the presence of Brown Heroin . If a brown powder is not present, the appearance of a purple-red color indicates either the presence of basic alkaloid substance (See Results below).
		The appearance of a green or blue-green color indicates the presence of Crack .
		If any of the above colors develop, proceed to Disposal. If none of the above colors develop, proceed to Step 6.

Chamber 2 Identification	6	From both sides of the kit, squeeze on the circle of Chamber 2 until the capsule breaks.	7	Gently shake test unit for 2-3 seconds.

Chamber 2 Results		The appearance of a purple-red color indicates the presence of one of these suspected drugs: Heroin, Amphetamine, Methamphetamine, Ecstasy/MDMA, Buphedrone, Cathinone, DMT, Ketamine, Mephedrone, Methadone, MethCathinone, Methylone, mCPP, MDMAI, MDPBP, MDPV, MPA, PCP, Pentylone, PMA, PMMA, 2-aminoindane (2-AI), 3-Methylbutyrfentanyl, 3-Methylfentanyl, 4-Fluorobutyrfentanyl, 4-Fluoroisobutyrfentanyl, 4-Methoxybutyrfentanyl, other synthetic/designer drugs.
		The appearance of a green or blue-green color indicates the presence of Cocaine .
		If none of the above colors develop, none of the suspected drugs listed above are present.



Disposal

The test kit is safe to dispose of in the trash.