

## **Procedure Card for X1-1000**

## COCAINE/CRACK IDENTIFICATION

**Substance & Trace Test** 



As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.



## 2 Sample suspected substance or surface. Touch or Swipe

Touch probe to the suspected substance.

Swipe either side of the probe across the suspect surface.

Secure probe into test unit. Press firmly.



Do not continue unless probe is secured properly in place.

Chamber 1 dentification From both sides of the kit, squeeze on the circle of **Chamber 1** until the capsule breaks.



5

Gently shake test unit for 2-3 seconds.



Chamber 1 Results



The appearance of turquoise color particles adhering to the sampler tip or precipitating out of solution indicates the presence of **Cocaine**.

\* Color may disappear with time. Note: The liquid may remain pink.



If a turquoise color develops, proceed to Disposal. If no turquoise color develops, proceed to Step 6.

Chamber 2 dentification

6

From both sides of the kit, squeeze on the circle of **Chamber 2** until the capsule breaks.



7

Gently shake test unit for 2-3 seconds.



namber 2 Results



The appearance of turquoise color particles adhering to the sampler tip or precipitating out of solution indicates the presence of **Crack (Cocaine Base)**.

\* Color may disappear with time. Note: The liquid may remain pink.



If no turquoise color develops, neither Cocaine nor Crack are present



Disposal

The test kit is safe to dispose of in the trash.



Manufactured by IDenta Ltd

www.identa-corp.com
Patented Technology • Made in Israel