

MARIJUANA/HASHISH IDENTIFICATION

Substance Test



Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the probe from test unit.	2	Sample suspected substance. Touch Touch the probe to the suspect substance. Use 10-15 particles of plant material. (Too much material will cause a radical intense color change, making identification difficult.)	3	Secure probe into test unit. Press firmly.	<p>Do not continue unless probe is secured properly in place.</p>

Identification	4	From both sides of the kit, squeeze on the circle of Chamber 1 until the capsule breaks.	5	Gently shake test unit for 1 minute.

Identification	6	From both sides of the kit, squeeze on the circle of Chamber 2 until the capsule breaks.	7	Gently shake test unit for 10 seconds.
			<p>Wait up to 3 minutes for color to fully develop. The color may develop sooner.</p>	

Results	<p>If a blue-violet or purple color develops, proceed to step 8. If no blue-violet color develops, neither Marijuana nor Hashish are present. Proceed to Disposal.</p>
---------	---

Confirmation	8	From both sides of the kit, squeeze on the circle of Chamber 3 until the capsule breaks.	9	Vigorously shake test unit for 20 seconds.
			<p>Hold test unit steady and keep upright, until 2 layers are visibly separated. This may take up to 2 minutes.</p>	

Confirmation Results	<p>The appearance of a color in the lower layer of liquid strongly indicates the presence of Marijuana or Hashish.</p>
	<p>If the lower layer of liquid does not turn a color, neither Marijuana nor Hashish are present.</p>



Disposal

The test kit is safe to dispose of in the trash.

01/19