



#### Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

This test is designed to enhance positive results from **Amphetamine/Methamphetamine Identification Kit (X1-5000)**. Only proceed to use this test after obtaining an orange result from the Identification Kit.

Test Preparation	1	Remove the probe from test unit.	2	Sample suspected substance.			
		Touch		or	Swipe	or	Dig
		Touch probe to the suspected substance.		Touch probe to the suspected substance.	Swipe either side of the probe across the suspected substance.	Firmly press probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance.	
Test Preparation	3	Secure probe into test unit. Press firmly.	Confirmation	4	From both sides of the kit, squeeze on the circle of <b>Chamber 1</b> until the capsule breaks.	5	Gently shake test unit for 2-3 seconds.
		<b>STOP</b> Do not continue unless probe is secured properly in place.					
Confirmation	6	From both sides of the kit, squeeze on the circle of <b>Chamber 2</b> until the capsule breaks.	7	Do <b>NOT</b> shake or agitate test unit.			
Results	<p>The appearance of a blue color indicates the presence of <b>Methamphetamine</b>, if a positive result was received with Amphetamine/Methamphetamine Identification Kit (X1-5000).</p>						
	<p>If after a minute no blue color has appeared or a pink color appears, <b>and</b> if a <b>positive</b> result was received with Amphetamine/Methamphetamine Identification Kit (X1-5000), the presence of <b>Amphetamine</b> is indicated.</p>						



#### Disposal

The test kit is safe to dispose of in the trash.

01-19