

LSD IDENTIFICATION

Substance Test



Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1 Remove the probe from test unit.	2 Sample suspected substance.	
		<div>Roll</div> <div>or</div> <div>Dig</div> <p>Roll up the Blotter (Stamp) and insert it into the hole.</p>	<p>Firmly press probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance.</p>

Test Preparation	3 Secure probe into test unit. Press firmly.	<div>STOP</div> <p>Do not continue unless probe is secured properly in place.</p>	Identification	4 From both sides of the kit, squeeze on the circle of Chamber 1 until the capsule breaks.	5 Gently shake test unit for 20 seconds.

Identification	6 From both sides of the kit, squeeze on the circle of Chamber 2 until the capsule breaks.	7 Gently shake test unit for 2-3 seconds.

Results



The appearance of a purple-violet color indicates the presence of **LSD** (lysergic acid diethylamide) or **DMT** (dimethyltryptamine).



If no violet-purple color develops, LSD and DMT are **not** present.



Disposal

The test kit is safe to dispose of in the trash.

01-19