



Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1

Remove the probe from test unit.



2

Sample suspected substance.

Dip

Place the absorbent paper of the probe into the liquid. Make sure the paper is wet.



3 Secure probe into test unit. Press firmly.




STOP

Do not continue unless probe is secured properly in place.

Test Preparation

3

Secure probe into test unit. Press firmly.

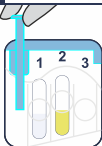

STOP

Do not continue unless probe is secured properly in place.

Identification

4

From both sides of the kit, squeeze on the circle of **Chamber 1** until the capsule breaks.



5


Gently shake test unit for 2-3 seconds.



Identification


6

From both sides of the kit, squeeze on the circle of **Chamber 2** until the capsule breaks.

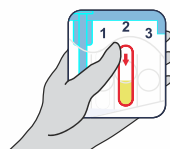

7

Gently shake test unit for 2-3 seconds.

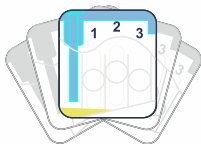

Results



The appearance of a light orange color indicates the presence of **GBL** (γ -Butyrolactone).



If a light orange color does not develop or the liquid remains a yellow color, GBL is not present.



Disposal

The test kit is safe to dispose of in the trash.

0280T