

MORPHINE IDENTIFICATION

Substance Test



Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the probe from test unit.	2	Sample suspected substance. Dig	3	Secure probe into test unit. Press firmly.	<p>Do not continue unless probe is secured properly in place.</p>
			<p>Firmly press probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance.</p>				

Identification	4	From both sides of the kit, squeeze on the circle of Chamber 1 until the capsule breaks.	5	Gently shake test unit for 2-3 seconds.

Results		If the substance is a tablet, the appearance of a purple color indicates the presence of an opiate such as Morphine or Codeine .
		If no purple color develops, an opiate, such as Morphine or Codeine, is not present.
		Proceed to Dilution (steps 6-9), regardless of whether or not a color appeared.

Dilution	6	From both sides of the kit, squeeze on the circle of Chamber 2 until the capsule breaks.	7	Gently shake test unit for 2-3 seconds.
			<p>Wait 5 minutes. The test window will warm up. This is normal.</p>	

Dilution	8	From both sides of the kit, squeeze on the circle of Chamber 3 until the capsule breaks.	9	Gently shake test unit for 2-3 seconds.
			<p>The test window will warm up. This is normal.</p>	



Disposal

After Dilution, the test kit is safe to dispose of in the trash.

01-19