

PARACETAMOL IDENTIFICATION

Substance Test




**Read
Before Use**

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1 Remove the probe from test unit.

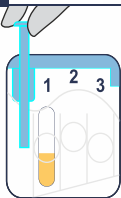

2 Sample suspected substance.

Touch or Swipe


Touch probe to the suspected substance.



Swipe either side of the probe across the suspected substance.




3 Secure probe into test unit. Press firmly.

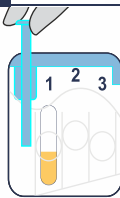


Do not continue unless probe is secured properly in place.

Identification

4 From both sides of the kit, squeeze on the circle of **Chamber 1** until the capsule breaks.

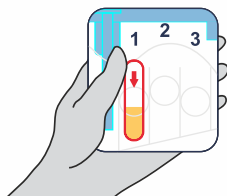

5 Gently shake test unit for 2-3 seconds.

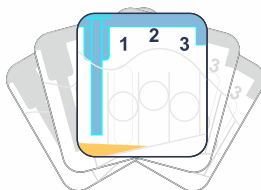

Results



The appearance of a blue color indicates the presence of **Paracetamol**.



If no blue color develops, Paracetamol is not present.



Disposal

The test kit is safe to dispose of in the trash.

01-19