

METHADONE IDENTIFICATION

Substance Test




Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1

Remove the probe from test unit.

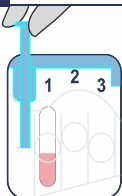

2

Sample suspected substance.

Touch or Swipe

Touch probe to the suspected substance.




Swipe either side of the probe across the suspected substance.



3

Secure probe into test unit. Press firmly.

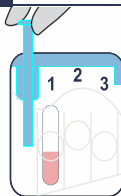


Do not continue unless probe is secured properly in place.

Identification


4

From both sides of the kit, squeeze on the circle of **Chamber 1** until the capsule breaks.

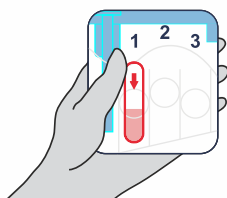

5

Gently shake test unit for 2-3 seconds.

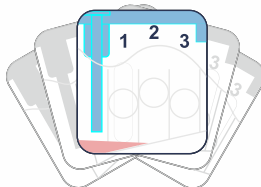

Results



The appearance of turquoise color particles adhering to the sampler tip or precipitating out of solution indicates the presence of **Methadone**.

** Note: Color may disappear with time. The liquid will remain pink.*



If no turquoise color develops, Methadone is not present.



Disposal

The test kit is safe to dispose of in the trash.

01-19