



**Read  
Before  
Use**

As a precaution, gloves and eye protection should be worn when handling substances and using test units. If a spill occurs, thoroughly rinse affected area with water. Keep out of reach of children. Store the bottle in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

| Step                       | Unit                                                                              | Illustration                                                                      | Directions                                                                                                                            |
|----------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------|
| <b>Sample<br/>Material</b> |  |  | <p><b>Remove a sheet of test paper from the pad.</b></p> <p><b>Wipe the test paper across the suspected substance or surface.</b></p> |

|                                                      |                                                                                    |                                                                                    |                                                                                                                    |
|------------------------------------------------------|------------------------------------------------------------------------------------|------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|
| <b>Testing<br/>Procedure<br/>(Turquoise<br/>Cap)</b> |  |  | <p><b>Open and remove the cap from the Turquoise Cap bottle.</b></p> <p><b>Apply a drop to the test paper.</b></p> |
|------------------------------------------------------|------------------------------------------------------------------------------------|------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|

|                | Color                                                                               | Description      | Substance Indicated      |
|----------------|-------------------------------------------------------------------------------------|------------------|--------------------------|
| <b>RESULTS</b> |  | <b>Turquoise</b> | <b>Indicates Cocaine</b> |

01-12