



**Read
Before
Use**

As a precaution, gloves and eye protection should be worn when handling substances and using test units. If a spill occurs, thoroughly rinse affected area with water. Keep out of reach of children. Store the bottle in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Step	Unit	Illustration	Directions
Sample Material			Remove a sheet of test paper from the pad. Wipe the test paper across the suspected substance or surface.

Testing Procedure (Orange Cap)	CAUTION: The reagent in the bottle contains an acid. Avoid skin contact with the reagent.		
			Open and remove the cap from the Orange Cap bottle. Apply a drop to the test paper.

	Color	Description	Substance Indicated
RESULTS		Orange-Red	Indicates Amphetamine or Methamphetamine, if color appears within 1 minute
		Purple	Indicates an Opiate such as, Heroin, Morphine or Codeine
		Dark Purple to Black	Indicates Ecstasy/MDMA