



Cocaine Multi-Use Tests

DR-1000

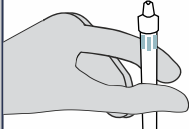
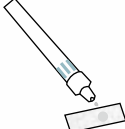


Read Before Use

As a precaution, gloves and eye protection should be worn when handling substances and using test units. If a spill occurs, thoroughly rinse affected area with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

This test consists of one tube. Since the tube contains enough liquid to test many samples, do not dispose of the tube until all desired samples are tested. For additional tests with the same tube, take a new sheet of test paper from the pad to test the next sample.

Test Preparation	Substance		Traces	
	1	2	Traces	
	Remove a sheet of test paper from the pad.	Touch the test paper on the suspected substance.	Wipe the test paper across the suspected surface.	
				

Identification	3		4	
	With thumb and forefinger , squeeze the test tube until the ampule breaks.		Apply a drop to the test paper.	
				

Results	The appearance of turquoise color particles indicates the presence of Cocaine .	
		
	If no turquoise color develops, Cocaine is not present.	
		



Disposal

When no further testing is desired, the tube is safe to dispose of in the trash.

01-19



Manufactured by IDenta Ltd * Made in Israel
www.identa-corp.com * Patented Technology

LOT:

EXP: